

DENTAL LIGHT

CLESTA LED DENTAL LIGHT

TYPE

301 Pole Mount

302 Ceiling Mount

305 Track Mount

320X Unit Mount

Parts List



 **Belmont[®]**

Table of Contents

1. Head Section

1-1. Head (301/ 302 / 305 / 320X) - - - - -	1
---	---

2. Arm Section

2-1. Arm (301) - - - - -	3
2-2. Arm (302) - - - - -	4
2-3. Arm (305) - - - - -	5
2-4. Arm (320X / 320XN / 320XC) - - - - -	6

3. Power Supply Box Section

3-1. Power Supply Box (301) - - - - -	7
3-2. Power Supply Box (302) - - - - -	9
3-3. Power Supply Box (305) - - - - -	11
3-4. Power Supply Box (320X / 320XN / 320XC) - - - - -	13

4. Switch Box Section

4-1. Switch Box (320X / 320XC) - - - - -	15
4-2. Switch Box (320XN) - - - - -	16

5. Ceiling Pole Section


5-1. Ceiling Pole (302) - - - - -	17
-----------------------------------	----

6. Track Section

6-1. Track (305) - - - - -	19
----------------------------	----

1. Head Section

1-1. Head (301 / 302 / 305 / 320X)

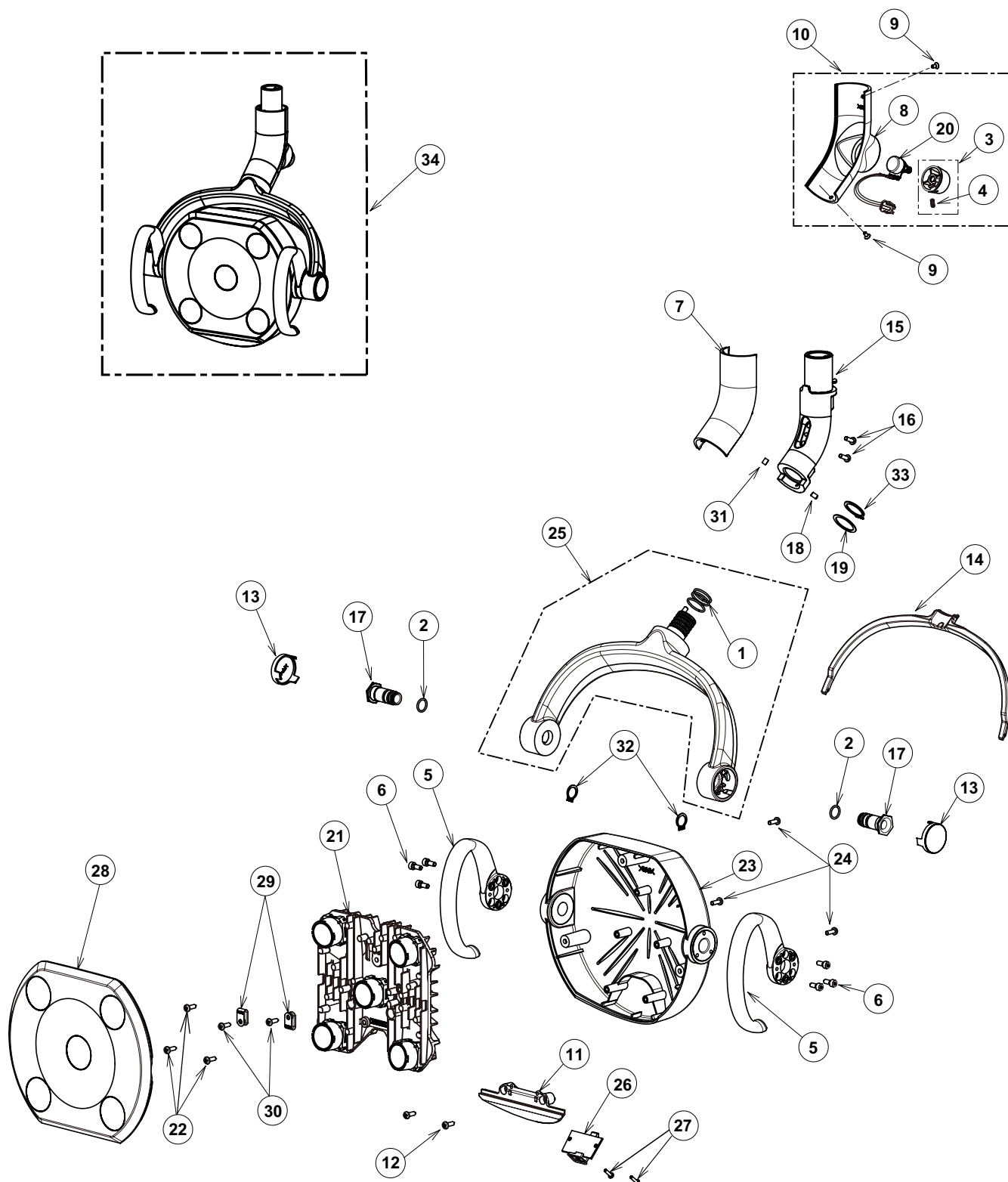


1. Head Section

1-1. Head (301 / 302 / 305 / 320X)

ID No.	Manufacture ID	Parts No.	Description	Qty
1	G09	FJMA83A0	O-Ring S18	3
2	G09	1E03JKA0	O-Ring S11.2	2
3	G09	1P0419A0	Volume Knob w/ Set Screw (CN-80)	1
4	G09	ZA774JNG	Set Screw M3 x 6 Black (Cup Point) (for Volume Knob)	1
5	G09	1A07L6B0	Handle (CN-80)	2
6	G09	ZA761LPU	Cap Screw M4 x 8 SUS for Handle	6
7	G09	1A0B9BA0	Yoke Front Cover (RAL9003) w White Color	1
	G09	1A0HVKA0	Yoke Front Cover (CN-80) w Light Grey Color	
8	G09	1A0B9CB0	Yoke Rear Cover (RAL9003) w White Color	1
	G09	1A0HVLA0	Yoke Rear Cover (CN-80) w Light Grey Color	
9	G09	ZA112JNY	Flat Head Screw M3 x 6 for York Rear Cover	2
10	G09	1G03JAA0	Yoke Rear Cover Assembly (RAL9003) w White Color (w/Volume & Knob)	1
	G09	1G059EA0	Yoke Rear Cover Assembly (CN-80) w Light Grey Color (w/Volume & Knob)	
11	G09	1A0B9EA0	Sensor Cover	1
12	G09	AFFX58A0	Pan Head Tapping Screw M4 x 10 for Sensor Cover	2
13	G09	1A0B9FA0	Yoke Cap (CN80)	2
14	G09	1A0B9GB0	Yoke Packing (CN80)	1
15	G09	1G03S8A0	Yoke Shaft Assembly	1
16	G09	AFFX58A0	Pan Head Tapping Screw M4 x 10 for Yoke Front Cover	2
17	G09	1A0B9MA0	Head Rotation Shaft	2
18	G09	ZA771LNU	Set Screw M4 x 6 SUS (Flat Point)	1
19	G09	1A0B9RA0	Yoke Washer	1
20	G09	1A0BKFA0	Intensity Control Volume assy (w/Harness)	1
21	G09	1G03G7B0	LED Module (5 LEDs)	1
22	G09	AFFX58A0	Pan Head Tapping Screw M4 x 10 for LED Module	3
23	G09	1G03G8A0	Frame Cover (RAL9003) w White Color	1
	G09	1G058XA0	Frame Cover (CN80) w Light Grey Color	
24	G09	AFFX58A0	Pan Head Tapping Screw M4 x 10 for Lens Cover	3
25	G09	1G03SAA0	Yoke Assembly (RAL9003) w White Color	1
	G09	1G05BPA0	Yoke Assembly (CN80) w Light Grey Color	
26	G09	1G03GLA0	Sensor PCB (w/Sensor Rubber Cover)	1
27	G09	CECN46A0	Delta PT Screw 3 x 8 for Sensor PCB	2
28	G09	1G03L0A0	Lens Cover (RAL9003) w White Color	1
	G09	1G058ZA0	Lens Cover (CN80) w Light Grey Color	
29	G09	APNA19A0	Nylon Clamp NK-2N	2
30	G09	1E00AAA0	Binding Head Screw M4 x 8 for Nylon Clamp	2
31	G09	DATT28B0	Brake Rubber	1
32	G09	ZG041RAG	C-Ring M12 Black	2
33	G09	ZG041VAG	C-Ring M20 Black	1
34	G09	1P0401A0	300 LED Light Head Assembly (RAL9003) w White Color	1
	G09	1P0402A0	300 LED Light Head Assembly (CN80) w Light Grey Color	

2. Arm Section


2-1. Arm (301)

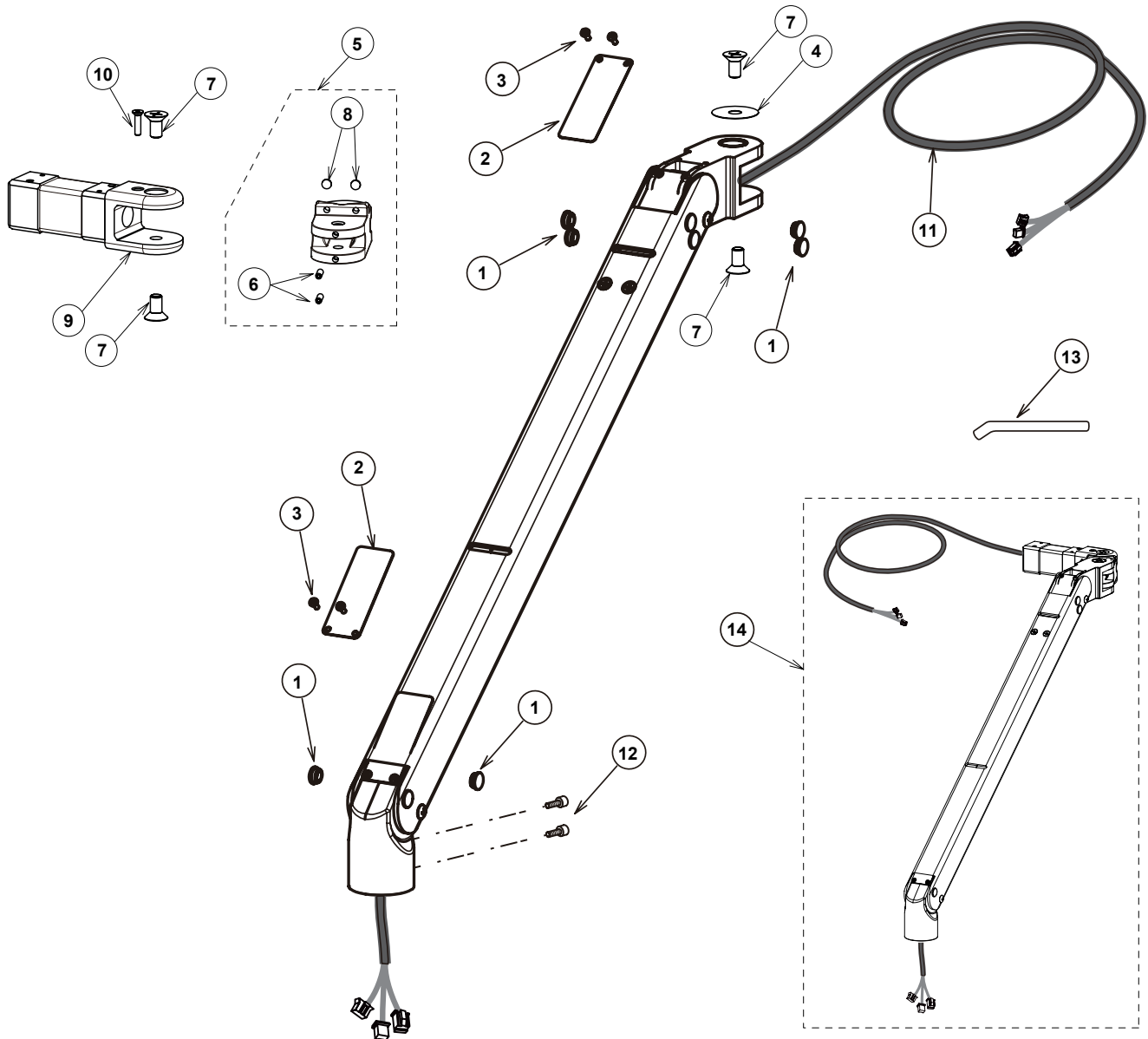

ID No.	Manufacture ID	Parts No.	Description	Qty
1	G09	CAKB10A0	Cap (White)	6
2	G09	CAMD09B0	Yoke Joint Cover	2
3	G09	ZA227JNY	Pan Head Screw M3 x 6 for York Joint Cover	4
4	G09	CAMF12A0	Washer 32 x 8.5 x 0.3	1
5	G09	HLU0050W	H-Shaped Arm Joint Assembly (RAL9003) w White Color	1
6	G09	ZA774LSY	Set Screw M4 x 12 (Cup Point)	2
7	G09	1E03JZA0	Countersink Screw M8 x 15	4
8	G09	CAMP65A0	Stopper (Large)	2
9	G09	HLU0040W	Swivel Arm Knuckle (RAL9003) w White Color	1
10	G09	1E03QSA0	Countersink Screw for L/R Fixing M4 x 15	1
11	G09	1A0BFTA0	Inside Arm Lead Wire	1
12	G09	ZA761NRU	Cap Screw M5 x 10 SUS for Head	2
13	G09	CANF43B0	Adjustment Bar	1
14	G09	1P0404A0	Balance Arm Assy (RAL9003) w White Color	1

2. Arm Section


2-2. Arm (302)

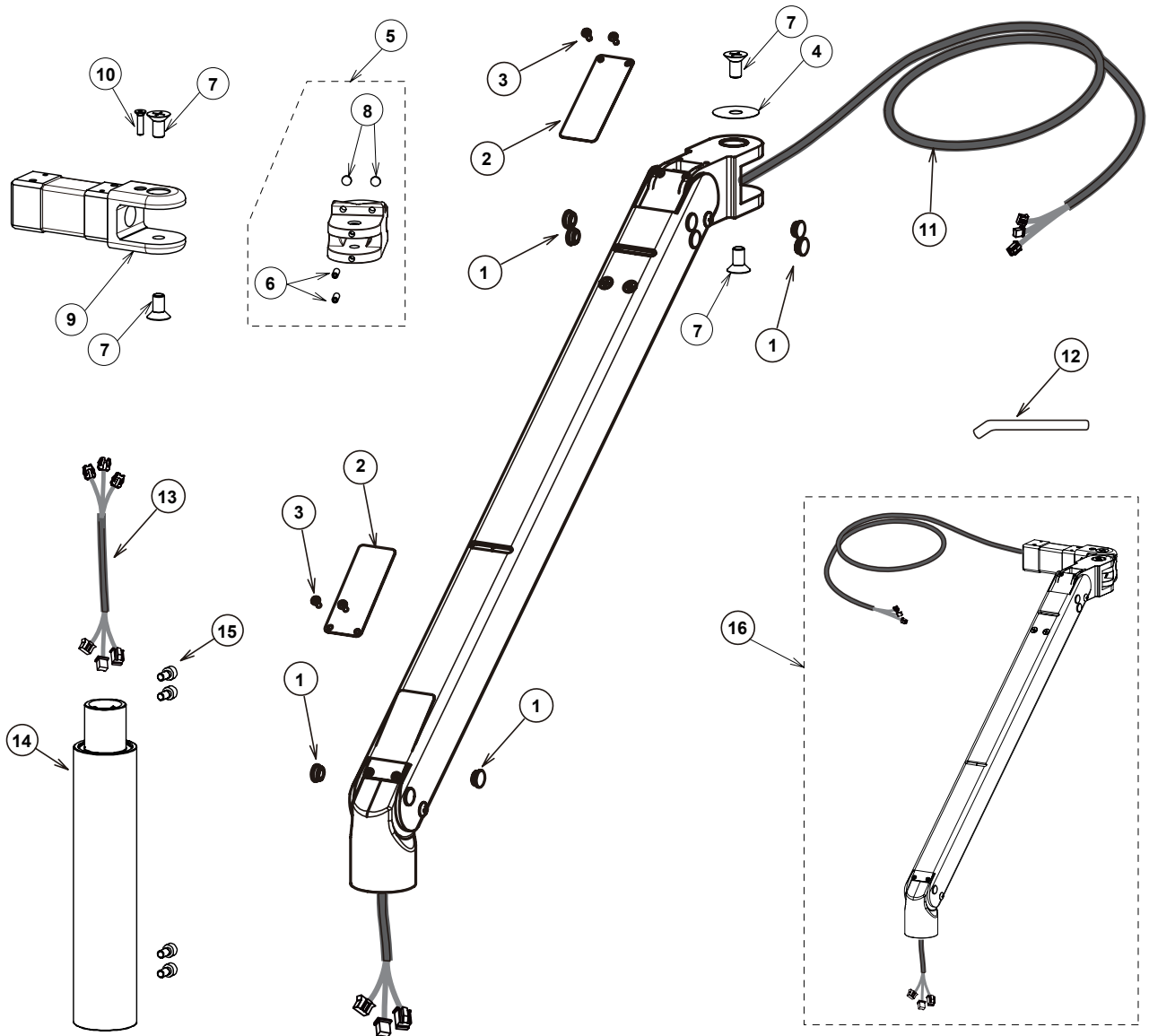

ID No.	Manufacture ID	Parts No.	Description	Qty
1	G09	CAKB10A0	Cap (White)	6
2	G09	CAMD09B0	Yoke Joint Cover	2
3	G09	ZA227JNY	Pan Head Screw for Yoke Joint Cover M3 x 6	4
4	G09	CAMF12A0	Washer 32 x 8.5 x 0.3	1
5	G09	1G05CGA0	H-Shaped Arm Joint Assembly (CN80) w Light Grey Color	1
6	G09	ZA774LSY	Set Screw M4 x 12 (Cup Point)	2
7	G09	1E03JZA0	Countersink Screw M8 x 15	4
8	G09	CAMP65A0	Stopper (Large)	2
9	G09	1A0HVTA0	Swivel Arm Knuckle (CN80)	1
10	G09	1E03QSA0	Countersink Screw for L/R Fixing M4 x 15	1
11	G09	1A0BFTA0	Inside Arm Lead Wire	1
12	G09	CANF43B0	Adjustment Bar	1
13	G09	1A0DEEA0	Extension Harness (100mm)	1
14	G09	1A0HVPA0	Extension (CN80) w Light Grey Color	1
15	G09	ZA761NRU	Cap Screw M5 x 10 SUS	4
16	G09	1P0407A0	Balance Arm Assy (CN80) w Light Grey Color	1

2. Arm Section


2-3. Arm (305)

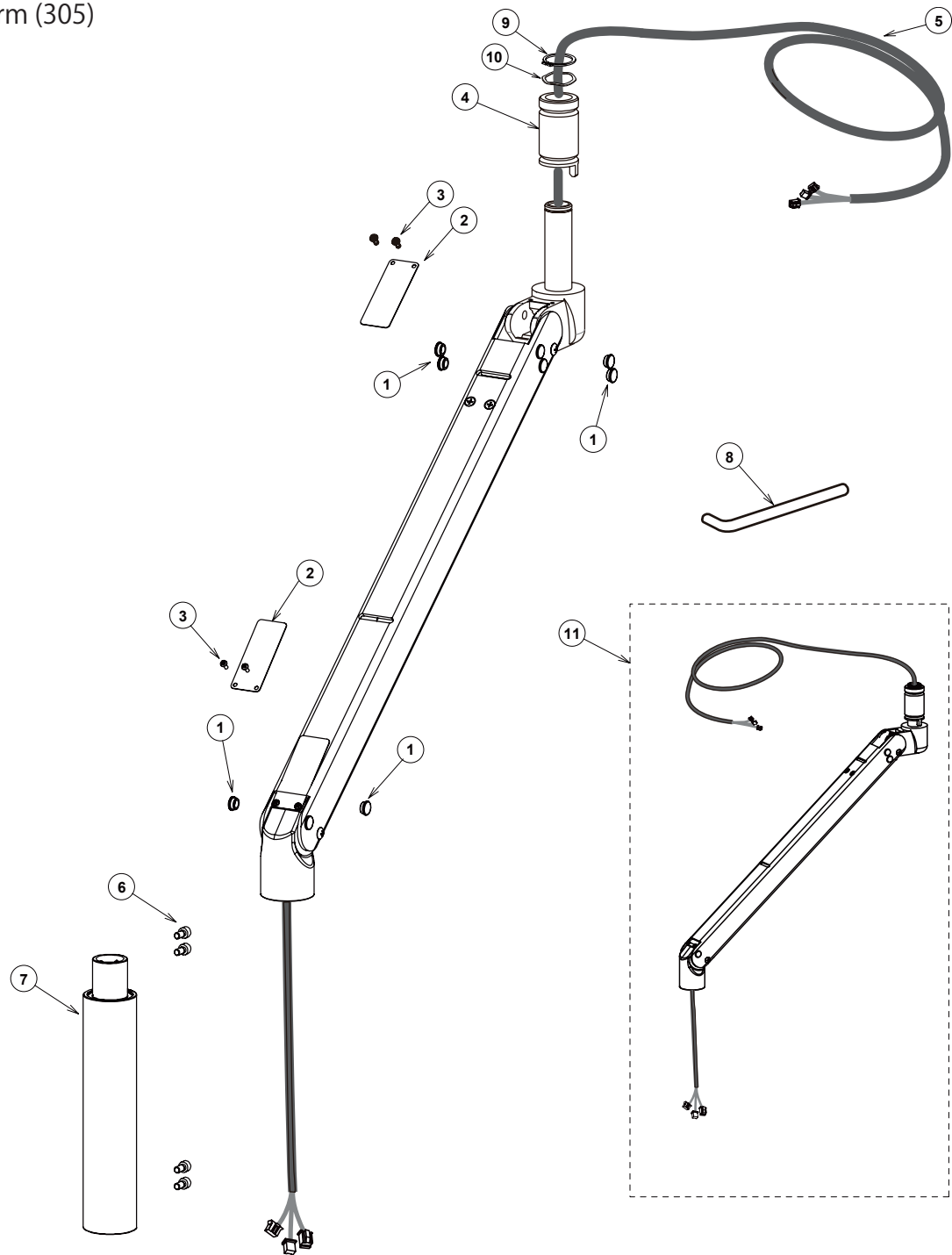

ID No.	Manufacture ID	Parts No.	Description	Qty
1	G09	CAKB10A0	Cap (White)	6
2	G09	CAMD09B0	Yoke Joint Cover	2
3	G09	ZA227JNY	Pan Head Screw M3 x 6 for York Joint Cover	4
4	G09	CAMG97A0	Pole Collar	1
5	G09	1A0BFTA0	Inside Arm Lead Wire	1
6	G09	ZA761NRU	Cap Screw for Head M5 x 10	4
7	G09	1A0HVPA0	Extension (CN80) w Light Grey Color	1
8	G09	CANF43B0	Adjustment Bar	1
9	G09	CENX95A0	C-Ring 1STW-22	1
10	G09	CENX96A0	Wave Washer WW-23	1
11	G09	1G05CSA0	Balance Arm Assembly (CN80) w Light Grey Color	1

2. Arm Section


2-4. Arm (320X / 320XN / 320XC)

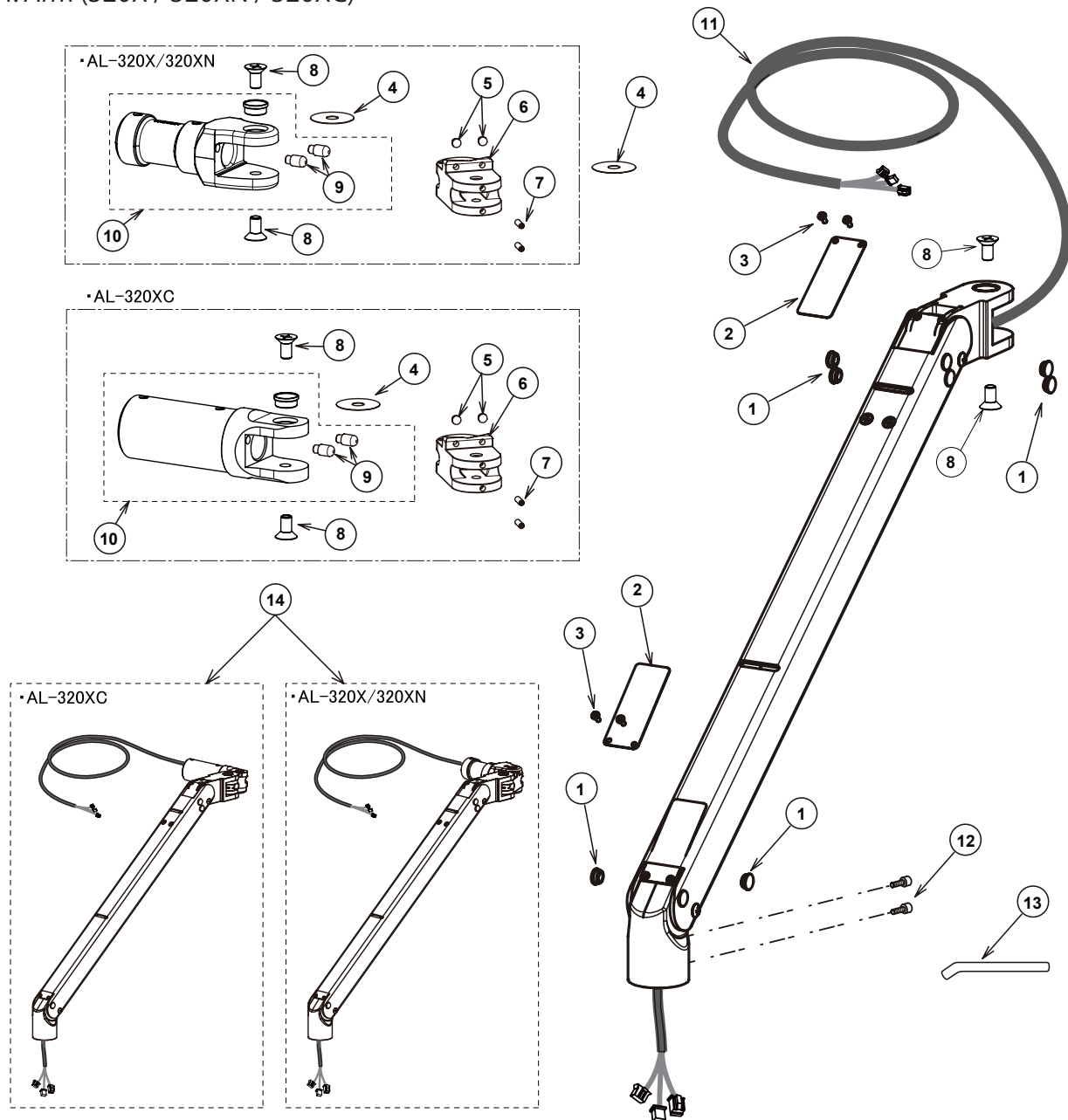

ID No.	Manufacture ID	Parts No.	Description	Qty
1	G09	CAKB10A0	Cap (White)	6
2	G09	CAMD09B0	Yoke Joint Cover	2
3	G09	ZA227JNY	Pan Head Screw for York Joint Cover M3 x 6	4
4	G09	CAMF12A0	Washer 32 x 8.5 x 0.3	2
5	G09	CAMP65A0	Stopper (Large)	2
6	G09	CATN97BR	H-Shaped Arm Joint (RAL9003) (320X, 320XN) w White Color	1
	G09	1A0HVXA0	H-Shaped Arm Joint (CN80) (320XC) w Light Grey Color	
7	G09	ZA774LSY	Set Screw M4 x 12 (Cup Point)	2
8	G09	1E03JZA0	Countersink Screw M8 x 15	4
9	G09	1A0G2VA0	Stopper RL L=9.5	2
10	G09	1P049LA0	Swivel Arm Knuckle Assy (RAL9003) (320X, 320XN) w White Color	1
	G09	1P04CDA0	Swivel Arm Knuckle Assy (CN80) (320XC) w Light Grey Color	
11	G09	1A0DEPA0	Inside Arm Lead Wire	1
12	G09	ZA761NRU	Cap Screw for Head M5 x 10 SUS	2
13	G09	CANF43B0	Adjustment Bar	1
14	G09	1P040CA0	Balance Arm Assy (RAL9003) (320X, 320XN) w White Color	1
	G09	1P040DA0	Balance Arm Assy (CN80) (320XC) w Light Grey Color	

3. Power Supply Box Section

3-1. Power Supply Box (301)

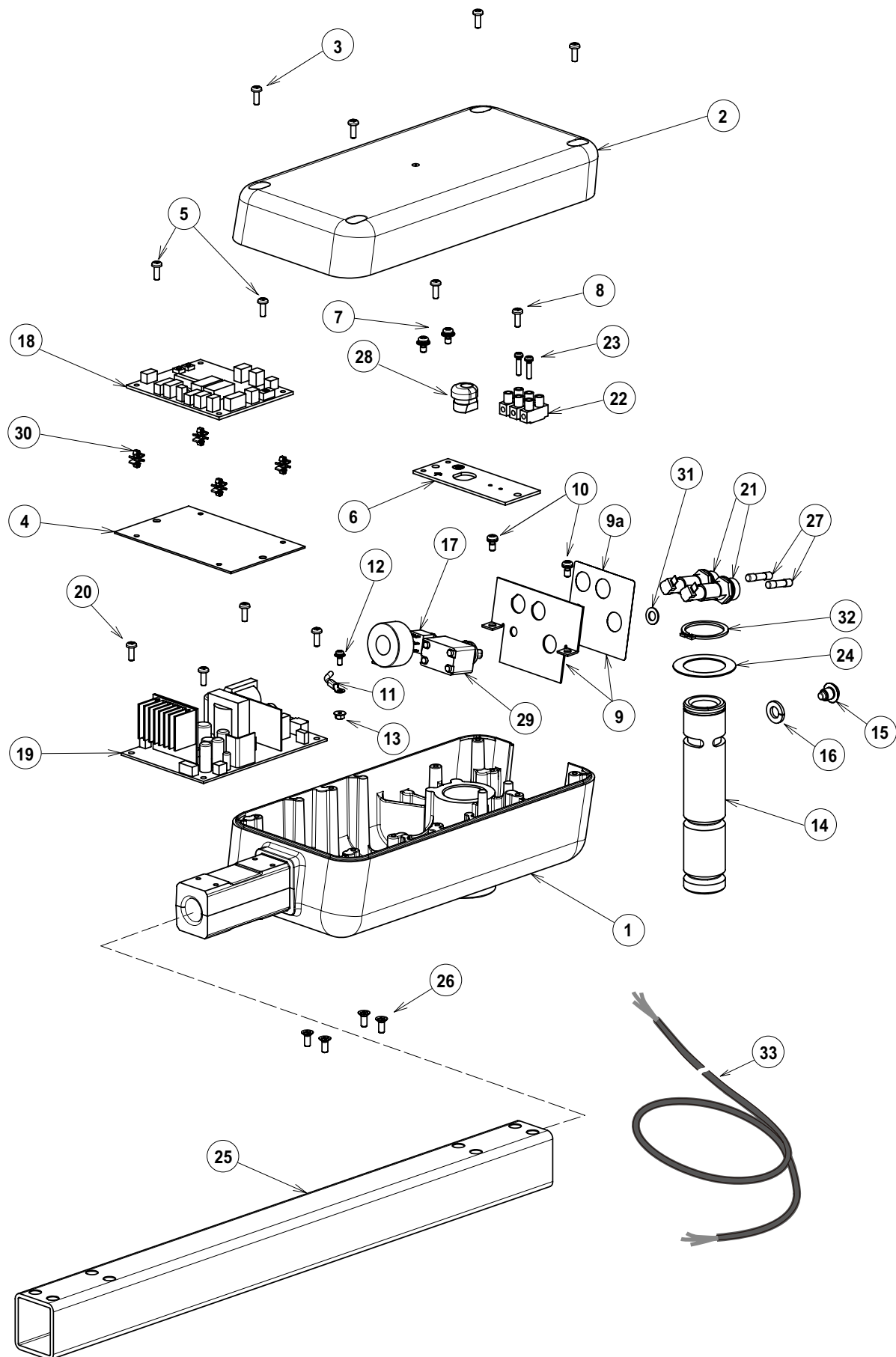


3. Power Supply Box Section

3-1. Power Supply Box (301)

ID No.	Manufacture ID	Parts No.	Description	Qty
1	G09	1A0BFMA0	Housing (RAL9003) w White Color	1
2	G09	1A0BFNA0	Housing Cover (RAL9003) w White Color	1
3	G09	ZA111LPY	Pan Head Screw M4 x 8 for Housing Cover	4
4	G09	1A0BFPA0	Control PCB Fixing Plate	1
5	G09	ZA111LPY	Pan Head Screw M4 x 8	2
6	G09	1A0BFQB0	Electric Parts Fixing Plate	1
7	G09	ZA88JLRL	Sems Screw P3 M4 x 10	2
8	G09	ZA111LPY	Pan Head Screw M4 x 8	2
9a	G09	1A0EPCA0	Switch Seal for 301 (RAL9003) w White Color	1
9	G09	1P049XA0	Back Plate W/ Switch Seal (RAL9003) w White Color	1
10	G09	ZA111LPY	Pan Head Screw M4 x 8 for Back Plate	2
11	G09	1A0BMLA0	Functional Earth Line for Power PCB	1
12	G09	ZA88JJPY	Sems Screw P3 M3 x 8 for Earth Line	1
13	G09	ZA901JAY	Flange Nut	1
14	G09	1A07W1B0	Rotation Shaft	1
15	G09	1A07W2A0	Rotation Stopper Bolt M8	1
16	G09	ZB511PAY	Spring Washer M8 for Rotation Stopper Bolt	1
17	G09	1A0HMMA0	S/M Mode Switch Assembly (with wire & Ferrite core)	1
18	G09	1G02A0C0	Sensor Control PCB (DL-900)	1
19	G09	1G03JHB0	Power PCB (301)	1
20	G09	ZA111LPY	Pan Head Screw M4 x 8 for Power PCB.	4
21	G09	AEFR93A0	Fuse Holder F-400-01A2	2
22	G09	CAML90B0	Terminal Block	1
23	G09	ZA88JJUY	Sems Screw P3 M3 x 16 for Terminal Block	2
24	G09	501230	Washer 43 x 29 x 0.8	1
25	G09	HLU0031W	Swivel Arm 485mm (4 holes) (RAL9003) w White Color	1
26	G09	ZA112LRY	Flat Head Screw M4 x 10	4
27	G09	CEEN94A0	Fuse 250V 0.8A (T)	2
28	G09	CEGN46A0	Cord Bush	1
29	G09	CENX56B0	Main Switch (0145-1220_6A)	1
30	G09	CERN39A0	Double Locking Spacer (WLS-04-0)	4
31	G09	ZB561OAM	Flat Washer M6 x 11.5 x 0.8	1
32	G09	501231	C-Ring STW-29	1
33	G09	1A0BMMA0	CT Code 5000mm	1

3. Power Supply Box Section
3-2. Power Supply Box (302)

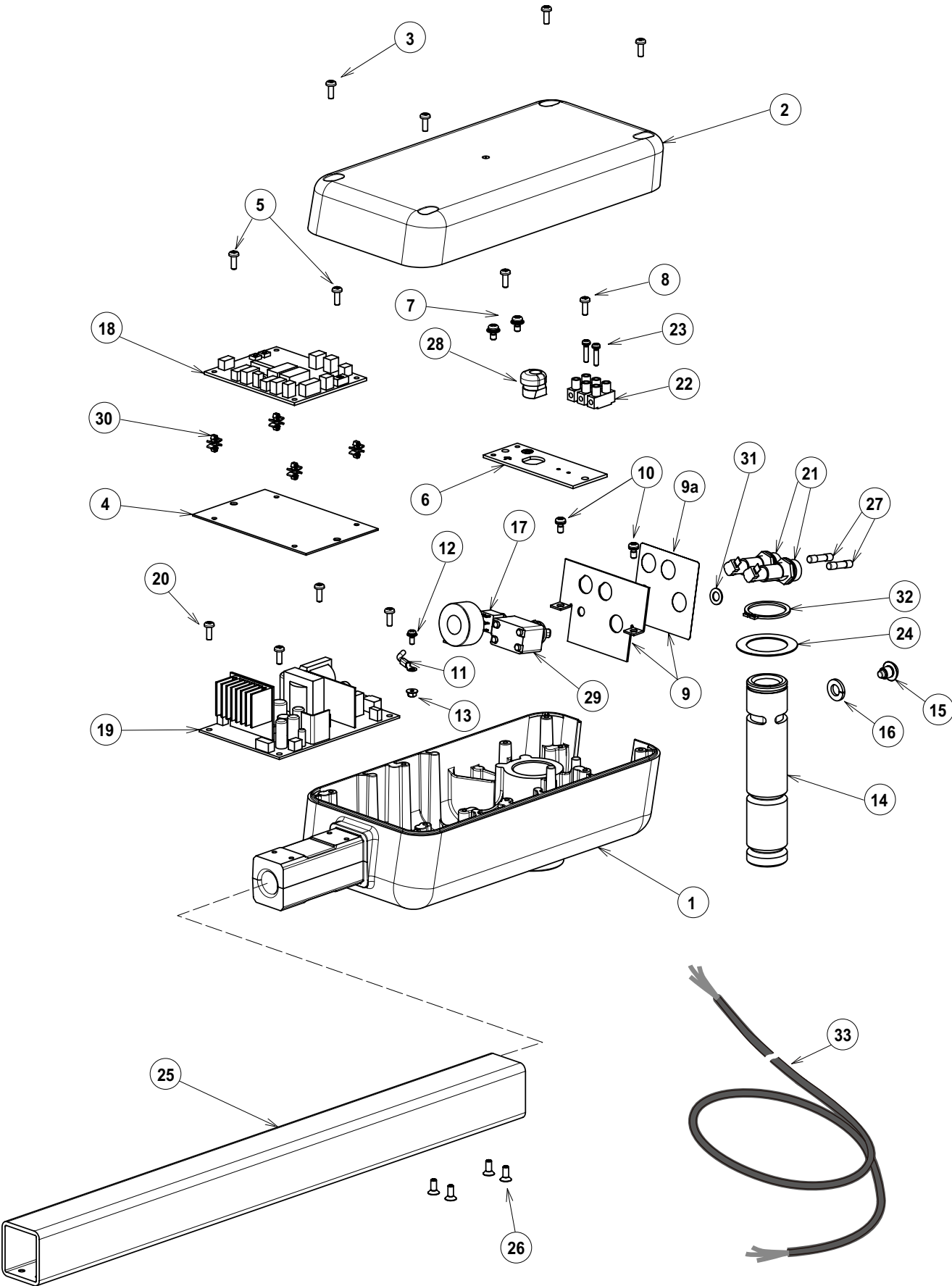


3. Power Supply Box Section

3-2. Power Supply Box (302)

ID No.	Manufacture ID	Parts No.	Description	Qty
1				1
	G09	1A0HVNA0	Housing (CN80) w Light Grey Color	
2				1
	G09	1A0HVJA0	Housing Cover (CN80) w Light Grey Color	
3	G09	ZA111LPY	Pan Head Screw M4 x 8 for Housing Cover	4
4	G09	1A0BFPA0	Control PCB Fixing Plate	1
5	G09	ZA111LPY	Pan Head Screw M4 x 8	2
6	G09	1A0BFQB0	Electric Parts Fixing Plate	1
7	G09	ZA88JLRL	Sems Screw P3 M4 x 10	2
8	G09	ZA111LPY	Pan Head Screw M4 x 8	2
9a				1
	G09	1A0J2GA0	Switch Seal for 302 (CN80) w Light Grey Color	
9				1
	G09	1P04A8A0	Back Plate W/ Switch Seal (CN80) w Light Grey Color	
10	G09	ZA111LPY	Pan Head Screw M4 x 8 for Back Plate	2
11	G09	1A0BMLA0	Functional Earth Line for Power PCB	1
12	G09	ZA88JJPY	Sems Screw P3 M3 x 8 for Earth Line	1
13	G09	ZA901JAY	Flange Nut	1
14	G09	1A07W1B0	Rotation Shaft	1
15	G09	1A07W2A0	Rotation Stopper Bolt M8	1
16	G09	ZB511PAY	Spring Washer M8 for Rotation Stopper Bolt	1
17	G09	1A0HMMA0	S/M Mode Switch Assembly (with wire & Ferrite core)	1
18	G09	1G02A0C0	Sensor Control PCB (DL-900)	1
19	G09	1G03JHB0	Power PCB (301)	1
20	G09	ZA111LPY	Pan Head Screw M4 x 8 for Power PCB.	4
21	G09	AEFR93A0	Fuse Holder F-400-01A2	2
22	G09	CAML90B0	Terminal Block	1
23	G09	ZA88JJUY	Sems Screw P3 M3 x 16 for Terminal Block	2
24	G09	501230	Washer 43 x 29 x 0.8	1
25				1
	G09	1A0HVSA0	Swivel Arm 485mm (4 holes) (CN80) w Light Grey Color	
26	G09	ZA112LRY	Flat Head Screw M4 x 10	4
27	G09	CEEN94A0	Fuse 250V 0.8A (T)	2
28	G09	CEGN46A0	Cord Bush	1
29	G09	CENX56B0	Main Switch (0145-1220_6A)	1
30	G09	CERN39A0	Double Locking Spacer (WLS-04-0)	4
31	G09	ZB561OAM	Flat Washer M6 x 11.5 x 0.8	1
32	G09	501231	C-Ring STW-29	1
33	G09	CADQ71B0	Power Cable 3200mm (SJT)	1

3. Power Supply Box Section

3-3. Power Supply Box (305)

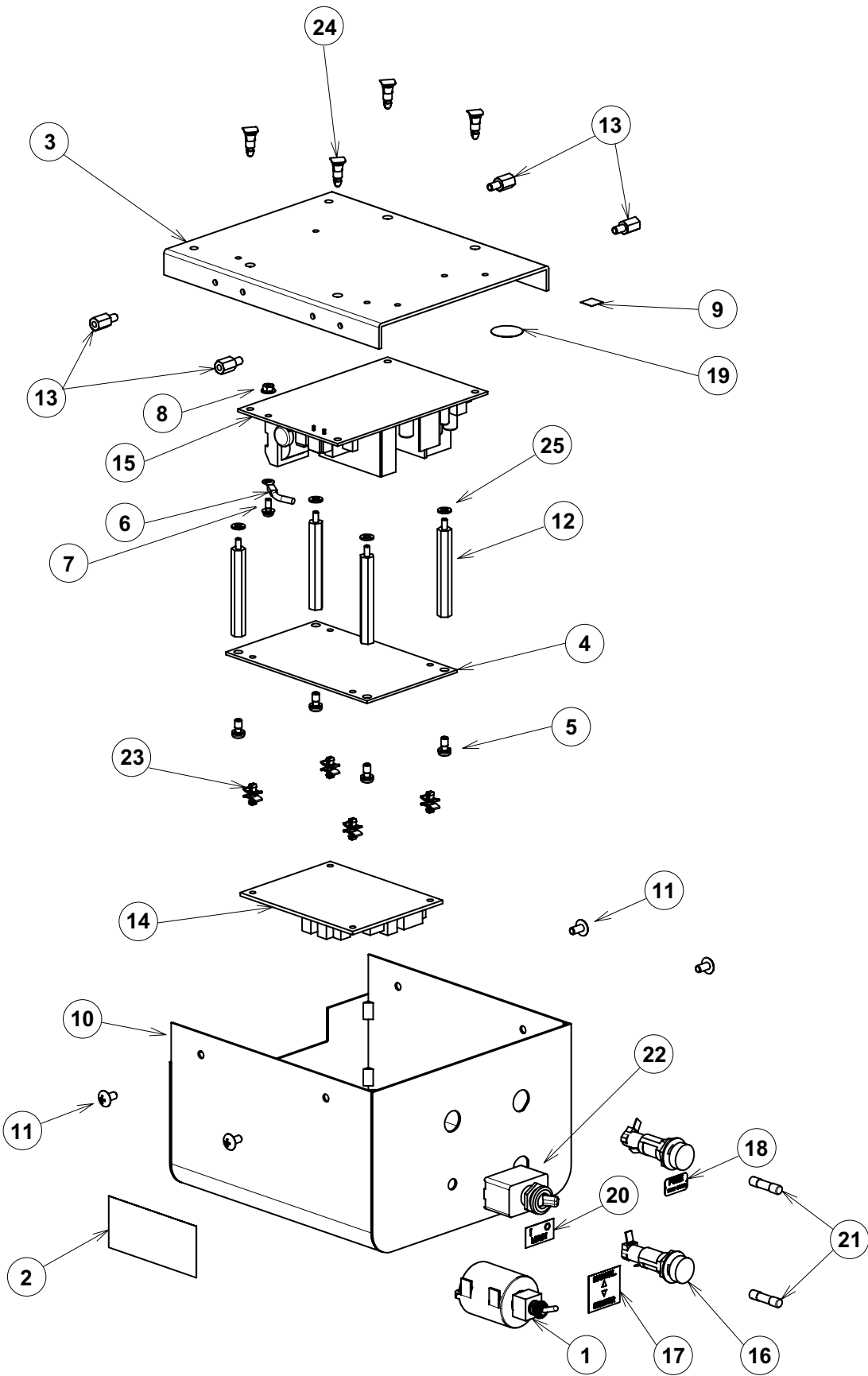


3. Power Supply Box Section

3-3. Power Supply Box (305)

ID No.	Manufacture ID	Parts No.	Description	Qty
1	G09	1A0HMMA0	S/M Mode Switch Assembly (with wire & Ferrite core)	1
2	G09	1A0HQZA0	Caution Label (English and French)	1
3	G09	1A07M5B0	Power PCB Fixing Plate	1
4	G09	1A07PKA0	Sensor Control PCB Fixing Plate	1
5	G09	ZA887LPY	Pan Head Sems Screw M4 x 8	4
6	G09	1A07X8A0	Functional Earth Line for Power PCB	1
7	G09	ZA88JJPY	Sems Screw P3 M3 x 8 for Functional Earth Line	1
8	G09	ZA901JAY	Flange Nut	1
9	G09	1A07YUA0	Earth Seal	1
10				1
	G09	1A07M8A0	Power Supply Box Cover (AL-905) (CN80) w Light Grey Color	
11				4
	G09	1E02HPA0	Truss Screw M4 x 8 (CN80) w Light Grey Color	
12	G09	1E02E9A0	Metal Spacer BSB-452	4
13	G09	1E04S0A0	Hex Metal Spacer BSB-410E	4
14	G09	1G02A0C0	Sensor Control PCB (DL-900)	1
15	G09	1G03JHB0	Power PCB (301)	1
16	G09	AEFR93A0	Fuse Holder F-400-01A2	2
17	G09	CADR07A0	Seal for S/M Mode Switch	1
18	G09	CAEP84A0	Fuse Seal 0.8A	1
19	G09	CAMT05A0	Earth Mark (N-7)	1
20	G09	CARO04A0	Seal for Main Switch	1
21	G09	CEEN94A0	Fuse 250V 0.8A (T)	2
22	G09	CEGN56B0	Main Switch (0145-1220_6A)	1
23	G09	CERN39A0	Double Locking Spacer (WLS-04-0)	4
24	G09	DESR05A0	Spacer RSPLS-8LU	4
25	G09	501209	Spring Washer M4	4

3. Power Supply Box Section

3-4. Power Supply Box (320X / 320XN / 320XC)

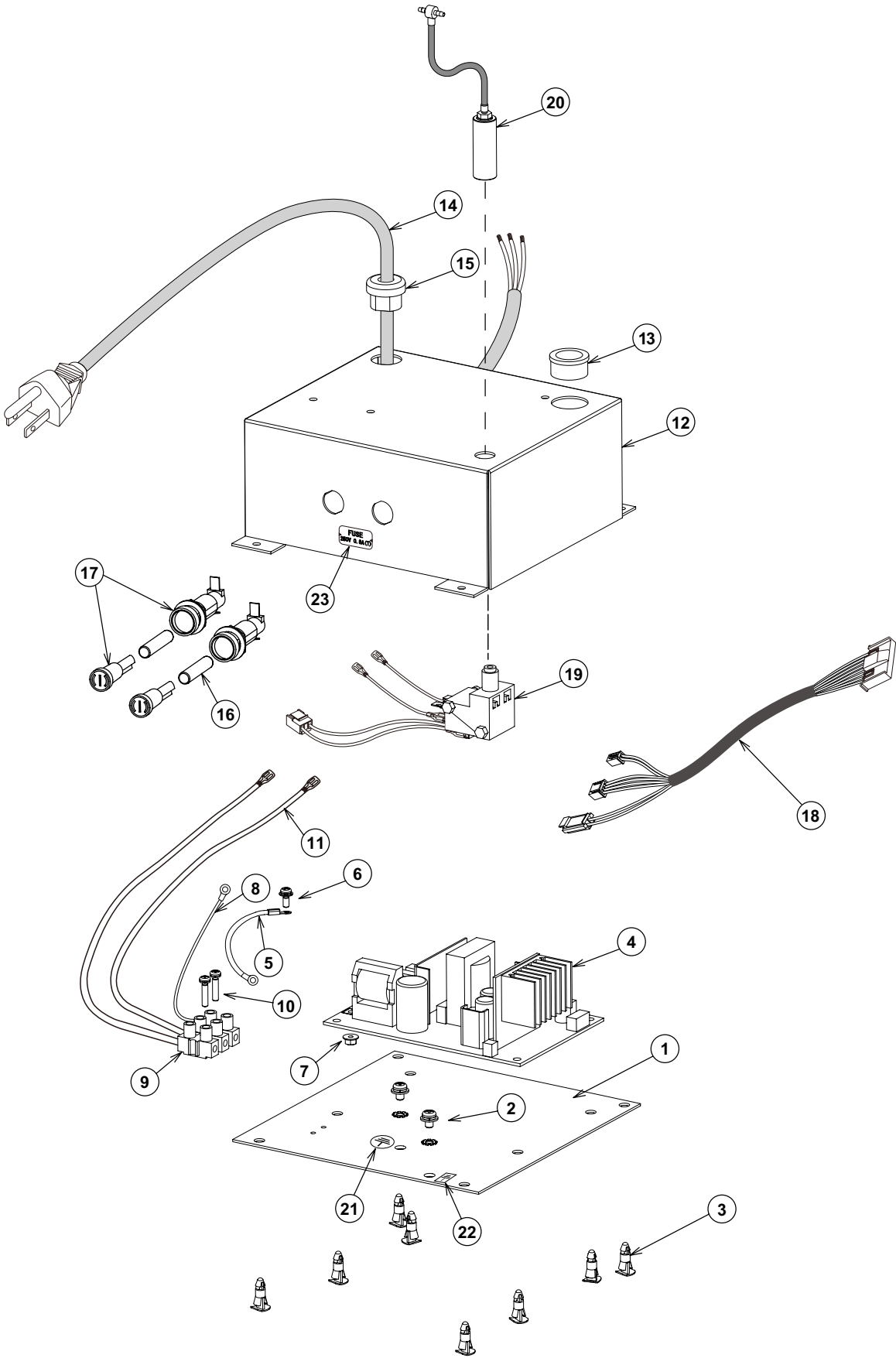


3. Power Supply Box Section

3-4. Power Supply Box (320X / 320XN / 320XC)

ID No.	Manufacture ID	Parts No.	Description	Qty
1	G09	1A08BWA0	Power Supply PCB Box Fixing Plate	1
2	G09	ZA88JLNM	Pan Head Sems Screw P3 M4 x 6	2
3	G09	DESR05A0	Spacer RSPLS-8LU	8
4	G09	1G03JHB0	Power PCB (301)	1
5	G09	1A07X8A0	Functional earth line for power PCB	1
6	G09	ZA88JJPY	Sems Screw P3 M3 x 8 for Functional Earth Line	1
7	G09	ZA901JAY	Flange Nut	1
8	G09	CADQ74A0	Earth Cable (701M)	1
9	G09	CAML90B0	Terminal Block	1
10	G09	ZA887JTY	Sems Screw P2 M3 x 14 for Terminal Block	2
11	G09	1A0DEFA0	Trunk Line for Fuse Holder (305)	1
12	G09	1A08BVB0	Power Supply Box Cover (AL-920X)	1
13	G09	AFFX96A0	Code Bush OCB-750	1
14	G09	1A04D6A0	Power Supply Code 1000L	1
15	G09	CESN07A0	Relief Bushing SR-6N3-4	1
16	G09	CEEN94A0	Fuse 250V 0.8A (T)	2
17	G09	AEFR93A0	Fuse Holder F-400-01A2	2
18	G09	G061A08CHA0	Signal Line for Power PCB	1
19	G09	1G02LKA0	Air Switch Body Assy	1
20	G09	1G02MBA0	Air Switch Cylinder Assy with Tubing & Fitting	1
21	G09	CAMT05A0	Earth Mark	1
22	G09	1A07YUA0	Earth Seal	1
23	G09	CAEP84A0	Fuse Seal 0.8A	1
1 to 23	G09	1G042EA0	Power Supply Box Assembly (320X / 320XN / 320XC)	1

4. Switch Box Section


4-1. Switch Box (320X / 320XC)

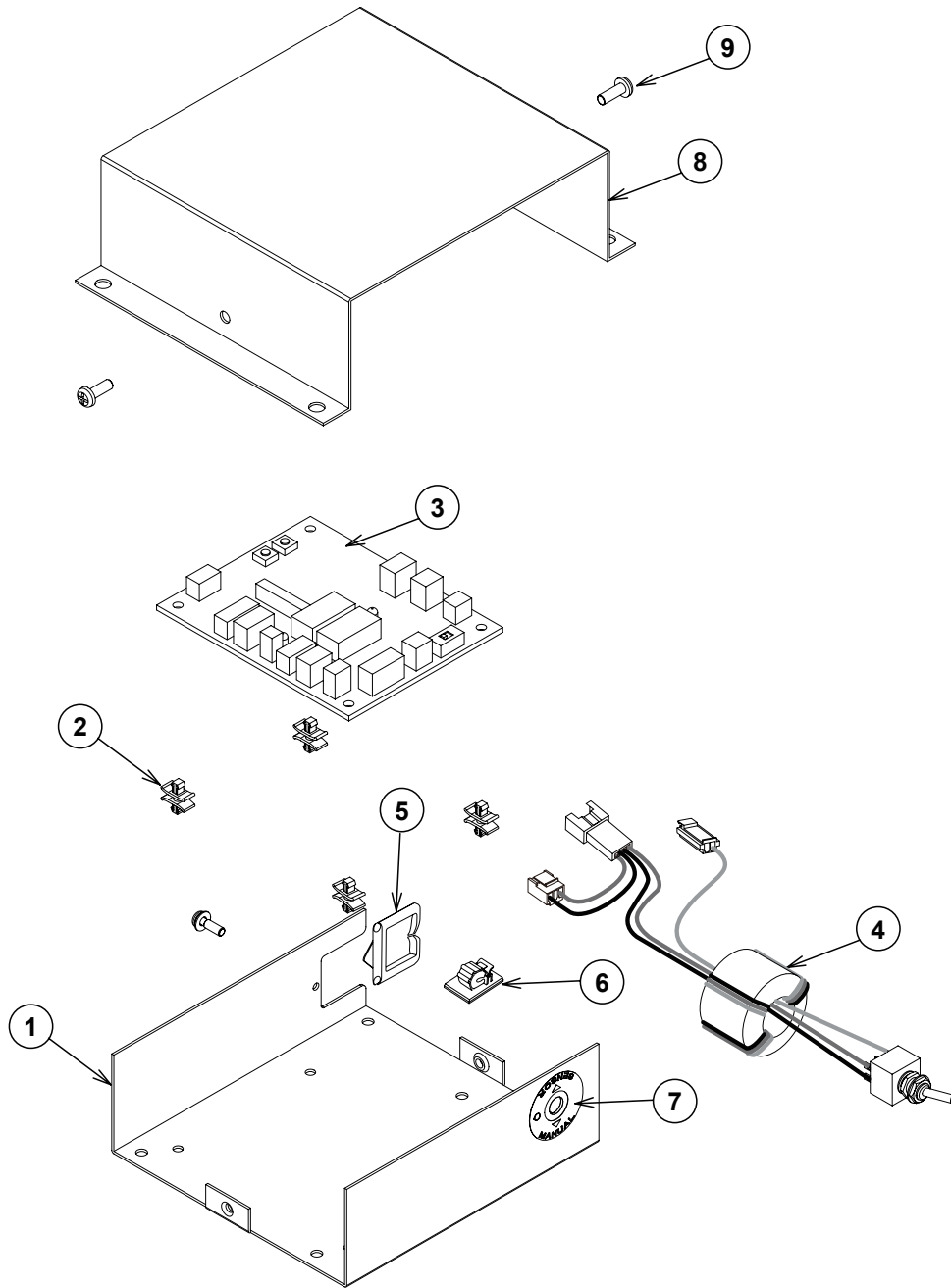

ID No.	Manufacture ID	Parts No.	Description	Qty
1	G09	1A08CTA0	Switch Box (RAL9003) w White Color	1
	G09	1A0HVMA0	Switch Box (CN80) w Light Grey Color	
2	G09	CERN39A0	Double Locking Spacer (WLS-04-0)	4
3	G09	1G02A0C0	Sensor Control PCB (DL-900)	1
4	G09	1A089UA0	S/M Mode Switch Assembly	1
5	G09	CETN14A0	Edge Saddle for Cable Protection	1
6	G09	CETN13A0	Clamp (w/Sponge)	1
7	G09	1A0HMYA0	Seal for S/M Mode Switch	1
8	G09	CATN46CR	Switch Box Cover (RAL9003) w White Color	1
	G09	1A0HVVA0	Switch Box Cover (CN80) w Light Grey Color	
9	G09	1E00VTA0	Pan Head Screw M4 x 10 (RAL9003) for PCB Box Cover w White	2
	G09	1A033FA0	Pan Head Screw M4 x 10 (CN80) for PCB Box Cover w Light Grey	

4. Switch Box Section


4-2. Switch Box (320XN)

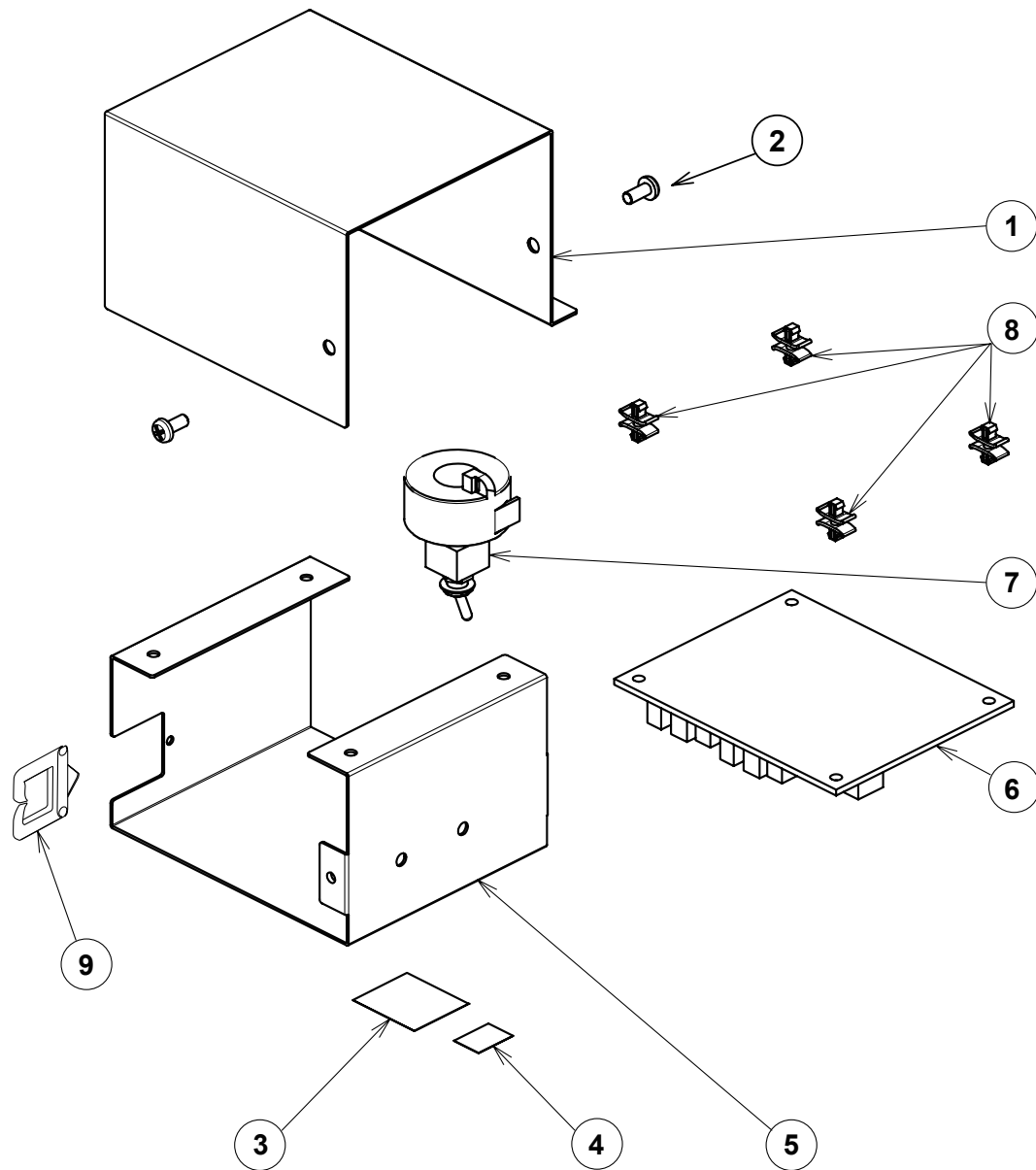

ID No.	Manufacture ID	Parts No.	Description	Qty
1	G09	1A0AFRA0	Switch Box Cover (530XN)	1
2	G09	ZA887LPU	Pan Head Sems Screw P2 M4 x 8 SUS	2
3	G09	CADR07A0	Seal for S/M Mode Switch	1
4	G09	U-31860	Label for Power Supply	1
5	G09	1A0B6SA0	Switch Box (920XN)	1
6	G09	1G02A0C0	Sensor Control PCB (DL-900)	1
7	G09	1A089UA0	S/M Mode Switch Assembly	1
8	G09	CERN39A0	Double Locking Spacer (WLS-04-0)	4
9	G09	CETN14A0	Edge Saddle for Cable Protection	1

5. Ceiling Pole Section

5-1. Ceiling Pole (302)

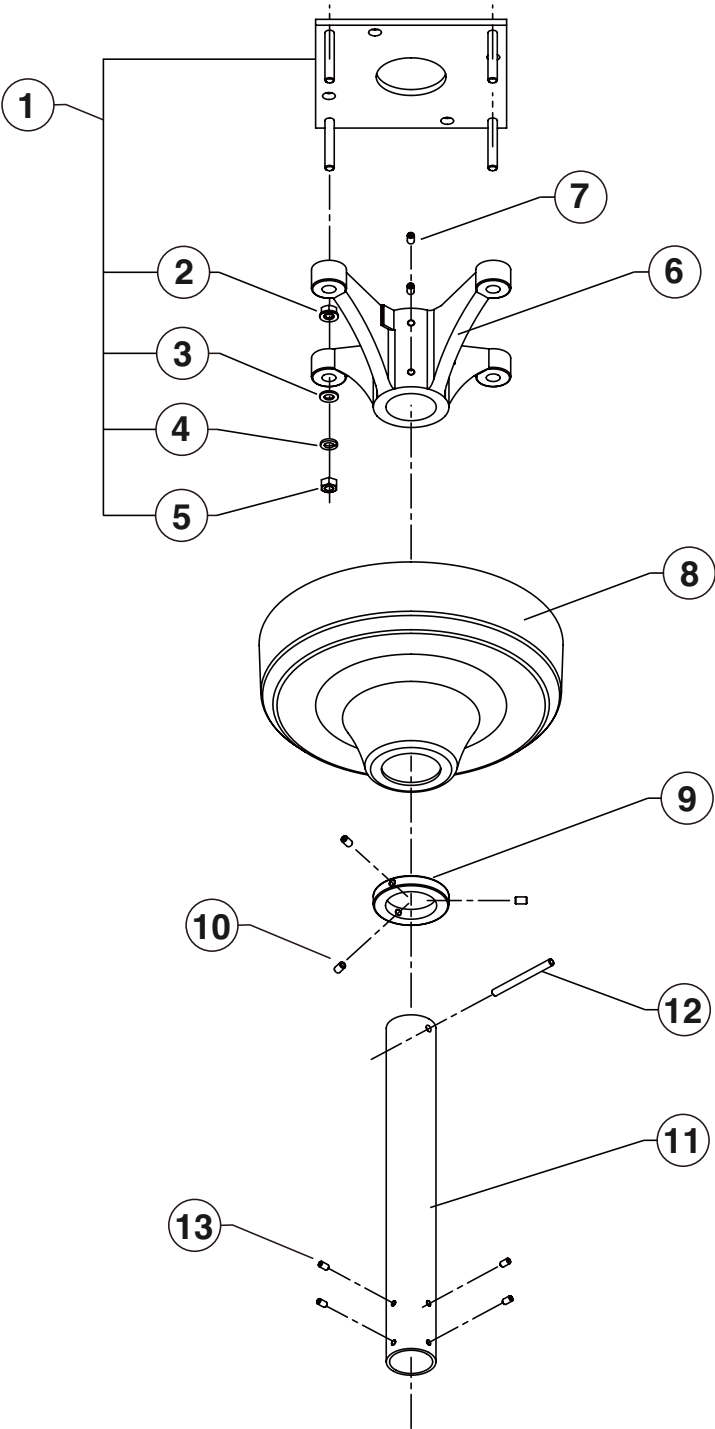


5. Ceiling Pole Section

5-1. Ceiling Pole (302)

ID No.	Manufacture ID	Parts No.	Description	Qty
1	G09	1P0211A0	Mounting Plate with Nut (No.2 - 5)	1
2	G09	ZA901PAY	Flange Nut (Leveling Nut)	4
3	G09	CENE00B0	Flat Washer M8 x 22 x 1.63	4
4	G09	ZB511PAY	Spring Washer M8	4
5	G09	1E00ABA0	U Nut M8	4
6	G09	HLU0140	Ceiling Flange	1
7	G09	ZA774OPU	Set Screw M6 x 8 (Cup Point)	4
8				1
	G09	1A0827A0	Ceiling Cover (Gray) w Light Grey Color	
9	G09	CAMG76A0	Stopper Ring	1
10	G09	CENB35B0	Set Screw M6 x 8 (Cup Point) (Red Mec Thread Lock)	3
11	G09	LPCCG8	Ceiling Pole Set 340mm (CN80) 8FT	1
	G09	LPCCG8.5	Ceiling Pole Set 510mm (CN80) 8.5FT	
	G09	LPCCG9	Ceiling Pole Set 680mm (CN80) 9FT	
	G09	LPCCG9.5	Ceiling Pole Set 840mm (CN80) 9.5FT	
	G09	LPCCG10	Ceiling Pole Set 1000mm (CN80) 10FT	
12	G09	HLU0116	Roll Pin 6 x 70	1
13	G09	HLU0115	Set Screw M5 x 10 (Truncated cone point)	4
	G09	CGRN38B0	Ceiling Screw Set	1

6. Track Section

6-1. Track (305)

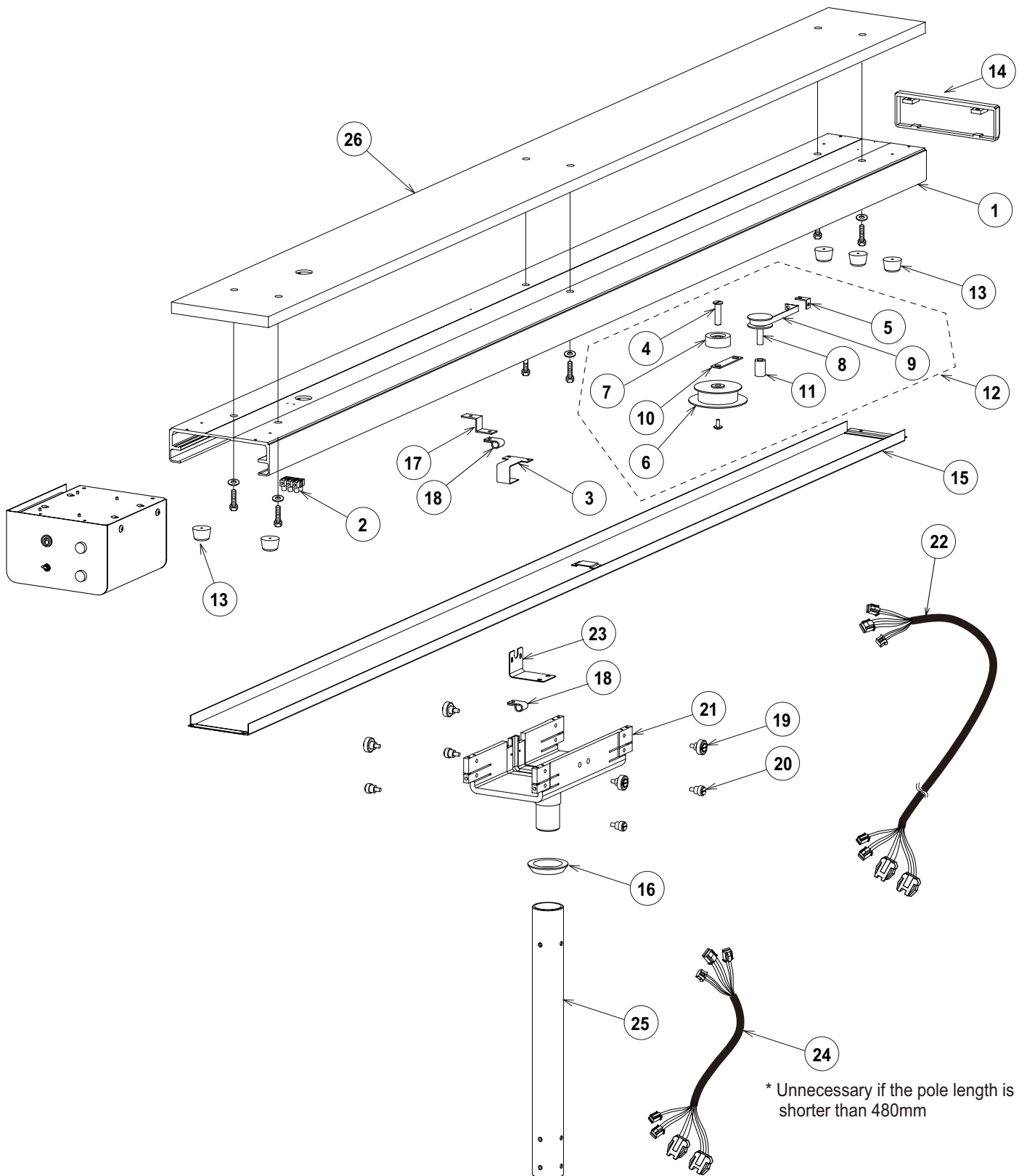

6. Track Section

6-1. Track (305)

ID. No.	Manufacture ID	Parts No.	Description	Qty.
1	G09	1A0824A0	Rail (Hammer Tone Coating CN80) w Light Grey Color	1
2	G09	CAML90B0	Terminal Block	1
3	G09	501643	Center Cover Hook (OP/NDL)	1
4	G09	CAMG88B0	Drum Shaft	1
5	G09	CAMG91A0	Constant Force Spring Plate	1
6	G09	CAMP70B0	Pulley (OP/NDL) Duracon	1
7	G09	CAMP71C0	Drum A Duracon	1
8	G09	CAMP72C0	Drum B Duracon	1
9	G09	CAMU63A0	Constant Force Spring	1
10	G09	CAMV62A0	Guide (OP/NDL) SPOC	1
11	G09	CAQN96B0	Drum Collar (White POM)	1
12	G09	1G0133A0	Pulley Assy (W/Seal)	1
13	G09	HLU1115	Stopper	5
14	G09	1G02EEA0	End Cover ASSY (CN80) w Light Grey Color	1
15	G09	1G02EFA0	Centre Cover ASSY (CN80) w Light Grey Color	1
16	G09	CAMG95A0	Trolley Collar (Couted)	1
17	G09	CAMV63A0	Lead Wire Guide (OP/NDL)	1
18	G09	CENZ15A0	Nylon Clamp NK-4N	2
19	G09	HLU0160	Roller Upper (OPNDL) ASSY	4
20	G09	HLU0165	Roller Lower (OPNDL) ASSY	4
21	G09	1A0823A0	Trolley (Hammer Tone Coating, CN80) w Light Grey Color	1
22	G09	1A087RA0	Trunk Line A (Power Supply Box to Yoke)(AL-905)	1
23	G09	CAMP73B0	Lead Wire Cover	1
24	G09	1A087TA0	Trunk Line C (Power Supply Box to Yoke)(AL-905)	1
25	G09	LPCCG8	T Pole Set 380mm (CN80) 8FT	1
	G09	LPCCG8.5	T Pole Set 530mm (CN80) 8.5FT	
	G09	LPCCG9	T Pole Set 680mm (CN80) 9FT	
	G09	LPCCG9.5	T Pole Set 840mm (CN80) 9.5FT	
	G09	LPCCG10	T Pole Set 1000mm (CN80) 10FT	
26	G09	CAPN00D0	Pallet (UL Type)	1
	G09	1G009XA0	Pallet Screw Set	1
	G09	CGRN39B0	Track Screw Set	1

NOTE



Importer for U.S.A.

BELMONT EQUIPMENT, Division of Takara Belmont USA, Inc.

101 Belmont Drive, Somerset, New Jersey 08873 U.S.A.

TEL.: (732) 469-5000 / (800) 223-1192 FAX.: (732) 526-6322 / (800) 280-7504

Importer for Canada

TAKARA CO, CANADA LTD.

2076 S. Sheridan Way, Mississauga, Ont., L5J2M4, Can. TEL.: (905) 822-2755 FAX.: (905) 822-6203

Book No. 1E05BLAE
2016-06