SERVICE MANUAL

DIGITAL PANORAMA AND CEPHALO RADIOGRAPH BelmaXCM

Notice to person for after-service

- * Before the repair working, read through this manual.
- * Take this Service manual with you to the repair.

TAKARA BELMONT U.S.A. INC.

Before Repair

Repair sometimes might require dangerous working for adjustment, checking, etc. Therefore, repair should be performed by qualified person or any person who receive proper education or training at our company. Please read and understand this service manual.

CONTENTS

01. When you read Service Manual	01-1/1
02. WARNING	02-1/7
03. Preparation for Repair	03-1/1
04. Specifications	04-1/1
05. Component	05-1/1
06. Setting	06-1/1
07. Trouble shooting	07-1/3
08. Treatment of Error Message	08-1/4
09. Wiring Diagram of Printed Board	09-1/2
10. Parts List	10-1/14
11. Contact Information	
12. Maintenance Check	12-1/1

01. INTRODUCTION

1. Read carefully before operation

2. The operating procedure of the panoramic radiography is the basic. Understand before other radiograph.

- 3. Observe "Warning" and "Prohibition" matters in the service manual.
- 4. Read the Operation Manual from 1st page in order to prevent any troubles, accidents and etc.
- 5. When you have any unclear matter during operation, consult with the operation manual.
- 6. Exemption from Responsibility

Be sure to observe the contents of Service Manual. The accident and breakage of this equipment due to an incorrect repair work are out of the scope of our responsibilities.

7. Warranty Period for Equipment

The warranty period is one year from the date of purchase. The charge-free warranty will be applied only to the cases that breakage, failure, etc. of this equipment occurred through normal use.

- 8. Available Period for Repair Parts and Service Parts is 10 years form discontinued of the equipment
- 9. The redemption period of the equipment is six years.
- 10. The disposal of equipment is within the scope of responsibilities of the customer. In case to disposal, consult with the dealer you purchased from or with us.

02.WARNING

M WARNING

For installation, operation, and maintenance, be sure to follow the instructions included in this manual. If the repair is mistaken by any chance, it causes damage and the accident of the equipment.

NOTE

Details of WARNING are stated as follows in this manual.



DANGER

Used for the immediate danger that causes death, serious injury, serious destruction of the property and occurrence of a fire.

WARNING

Used for indirect danger that causes death, serious injury, serious destruction of the property and occurrence of a fire.



CAUTION

Used for the danger that causes slight injury or medium level of trouble, partial loss of equipment and disappearance of the computer data.



NOTE

In using equipment, used for the helpful information.

DANGER

Liquids on this equipment will cause electric shock accident or equipment damage. This equipment is electric equipment. Keep liquids away from this equipment.

 \wedge

▲ DANGER

Shock hazard. This equipment is electric equipment and has some high-voltage portions inside. Turn off the power of equipment and unplug power cable from electrical outlet before opening cover of equipment for repair work.

A DANGER

Some repair works involve risks. Only qualified or trained persons may do repair works.

▲ DANGER

Some repair works involve risks. Service personnel must give instructions to outsiders to stay away from repair work area.

Do not make alterations to medical electronics equipment!

Alterations by user are prohibited.

Also, relevant pharmaceutical affairs law imposes following regulations on manufacturers.

That is, medical equipments need item-specific approval for manufacture, and "application for partial modification approval" is required when making functional changes in medical equipments. So unauthorized alterations are prohibited.

WARNING

Radiation Protection in Dentistry

 \wedge

Comply with the contents of each clause regarding protection against radiation exposure prescribed in relevant medical regulations when installing and using dental X-ray equipments.

M WARNING

This equipment for radiograph can cause hazard to service personnel if safe exposure conditions and the usages are not complied with.

Be sure to sterilize equipment portions to be touched by patient or operator before starting repair work.

After completing the repair work, sterilize equipment portions touched by service personnel.

While repairing, do not place anything that can be an obstacle within the range of equipment.

To avoid damages to equipment, measuring instruments, etc. and electric shock, service personnel must not remove covers of equipment except when necessary.

M WARNING

To avoid damages to equipment, measuring instruments, etc. and electric shock, turn off power of equipment and take extra care not to short-circuit with other circuit when connecting lead wire of measuring instrument to a circuit within the equipment during repair work.

M WARNING

To avoid damages to equipment, measuring instruments, etc. and electric shock, be sure to turn off power of equipment and use the parts specified by us when replacing machine parts and electric parts of the equipment.

To avoid X-ray exposure due to carelessness, be sure to install lead plate for X-ray protection on X-radiation aperture of X-ray generator when radiating X-rays during repair work.

WARNING

Be sure to use positioning attachment specified for each exposure mode when positioning patient.

WARNING

Be sure to make patient and nursing attendant to wear X-ray protective clothing. (Nursing attendant in this sentence means a person allowed by doctor.)

A

 \wedge

Be sure to operate X-ray exposure switch from outside of X-ray room.

MARNING

Operator must instruct patient not to move while X-raying.

WARNING

Watch patient, nursing attendant and equipment constantly while X-raying, and release X-ray exposure switch immediately if you find something abnormal.

A

/!\

 \mathbb{A}

WARNING

Contact the dealer nearest to you when scraping this equipment.

WARNING

CAUTION

After completing repair work, be sure to turn OFF power for safety.

When X-raying patient after completing repair work, take extra care for patient safety when positioning patient.

Damage etc. inside X-ray generator cannot be repaired on site. Depending on service personnel's judgment, the equipment will be returned to factory for repair or replacement.

Responsibility for managing the use and maintenance of medical equipments lies with the user (hospital or clinic). This equipment must be used by doctor or qualified person only. As repair or check inside equipment involves risks, contact the company you purchased from.

WARNING

When earthquake warning is issued, do not use this equipment. After an earthquake, be sure to conduct maintenance check of the equipment and confirm no abnormality before use. Default of the check and/or confirmation can harm patient.

WARNING

Be sure to set up X-ray room and install the equipment in the X-ray room.

 \mathbb{A}

 \wedge

 \wedge

A

WARNING

Do not place anything that can be an obstacle within the range of equipment movement.



Concerning to X-raying and approach to equipment must be done under the responsibility of user in case a repairer, a patient, or a nursing attendant use a pacemaker etc,

Be sure to use positioning attachment specified for each exposure mode when positioning patient.

MARNING

Be sure to make patient and nursing attendant to wear X-ray protective clothing. (Nursing attendant in this sentence means a person allowed by doctor.)

WARNING

Operator must instruct patient not to move while X-raying.

M WARNING

Be sure to sterilize and disinfect equipment portions touched by patient or operator after X-raying and at daily closing time.

Contact our sales office nearest to you when scraping this equipment.

 \mathbb{A}

WARNING

After using equipment, be sure to turn OFF power for safety.

Keep everyone out from X-ray room except repairer when radiating X-rays for repair.

MARNING

To avoid equipment failure, do not rotate arm by hand.

Do not move patient until arm reset operation completed after X-raying.

 \wedge

 \wedge

CAUTION

Take extra care for patient safety when positioning patient.

Take extra care for patient safety when moving sliding unit up and down.

▲ CAUTION

Do not look straight at positioning laser beam for your safety. Also, give this caution to patient and nursing attendant.

Do not radiate beam line to patient eye

03.Preparation for Repair

1. Manual

* BelmaX CM Service manual

* BelmaX CM Operation manual

2. Measurement Instruments

- 1) Digital Multimeter
- 2) Lead Wire for Measurement
- 3) Measurement Instrument of Insulation Resistance
- 4) X-ray detector paper

3. Tools

- 1) Screwdrivers for Cross Recessed Head Screw (Large, Small)
- 2) Screwdrivers for Slotted Head Screw (Small) (Insulation type)
- 3) Boxing Screwdriver (6,5,4,3mm)
- 4) Hexagon wrench
- 5) Nipper
- 6) Cutting Pliers
- 7) Electrical Soldering Iron (Insulation type), Solder
- 8) Tap
- 9) Tap Handle
- 10) Electric Drill
- 11) Drill
- 12) Crimping tool

4. Tools for adjustment

- 1) Test Piece of BelmaX CM
 - 2) Pb Plate for X-ray protection (thickness = 3mm)
 - 3) Filter (brass plate)
 - 4) Aluminum Filter (thickness = 35mm)
 - 5) X-ray detector paper
 - 6) Adjustment tools for equipment

5. Parts for replacement

1) Prepare parts, printed circuit boards, wire-harness, etc. according to the necessity of repair.

6. Others


- 1) Prepare screws, nut, crimp terminal, grease, etc. according to the repair.
- 2) Alcohol for disinfection
- 3) Clothe
- 4) Chemicals for cleaning

04. Specifications

Model	Digital	Panoramic and Cephal	lometric Radiograph BelmaX CM			
Input Power	120Vac 60Hz 1 <i>ϕ</i>					
Power	2.0kW					
High Tension Generator		High Tension (Generator (100kHz)			
Exposure Method		Ν	lanual			
Tube Voltage		60kV~100	0kV (1kV step)			
Tube Current		2.4.6.8.10.1	2mA (2mA step)			
X-ray Tube		D-052	S B(Toshiba)			
Focal Spot		0.5>	< 0.5mm			
Total Filtration		2.5mm	AI (Min)			
CCD sensor		Both as Panor	rama and Cephalo			
Exposure Mode	Panoramic Maxillary Sinus TMJ	Child Adult Orthoradial	Cephalo — Frontal			
Exposure Time	Panorama Maxillary Sinus	Frontal : 7sec / 12sec : 8sec	Cephalo Lateral : 2.9sec(short time mode) 4sec(normal time mode)			
	TMJ Lateral TMJ Frontal	: 3.0sec(×4) : 3.0sec(×2)	Frontal : 3.2~5sec			
Magnification	Panorama Maxillary Sinus TMJ Lateral TMJ Frontal	: 1.21~1.36 : 1.2~1.22 : appr.1.24 : appr.1.88	Cephalo Lateral & Frontal: 1.1			
Positioning Beam			beams			
Positioning tools	Panorama • Maxilla Chin Rest+ Head H T M J Lateral • Fr	lolding Rod	_			
Weight		417	b(189 kg)			

05. Component

1. Block diagram

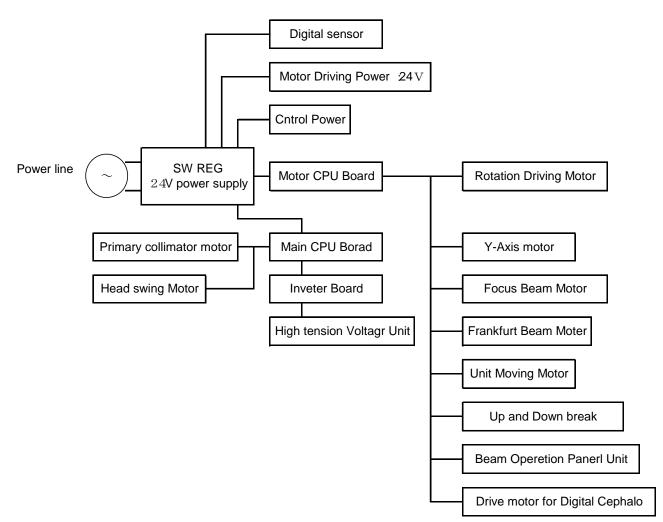

06. Setting

Connect an adjustment tools with the main body first.

Explanation of the setting content

1. Basic operation

- 1) Turn the power on.
- 2) After finishing initialization of the equipment, set Tube Voltage "0kV"
 - Press "SELECT Key" and blink the display of tube voltage
 - · Keep pressing "SELECT DOWN key"
- 3) Change it setting mode.
 - · Press "READY" during pressing "SELECT key"
- 4) Select setting content of the equipment $by T E C H U P \neq D O W N$ key.
- 5) Blink setting display of equipment by "SELECT"
- 6) Decide setting content by SELECT UP/DOWN key
- 7) Release setting mode of the equipment.
 - Press "READY" during pressing "SELECT" key

2. Mode-specific Operations

- 1) With or Without CEPHALO Setting Mode: OFF/R/L
 - · Without Cephalo: OFF
 - · With Cephalo on the left: R
 - · With Cephalo on the left: L
- 2) PREHEAT Voltage Setting Mode: 0 ~ 255
 - Setting of the tube current reference voltage
 - · If decreased the value, the rising of tube current becomes slower.
 - · If increased the value, the rising of tube current becomes faster.
- 3) EXHIBITION ON/OFF Setting Mode: ON/OFF
 - Operation mode for exhibition
 - When set to ON, release the equipment setting and press RESET key, the equipment repeats return-to-origin operation and radiograph operation.
- 4) XRAY COUNT Display Mode

Number of irradiations display mode

- · When press SELECT key and keep pressing SELECT DOWN key,
- the number of irradiation will be cleared.
- 5) POWER OFF ON/OFF Setting Mode: ON/OFF
 - Auto power-off function setting mode
 - When set to ON, the power turns off automatically in five minutes after the final key operation.
 - · When set to OFF, the power will not turn off until turning POWER switch to OFF.

07. Trouble Shooting

1. Unable to Turn On

If the surface of circuit protector is bulging and in a "trip" condition, unplug from outlet, wait for 60 seconds or more, and fully insert the plug into the outlet. The plug can be heat up. Be careful not to get burned.

After eliminating the cause, turn on again.



- Interception of MOTOR CPU BOARD fuse F1(3.15A)
- Check if electric wire is in contact with body of equipment due to braking, pinching, etc.
- Disconnection of wire of rotation axis
- Defect of switching power supply Defect of INV board
- Interception of MAIN CPU BOARD fuse F1(3.15A)

2. Arm doesn't rotate

- Interception of MOTOR CPU BOARD fuse F3(3.15A)
- Defect of motor driver
- Defect of motor

3. Y axis doesn't move

- Interception of MOTOR CPU BOARD fuse F2(3.15A)
- Defect of motor driver
- Defect of motor

4 Horizontal Beam doesn't Move

- Check if it's position for cephalometric radiograph
- Defective REST SW BOARD (when pressing switch makes no sound)
- Defective horizontal beam motor
- Defective micro switch for detecting vertical position
- Defective MOTOR CPU BOARD
- Defective CNK BOARD

5. Focus beam doesn't move

- Is it for Cephalo radiograph position?
- Defect of REST SW BOARD (In case there is no sound when you depress SW)
- Defect of MOTOR CPU BOARD
- Defect of CNK BOARD

6. Body doesn't move up and down

1) When operating Beam Line Operation Panel

- Defective RESET SW BOARD (No sound when press SW)
- Breaking of curl cord
- Defective sliding unit elevation motor driver Remove the top cover, turn on power and check the blinking interval of driver's LED.
- Overload protection function (Number of LED blinks: 2 times)
 - 1. Eliminate the cause that hinders the vertical motion (obstacle, overload)
 - 2. Check if it sounds when the operation button is pressed When it doesn't make any sound, replace BRAKE BOARD
 - 3. Replace the driver
- Open-phase protection function (Number of LED blinks: 3 times)
 - 1. Replace sliding elevation motor
 - 2. Replace the driver
- Over voltage protection function (Number of LED blinks: 4 times)
 Under voltage protection function (Number of LED blinks: 5 times)
- 1. Check if the voltage between MOTOR CPU BARD check pins, CP1 and CP2, is 24Vdc. When the deviation is large, replace SW power supply.
- 2. Replace the driver
- Over speed protection function (Number of LED blinks: 6 times)
 Defective MOTOR CPU BOARD and replacement
 - Defective MOTOR CPU BOARD
 Check that output voltage of CP8 varies while pressing UP/DOWN switch.
 If it does not vary, replace the board.

2) When operating CM switch

- Defective CM SW BOARD (when pressing switch makes no sound)
- Defective sliding unit elevation motor driver Remove the top cover, turn on power and check the blinking interval of driver's LED. After eliminating the cause, turn on power again.
- Defective MOTOR CPU BOARD Check that output voltage of CP8 varies while pressing UP/DOWN switch.
- Defective CNK BOARD

7. Primary Slit Doesn't Move

- Blowout of BEAM MOTOR BOARD fuse F1 (3A)
- Defective BEAM MOTOR BOARD
- Defective motor
- Blowout of MAIN CPU BOARD fuse F3 (3.15A)
- Defective MAIN CPU BOARD

8. Head Doesn't Rotate

- Blowout of BEAM MOTOR BOARD fuse F1 (3A)
- Defective BEAM MOTOR BOARD
- Defective motor
- Blowout of MAIN CPU BOARD fuse F2 (3.15A)
- Defective MAIN CPU BOARD

9. Unable to Irradiate X-rays

- Check if Motor CPU Board D16 and Main CPU Board D36 light in Ready on.
- Check which red LED on INV Board lights

Error content

DL1: LV Lights when output voltage drops abnormally such as output short-circuit.

DL2: OV Lights when output voltage rises abnormally

DL3: OC Lights when output current rises abnormally.

DL4: TH Lights when INV BOARD temperature rises abnormally

Response

- DL1: Check the wiring from INV BOARD to head
 - Check the input voltage between 1 and 2 pins of CN1 when READY ON. 120Vac
- DL2, DL3: Check between MAIN CPU BOARD check pins CP11 and CP5 (kV), CP14 and CP5 (PREHEAT), and CP15 and CP5 (mA).

DL4: Turn off power and recheck 30 minutes later.

	60kV	70kV	80kV	90kV
2mA	2.82	3.27	3.71	4.14
4mA	2.88	3.34	3.78	4.21
6mA	2.94	3.39	3.84	4.31
8mA	3.00	3.45	3.91	4.40
10mA	3.11	3.53	4.00	4.50
12mA	3.19	3.63	4.09	4.60

Table 1: Voltage between kV ref CP11 and CP5

Table 2: Voltage between mA ref CP15 and CP5 (Tube voltage is optional)

Tube Current	CP15-CPCG
2mA	1V
4mA	2V
6mA	3V
8mA	4V
10mA	5V
12mA	6V

08. Treatment of Error Message

When errors occur, error messages will display on operation panel for adjustment.

Take proper treatment for the indication.

1. SENSOR UNIT ERROR

1) Sensor is not installed in the proper place Install sensor in the proper place

2. 1COL MT: First Collimator Motor Operation Error

- 1) Abnormality of first collimator detection sensor Adjust the sensor position or replace the sensor
- Blowout of BEAM MOTOR BOARD F1 (3A) Replace the fuse or BEAM MOTOR BOARD
- 3) Defective motor Replace the motor4) Blowout of MAIN CPU BOARD fuse F3 (3.15A)
 - Replace the fuse. Replace MAIN CPU BOARD.

3. HEAD MT: Head Rotation Motor Operation Error HEAD

1) Abnormality of head rotation motor detection sensor

Adjust the sensor position or replace the sensor. 2) Blowout of BEAM MOTOR BOARD F1 (3A) Replace the fuse or BEAM MOTOR BOARD

3) Defective motor Replace the motor.

 Blowout of MAIN CPU BOARD F3 (3.15A) Replace the fuse. Replace MAIN CPU BOARD.

4. INVERTER: Inverter Error

- 1) Check if MOTOR CPU BOARD D16 and MAIN CPU BOARD D36 light when READY ON.
 - 1. When D16 doesn't light
 - Breaking of electric wire of rotary shaft
 - Defective MOTOR CPU BOARD
 - Defective MAIN CPU BOARD
 - 2. When D36 doesn't light Breaking of electric wire of rotary shaft Defective MAIN CPU BOARD Defective MOTOR CPU BOARD
 - 3. When D16 and D36 light Defective K-L1 Defective MOTOR CPU BOARD

2) Check which red LED of INV BOARD lights.

Error contents

DL1: LV Lights when output voltage drops abnormally such as output short-circuit.

DL2: OV Lights when output voltage rises abnormally

DL3: OC Lights when output current rises abnormally.

DL4: TH Lights when INV BOARD temperature rises abnormally.

Response

DL1: Check the wiring from INV BOARD to head

Check the input voltage between 1 and 2 pins of CN1 when READY ON. 120Vac When the voltage is lower than 120Vac

Breaking of electric wire of rotary shaft

Defective K-L1

DL2, DL3: Check between MAIN CPU BOARD check pins CP11 and CP5 (kV), CP14 and CP5 (PREHEAT), and CP15 and CP5 (mA).

DL4: Turn off power and recheck 30 minutes later.

Table 1. Voltage between kV ref CP11 and CP5

Voltage between kv for er fri dra er e					
	60kV	70Kv	80kV	90Kv	
2mA	2.82	3.27	3.71	4.14	
4mA	2.88	3.34	3.78	4.21	
6mA	2.94	3.39	3.84	4.31	
8mA	3.00	3.45	3.91	4.40	
10mA	3.11	3.53	4.00	4.50	
12mA	3.19	3.63	4.09	4.60	

Table 2 Voltage between mA ref CP15 and CP5 (Tube voltage is optional)

Tube	CP15-CPCG
Current	
2mA	1V
4mA	2V
6mA	3V
8mA	4V
10mA	5V
12mA	6V

5. THERMAL: Tube Temperature Error

1) As the head temperature is abnormally high, leave the equipment for 30 minutes or more before taking next radiograph.

6. RS CPU: Communication Error with MOTOR CPU BOARD

1) Check the continuity of rotary shaft harness.

In case of breaking, replace the harness, use spare wire as a temporary measure. In the case where the continuity exists, replace MOTOR CPU BOARD or MAIN CPU BOARD.

7. Y MT: Y-axis Motor Operation Error

- Abnormality of Y-axis detection sensor Adjust the sensor position or replace the sensor.
- 2) Blowout of MOTOR CPU BOARD F2 (3.15A) Replace the fuse or MOTOR CPU BOARD
- Defective motor driver Replace the motor driver.
- 4) Defective motor Replace the motor.

8. ROT MT: Rotation Motor Operation Error

- Abnormality of rotation detection sensor Adjust the sensor position or replace the sensor.
- Blowout of MOTOR CPU BOARD F3 (3.15A) Replace the fuse or MOTOR CPU BOARD.
- 3) Defective motor driver Replace the motor driver.
- 4) Defective motor Replace the motor.

9. BEAM MT: Indicator Operation Error

When indicator (beam operation portion) is operating

- Replace the sensor. When indicator (beam operation portion) is not operating
- 1) Replace the motor.
- 2) Replace MOTOR CPU BOARD.

10. UNIT MT: Planetary Portion Operation Error

1) When operating Beam Line Operation Panel

- Defective sliding unit elevation motor driver
 - Remove the top cover, turn on power and check the blinking interval of driver's

LED.

After eliminating the cause, turn on power again.

- Overload protection function (Number of LED blinks: 2 times)
 - 1. Eliminate causes that hinder the elevation motion (obstacle, overload)
 - 2.Check if pressing operation button makes a sound of brake.
 - When it makes no sound, replace BRAKE BOARD.
 - 3. Replace the driver
- Open-phase protection function (Number of LED blinks: 3 times)
 - 1. Replace sliding unit elevation motor.
 - 2. Replace the driver.

Over voltage protection function (Number of LED blinks: 4 times)

- Under voltage protection function (Number of LED blinks: 5 times)
- 1. Check if the voltage between MOTOR CPU BOARD check pins CP1 and CP2 is 24Vdc.

- 2. Replace the driver.
- Overspend protection function (Number of LED blinks: 6 times)
 - 1. Defective MOTOR CPU BOARD
 - Check that output voltage of CP8 varies while pressing UP/DOWN switch. If it does not vary, replace the board.


2)When operating CM switch

- Defective CM SW BOARD (when pressing switch makes no sound)
- Defective sliding unit elevation motor driver
- Remove the top cover, turn on power and check the blinking interval of driver's LED. After eliminating the cause, turn on power again. Defective MOTOR CPU BOARD Check that output voltage of CP8 varies while pressing UP/DOWN switch.
- Defective CNK BOARD

09. PRINTED CIRCUIT BOARD LAYOUT DRAWING

1. Overall View


The printed circuit board assemblies are housed in the portions shown in the drawing.

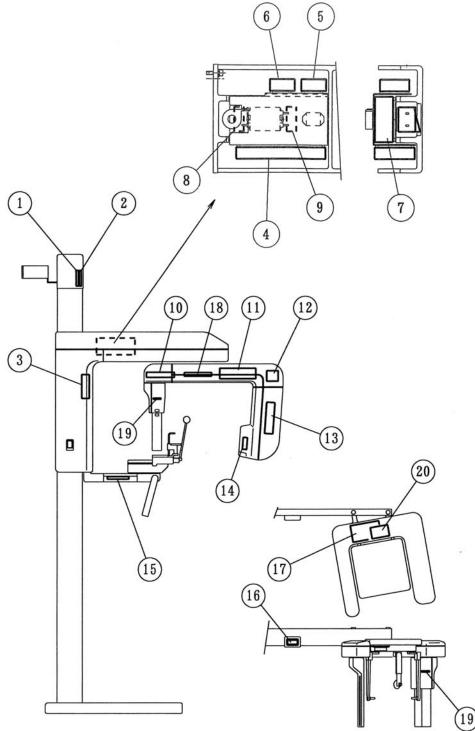

No.	Board Name		
1	DC brushless motor driver		
2	Brake board		
3	CNK board		
4	Motor CPU Board		
5	Rotation motor driver		
6	Y-axis motor driver		
7	Switching power supply		
8	NF1 board		
9	NF2 board		
10	Film motor driver		
11	High frequency inverter power supply		
12	Head swing motor driver		
13	High-voltage generator board		
14	Collimator motor board		
15	Main CPU board		
16	LCD indicator		
17	Switch board		
18	Beam switch board		
19	Vertical motion switch board		

10. Parts List


1. Stand

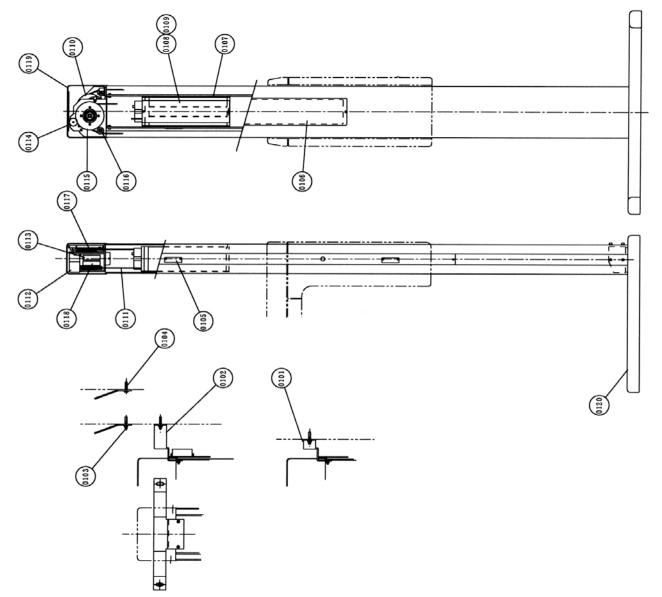


PART LIST: Stand

No	Drwing No	Part Name	Q Hyper-X	ty. Hyper-X CM	Remarks
0101	408-10311	Bracket for securing to wall	1	-	For Hyper-X only
0102	408-10312	Bracket for securing to wall	-	1	For Hyper-X CM only
0103	—	High anchor Ø8´50	2	2	Alternative
0104	-	Coach bolt C8-50	2	2	Allemative
0105	408-10304	Visor	2	2	
0106	408-10305	Stand rear cover	1	1	
0107	308-03621	Weight frame	1	1	
0108	408-10398	Weight (A)	2	3	Silver
0109	408-10399	Weight (B)	1	-	Black
0110	408-10326	Brake	1	1	
0111	408-10307	Wire	1	1	
0112	977-79030-04	Brushless motor AXH015K-20	1	1	
0113	977-79030-08	Motor driver board	1	1	
0114	408-10315	Gear (Small)	1	1	
0115	408-10314	Gear (Large)	1	1	
0116	408-10318	Pulley (2)	2	2	
0117	408-10313	Pulley (1)	2	2	
0118	408-10321	Brake board	1	1	
0119	408-09036-02	Stand top cover	1	1	
0120	208-01776-03	Stand bottom cover	1	1	

2. Sliding Unit

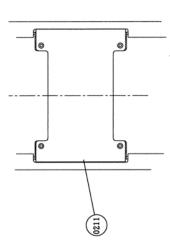


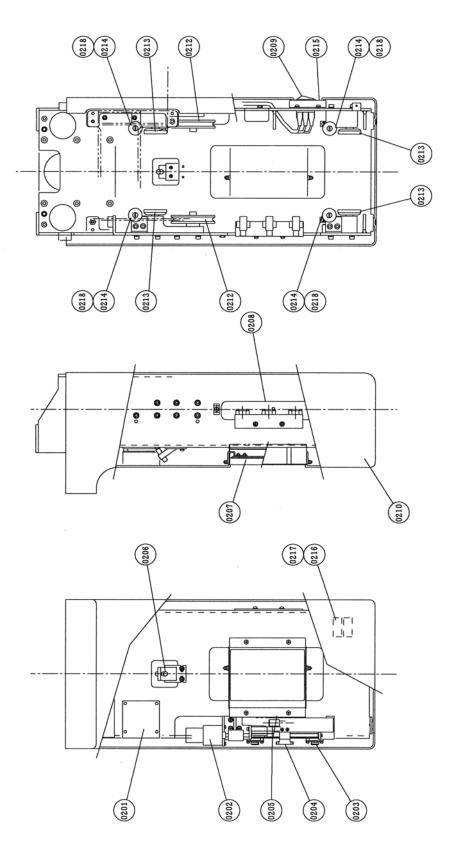

No	Drwing No	Part Name	Q Hyper-X	ty. Hyper-X CM	Remarks
0201	408-10156	CNK board	1	1	
0202	977-60001-07	DC motor TG-38E-LG66	1	1	
0203	964-30012-13	Microswitch ABJ241441	2	2	
0204	408-10296	Dog	1	1	
0205	962-05006	Laser marking projector LDV167LS	1	1	Horizontal beam
0206	962-05003	Laser marking projector LDV167LA	1	1	Median beam
0207	408-08591	Mirror	1	1	
0208	978-60007-02	Photomicrosensor EE-SY671	3	3	
0209	964-28001-02	POWER switch A8G-107-1G-24	1	1	
0210	108-01181	Sliding unit cover	1	1	
0211	408-10309	Rear cover	1	-	
0212	408-09751	Pulley	2	2	
0213	408-05331	Shaft	4	4	Bundled with 0218
0214	408-09751	Roller (2)	4	4	
0215	408-10253	Membrane sheet	1	1	
0216	965-85012-69	Circuit protector 15A	2	2	For 100V-spec.
0217	965-85012-67	Circuit protector 8A	2	2	For 200V-spec.
0218	030-03000-01	Bearing 6000ZZ	4	4	Bundled with 0213


PART LIST: Sliding Unit


3. Rotation Unit

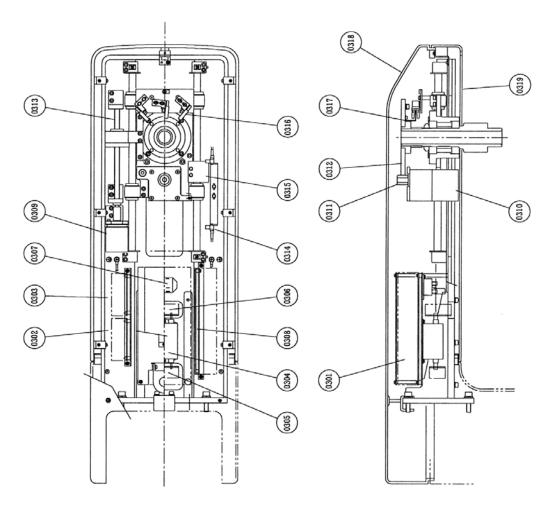

No	Drwing No	Part Nam e	Hyper-X	ty. Hyper-X CM	Rem arks
0301	974-80057	Switching power supply AWM-1C150S-	1	1	
0302	408-09935-02	Motor driver board	1	1	Y-AX IS
0303	408-09935-03	Motor driver board	1	1	ROTATION
0304	975-00003-03	Noise filter GT-215FJ	1	1	
0305	408-08757-00	NF(1) board	1	1	
0306	408-08758	NF(2) board	1	1	
0307	965-60041-17	Relay JM1aN-ZTM-DC24V	2	2	
0308	408-10151	Motor CPU board	1	1	
0309	977-79021-02	Stepping motor PK543-NA	1	1	Y-AXIS
0310	977-79024-03	Stepping motor PK564AN-TG20	1	1	ROTATION
0311	408-10340	Gear (Small)	1	1	
0312	308-02560	Gear (Large)	1	1	
0313	408-09682	Ball screw	1	1	
0314	978-60004-04	Photomicrosensor EE-SX671	2	2	Y-AX IS
0315	408-10345	Sensor fin (Y)	1	1	Y-AXIS
0316	978-60004-10	Photomicrosensor EE-SX673	1	2	ROTATION
0317	408-10343	Sensor fin (R)	1	1	ROTATION
0318	108-01182	Rotation unit cover (Top)	1	1	
0319	108-01183	Rotation unit cover (Bottom)	1	1	

PA<u>RT LIST: Rotation Unit</u>

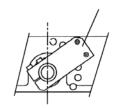


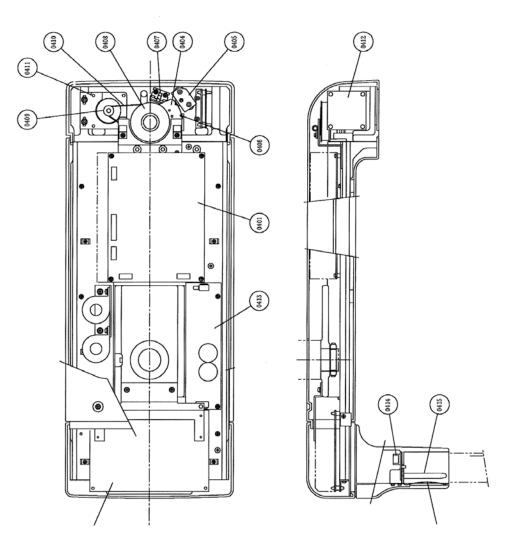

PART LIST:Arm

No	Drwing No	Part Nam e	Q Hyper-X	ty. Hyper-X CM	R em arks
0401	974-10056-01	High-frequency inverter power su	1	1	
0402	408-10145	MAN CPU BOARD	1	1	
0403	408-10423	Stationary plate	1	-	ForHyper-X only
0404	408-10416	Angle plate		1	ForHyper-XCM only
0405	937-50022-04	Plunger PBF-10-KSN	_	1	ForHyper-XCM only
0406	408-10419	Sensor fin		1	ForHyper-XCM only
0407	978-60004-07	Photomicrosensor EE-SX672		2	ForHyper-XCM only
0408	408-10415	Pulley (Large)	-	1	ForHyper-XCM only
0409	408-10414	Pulley (Small)		1	ForHyper-XCM only
0410	932-21005-18	Timing belt B 100 S 3M 252			ForHyper-XCM only
0411	977-79031-05	Stepping motor PK266M-02A	_	1	ForHyper-XCM only
0412	408-07586-00	Motor driver board		1	ForHyper-XCM only
0413	974-80058-07	Switching power supply LDA50	1	1	
0414	408-10450	CNP BOARD	1	1	
0415	408-10483	Shaft	1	1	
0416	408-10486	Plate spring	1	1	


4. Chinrest Unit

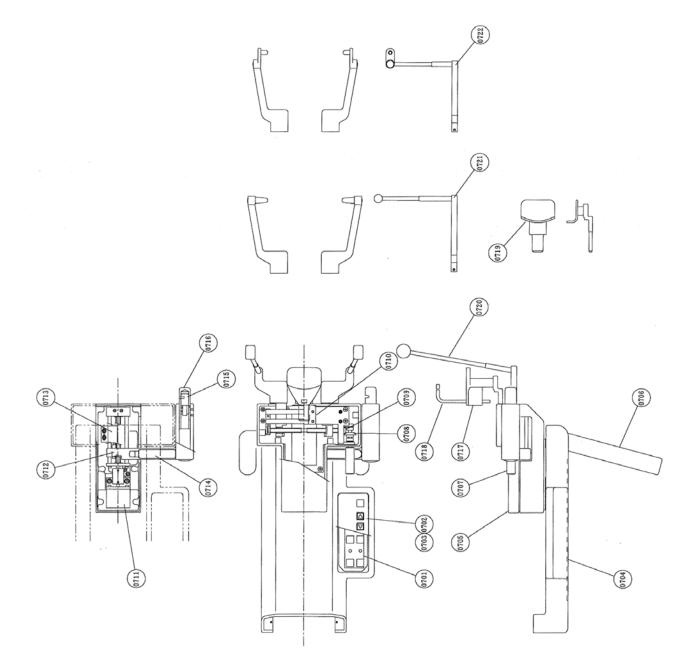

PART LIST: Chinrest Unit

No	Drwing No	Part Nam e	Q ty. Hyper-X Hyper-X CM	R em arks
0701	408-10152	Switch board	1	
0702	408-10379	Frame	1	
0703	408-10331	Membrane sheet	1	
0704	408-10381	Bottom cover	1	
0705	108-01188	Holding portion cover	1	
0706	408-10380	Handle	2	
0707	408-06071	Knob	1	
0708	408-06081	Gear (Large)	1	
0709	408-06082	Gear (Small)	1	
0710	408-10396	Holder guide	1	
0711	977-79010-05	Stepping motor	1	PX243-03A
0712	978-60011	Proximity sensor	1	APS4-12S
0713	408-10387	Sensor fin	1	
0714	408-10388	Lid	1	
0715	962-05006	Laser marking projector	1	LDV167LS
0716	408-09316	Beam projector mount	1	
0717	308-02261	Rest for bite	1	
0718	408-05861	Bite fork	1	
0719	408-05083-02	Rest for maxillary sinus	1	
0720	308-03599-01/-02		1set	
0721		Ear rod for TMJ lateral	1set	
0722	308-03601-01/02	Ear rod for TMJ frontal	1set	

6. Head

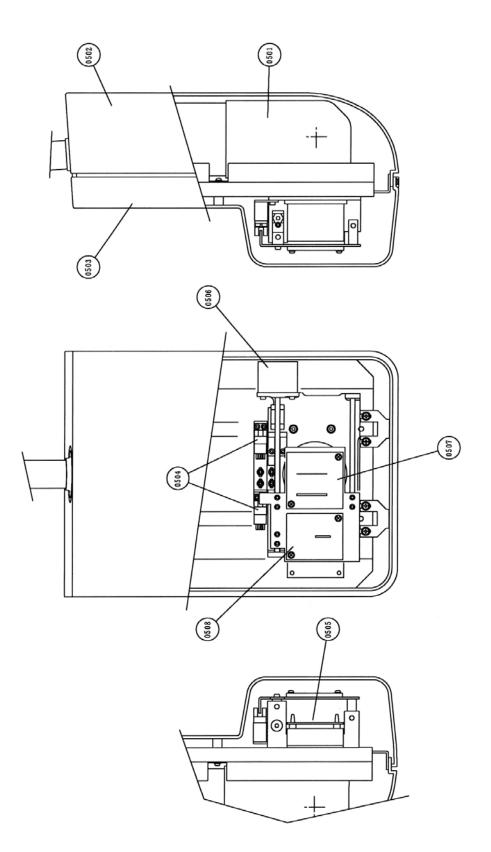

PART LIST:Head

			Q ty.			
No	Drwing No	Part Nam e		CM R()	CM(L)	Rem arks
			Hyper−x	Hyper-X CM	Hyper-X CM	
0501	108-01177	X-ray generator Assy	1	1	1	
0502	108-01179	Generator cover (Rear)	1	1	1	
0503	108-01178-01	Generator cover (Front)	1	1	1	
0504	978-600004-13	Photomicrosensor	1	2	2	
0505	408-07586-02	Motor driver board	1	1	1	
0506	977-79010-05	Stepping motor	1	1	1	
0507	408-10478	Mask (Panorama)	1	1	1	
0508	408-10479	Mask (Cephalo)	_	1	1	Hyper-X CM

- **T** (L000) .
- 7. Cephalometric Radiography Unit

			Q	ty.		
No	Drwing No	Part Nam e	CM R()	CML()	Rem arks	
			hyper-X CM	hyper-X CM		
0801	408-10243	Mounting plate	1	1		
0802	308-03624-01	Aım	1	1		
0803	308-030652	Case	1	1		
0804	408-10246	Membrane sheet	1	1		
0805	408-1062	CM SW BOARD	1	1		
0806	408-05027	Ear rod arm	2	2		
0807	408-05028	Ear rod	2	2		
0808	308-03627	Forehead Holding Rod (2)	1	1		
0809	308-03626	Second slit	1	1		
0810	408-02531-03	Label	1	1		
0811	988-50054	CCD sensor	1	1		

PART LIST: Cephalom etric Radiography Unit

11. CONTACT

Please contact our sales office or a distributor near you.

Belmont Equipment

A Division of TAKARA BELMONT USA, Inc. 101 Belmont Drive Somerset, NJ 08873 Toll Free (800) 223-1192 Toll Free Fax (800) 280-7504 www.belmontequip.com

TAKARA COMPANY, CANADA, LTD.

2706 South Sheridan Way Mississauga, Ontario, Canada L5J 2M4 Toll Free (800) 268-5351 Fax (905) 822-6203 www.takarabelmont.ca

12. MAINTENANCE CHECK

High voltage is applied to some parts inside the equipment. Take extra care when removing the case of equipment.

Before beginning repair work, turn OFF the power of equipment and unplug from electrical outlet. After unplugged the equipment, wait for 10 minutes or more before starting a repair work.

Specialized knowledge, experience and special measuring instruments are required to check this equipment. To maintain the performance of equipment, please perform daily check (with eyes) and implement periodic maintenance by dealer service personnel.

		Maintenance by service personnel: 1 or 2 tim	es/year	
Check Item	Purpose of Check	Contents of Check	Method	
Electricity Condition	Check power supply voltage range	Check the compatibility between the power supply voltage connected to the equipment and the specified voltage for the equipment.		
Appearance and Indication	(1) Appearance of equipment	a. Deformation, flaw, nameplate b. Cautions	0	
Indication	(2) Inside of equipment	Check defacement and dust.	0	
	Temperature, humidity, gas	Check the compatibility of environmental ambient of the place where the equipment will be used.	•	
	Levelness of equipment	Check the effect on the equipment	•	
Installation Place	Floor and equipment stabilizing condition	Check vibration and movement stability of the equipment.		
	Check obstacles	bstacles Check that there is no obstacle within the movement range of the equipment.		
	Rust development condition	Check the condition of rust development functional safety.	0	
Operation Test	Operation test before check	Check operating condition of the equipment.	۲	
Operation Test	Operation test after check	Check operating condition of the equipment.	۲	
	Insulation resistance	Check resistance between power line and the earth.	•	
Safety Test for	Leaked current from the outer cover	Check the current leaking from the outer cover of the equipment to the earth.	•	
Electric Shock	Earthing resistance	Check the resistance between exposed metal portion of the equipment and the earthing point.	•	
	Resistance of earthing wire	Check the resistance between the earth terminal of the equipment and earthing point.	•	

Dental X-ray Equipment Maintenance Check List

Check Item	Purpose of Check	Contents of Check		
	Check operation of power supply circuit	Check the voltage of power supply circuit.		
	Check operation of radiographic circuit	Check the operating waveform and setting values of control circuit.		
Operation Accuracy of	Check operation of operation circuit	Check operation of operation sequence.		
Equipment	Accuracy of positioning mechanism	Check deterioration of the positioning mechanism.		
	Check operation of protection circuit	Check the setting values and operating condition of protection circuit.		
	Check operating condition indicator	Check the circuit function to indicate the operating condition.	•	
ndication during X-ray irradiation X-ray and the indication are in sync		Check the operation of indicator during X-ray irradiation.	۲	
	X-ray tube housing	Check leakage of insulating oil.		
	Low voltage cable	Check wear, flaw, tension and twist.	0	
X-ray Generator	Irradiation cone	Check looseness, deformation and damage.	0	
	Filter	Check coming off and damage.		
	Slit Plate (Collimator)	Check off-alignment of irradiation field and irradiation width.		
Radiographic	Rotation and movement of X-ray generator Movement of arm	Check slip, abnormal sound and stopping accuracy.		
Mechanism	Patient positioning mechanism	Play, looseness, operationality and stability.		
	Light for positioning	Check the brightness and position accuracy.		
	Wire rope	Check breaking of wire and end portion.	0	
Floyeting	Braking mechanism	Check the movement.		
Elevating Mechanism	Electromagnetic lock			
	Upward and downward movement condition	Check smoothness of movement		
	X-ray tube voltage	Check X-ray tube voltage.		
X-ray Output	X-ray tube current	Check X-ray tube current.		
	Exposure time	Check exposure time.		

Symbol	Checking Method	
0	Check with eyes	
 Check by using measuring instrumen 		
Check by operating the equipment.		