



Bel-Halo DENTAL LIGHT

PARTS LIST

(D240 Series)


TABLE OF CONTENTS

	PAGE
1 . HEAD SECTION -----	4
2 . ARM SECTION (D241, D242) -----	6
3 . ARM SECTION (D245, D247, D248) -----	8
4 . ARM SECTION (D247, D248) -----	10
5 . ARM SECTION (D246G-QU/EV) -----	12
6 . ARM SECTION (D249) -----	14
7 . POWER SUPPLY BOX (D241) -----	16
8 . POWER SUPPLY BOX (D242) -----	18
9 . POWER SUPPLY BOX (D245) -----	20
10 . POWER SUPPLY BOX (D247) -----	22
11 . POWER SUPPLY BOX (D248) -----	24
12 . POWER SUPPLY BOX (D246G-QU) -----	26
13 . SWITCH BOX (D246G-QU)-----	28
14 . TRANSFORMER BOX SECTION (D246G-EV, D249) -----	30
15 . SWITCH BOX (D246G-EV, D249W-X)-----	32
16 . SWITCH BOX (D249W-XN)-----	34
17 . TROLLEY AND RAIL SECTION (D245) -----	36
18 . CEILING POLE SECTION (D242) -----	38


WIRING DIAGRAM

1 . AL-D241G / AL-D241W -----	40
2 . AL-D242G -----	41
3 . AL-D245G -----	42
4 . AL-D247G -----	43
5 . AL-D248G -----	44
6 . AL-D246G-QU-----	45
7 . AL-D246G-EV / AL-D249W-X*-----	46
8 . AL-D246G-EV-----	47

NO.	PARTS NO.	DESCRIPTION	QTY	APPLICABLE SERIAL NO.	NOTE
1	1P03KQA0	YOKE WITH BUSH (CN-80)	1		
	1P034FA0	YOKE WITH BUSH (RAL9003)			
2	1P03GFA0	SECOND YOKE WITH ROLL PIN (CN-80)	1		
	1P03M3A0	SECOND YOKE WITH ROLL PIN (RAL9003)			
3	1A0WFYA0	OUTER FRAME COVER (CN-80)	1		
	1A0WFXA0	OUTER FRAME (RAL9003)			
4	1G095TA0	LED MODULE	1		
5	1A07JHB0	LENS COVER	1		
6	1A07JJA0	REFLECTOR	1		
7	1A07JKA0	OUTER FRAME COVER (CN-80)	1		
	1A081NA0	OUTER FRAME COVER (RAL9003)			
8	1A07JLA0	YOKE COVER (CN-80)	1		
	1A081MA0	YOKE COVER (RAL9003)			
9	1A07JMB0	LED DISPLAY COVER	1		
10	1A07JNC0	VOLUME GRIP	1		
11	1A07JPB0	YOKE CAP (CN-80)	2		
12	1A07JQA0	YOKE PACKING (CN-80)	1		
13	1A07JRB0	HEAD ROTATION SHAFT (FIXED)	1		
14	1A07JSB0	HEAD ROTATION YOKE SHAFT (ADJUSTMENT)	1		
15	1A07JTB0	NUT FOR PATTERN ROTATION	1		
16	1A07JUA0	SLIDE WASHER FOR YOKE ROTATION	2		
17	1A07JVA0	SLIDE WASHER FOR HEAD ROTATION SHAFT	1		
18	1A07JWA0	SENSOR COVER	1		
19	1A07JYA0	SLIDE WASHER	1		
20	1A0829B0	BELMONT LOGO LABEL	1		
21	1A07K0A0	PLATE FOR PATTERN ROTATION	1		
22	1A07L6B0	HANDLE (CN-80)	2		
23	1A07M9A0	VOLUME PART OF INTENSITY CONTROL WITH HARNESS	1		
24	1G027ZC0	LED DISPLAY PCB	1		
25	1G027TD0	SENSOR PCB ASSY	1		
26	CAHC89A0	WASHER 24x14.2x0.5	2		
27	CEFN35A0	ROLE PIN WAVEFORM M2.5x6 SUS	1		
28	CEIA10A0	WAVE WASHER	2		
29	CENX96A0	WAVE WASHER	2		
30	ZA111JPY	PAN HEAD SEMS SCREW M3x8 CHROME	7		
31	CAGP69B0	YOKE STOP SCREW	1		
32	1G027VA0	YOKE COVER ASSY (CN-80)	1		
	1G02E1A0	YOKE COVER ASSY (RAL9003)			
33	CGGN66A0	YOKE SHAFT SET	1		
34	1P072GA0	LIGHT HEAD ASSY (CN-80)	1		
	1P072FA0	LIGHT HEAD ASSY (RAL9003)			

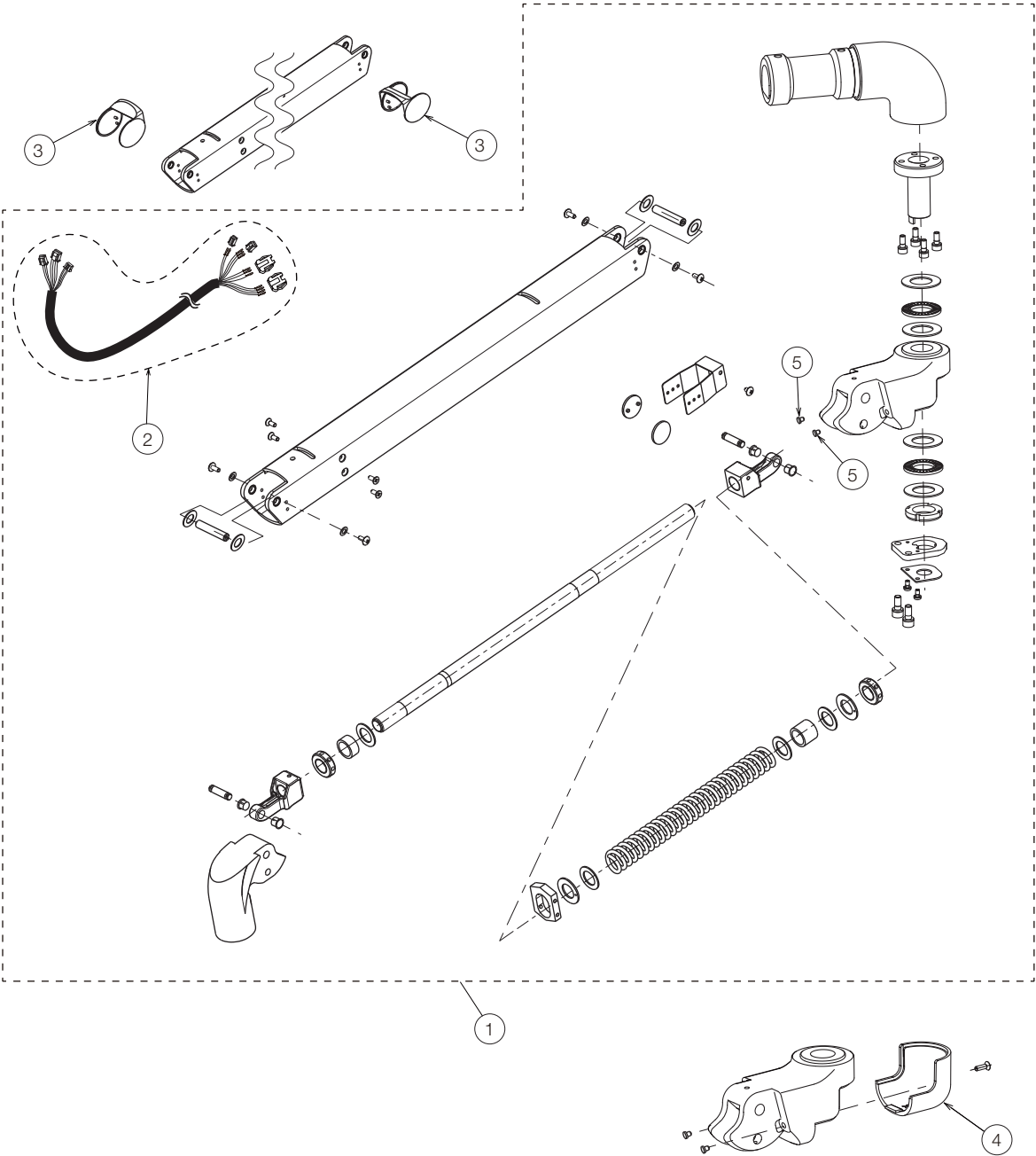


NO.	PARTS NO.	DESCRIPTION	QTY	APPLICABLE SERIAL NO.	NOTE
1	1P078EA0	BALANCE ARM ASSY FOR POST/CEILING MOUNT TYPE (CN-80)	1		
	1P078FA0	BALANCE ARM ASSY FOR POST/CEILING MOUNT TYPE (RAL9003)			
2	1A087KA0	LIGHT CABLE IN ARM (D241/D242)	1		
3	CARA01A0	ROTATION PART COVER	2		
4	1A07K8A0	JOINT COVER (CN-80)	1		
	1A0821A0	JOINT COVER (RAL9003)			
5	CAMP65A0	LARGE STOPPER (6 NYLON)	2		

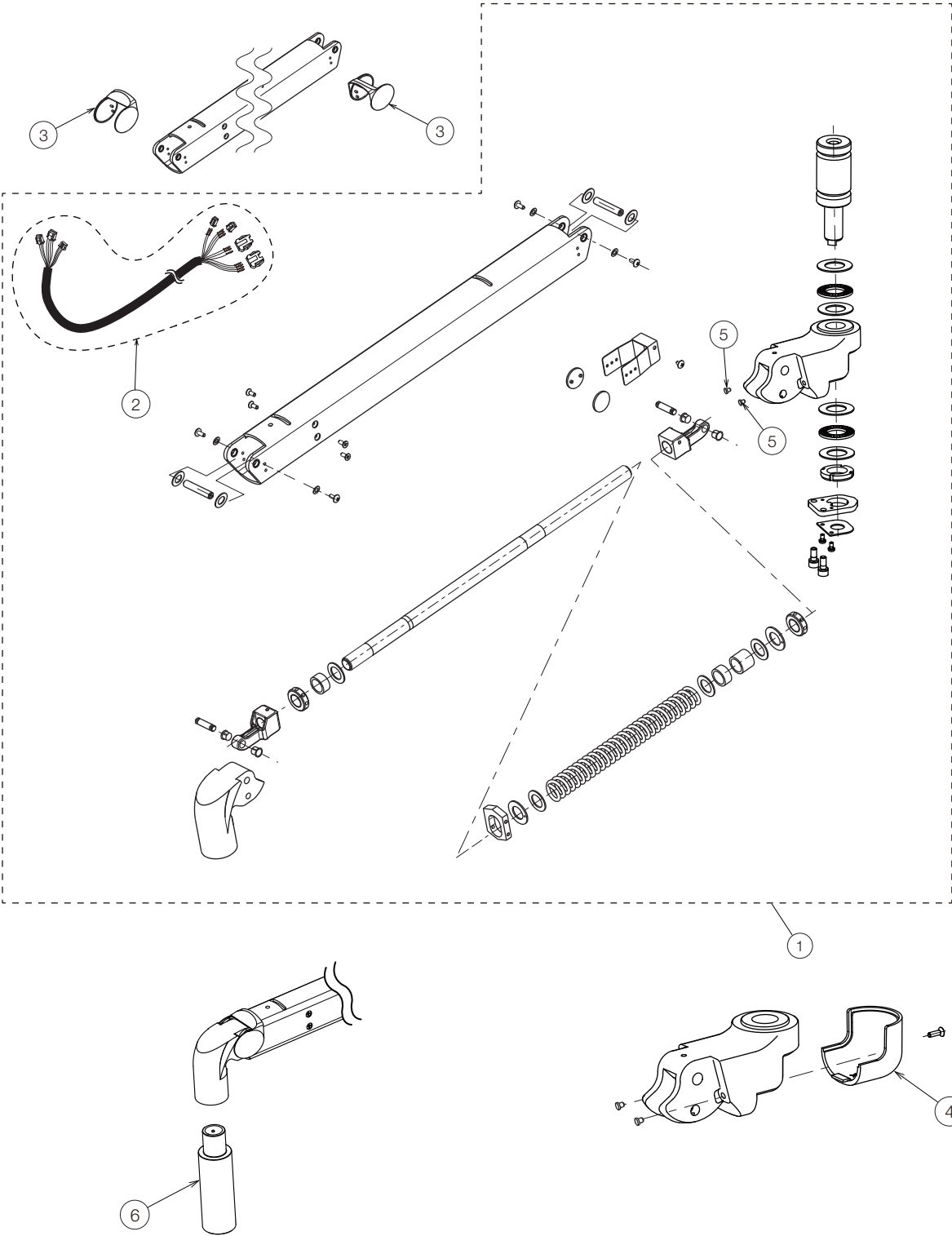


NO.	PARTS NO.	DESCRIPTION	QTY	APPLICABLE SERIAL NO.	NOTE
1	1P039QA0	BALANCE ARM ASSY (CN-80)	1		
2	1A087SA0	LIGHT CABLE IN ARM	1		
3	CARA01A0	ROTATION PART COVER	2		
4	1A07K8A0	JOINT COVER (CN-80)	1		
5	CAMP65A0	LARGE STOPPER (6 NYLON)	2		
6	1A084SB0	EXTENSION (CN-80)	1		


FOR D247

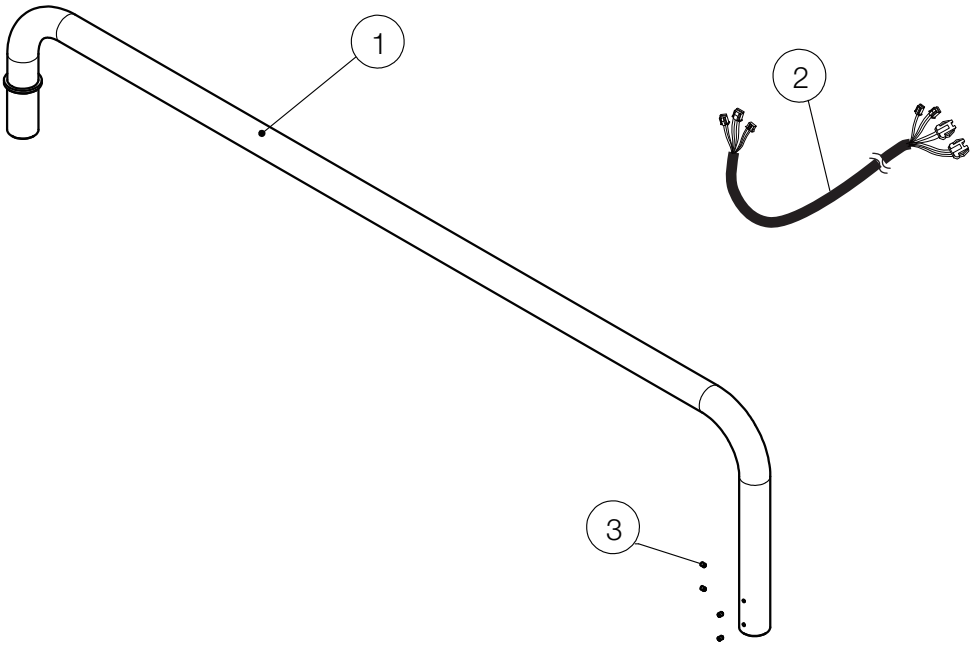


FOR D248

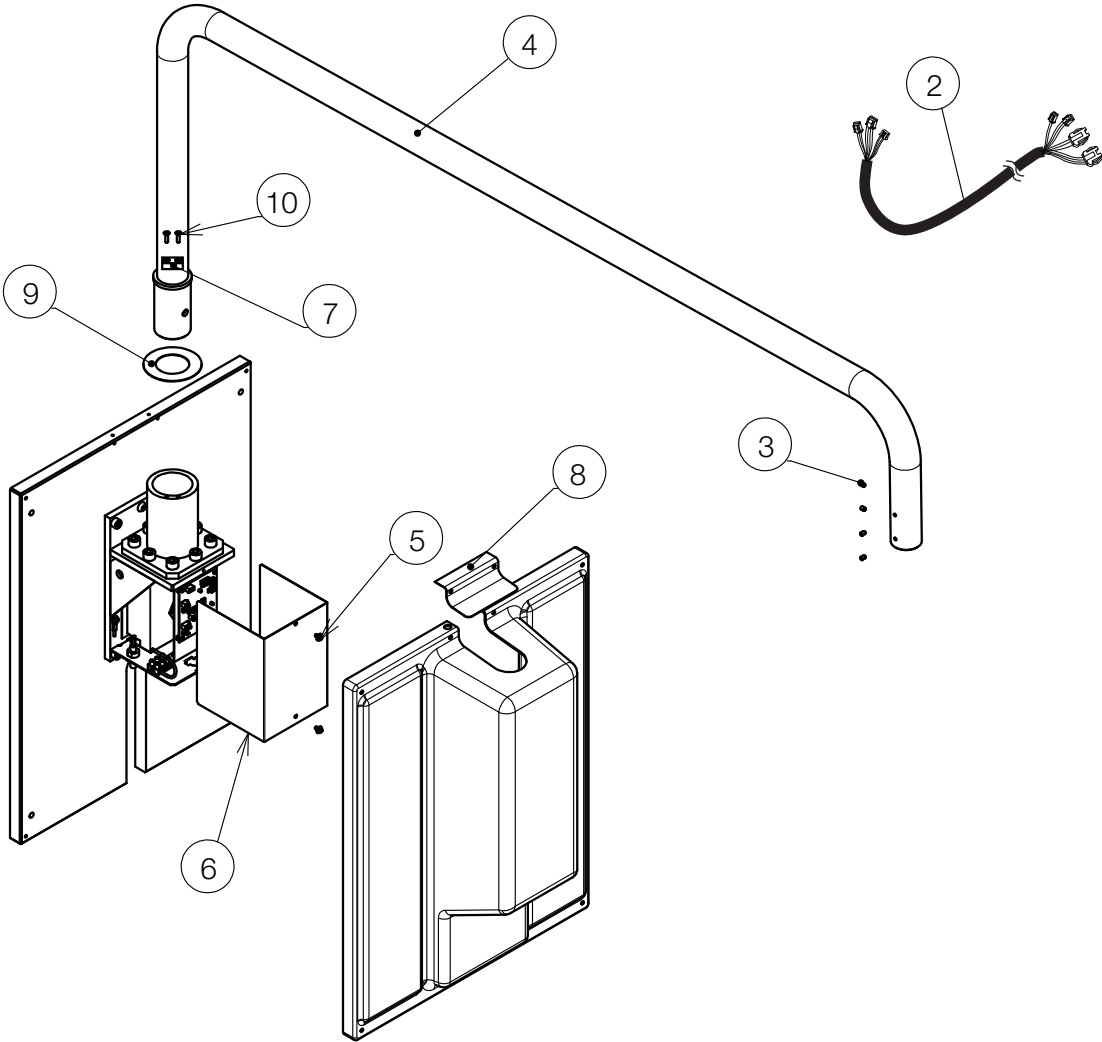


NO.	PARTS NO.	DESCRIPTION	QTY	APPLICABLE SERIAL NO.	NOTE
1	1A0AH6A0	LIGHT POLE FOR CABINET MOUNT(CN-80)	1		
2	1A09GUB0	LIGHT CABLE IN ARM (D247)	1		
	1A09GNA0	LIGHT CABLE IN ARM (D248)			
3	ZA772NPY	HEX SOCKET SET SCREW M5X8 CHROME	4		
4	1A0ALVA0	LIGHT POLE FOR WALL MOUNT (CN-80)	1		
5	ZA88JLPY	PAN HEAD SEMS SCREW 3P M4X8 CHROME	2		
6	CATP03B0	COVER FOR BRACKET	1		
7	CATP04A0	RETAINER METAL	1		
8	1A0ALZA0	FRONT TOP COVER	1		
9	CATN92A0	POLE DISK	1		
10	ZA112L#U	FLAT HEAD SCREW M4X15 SUS	2		

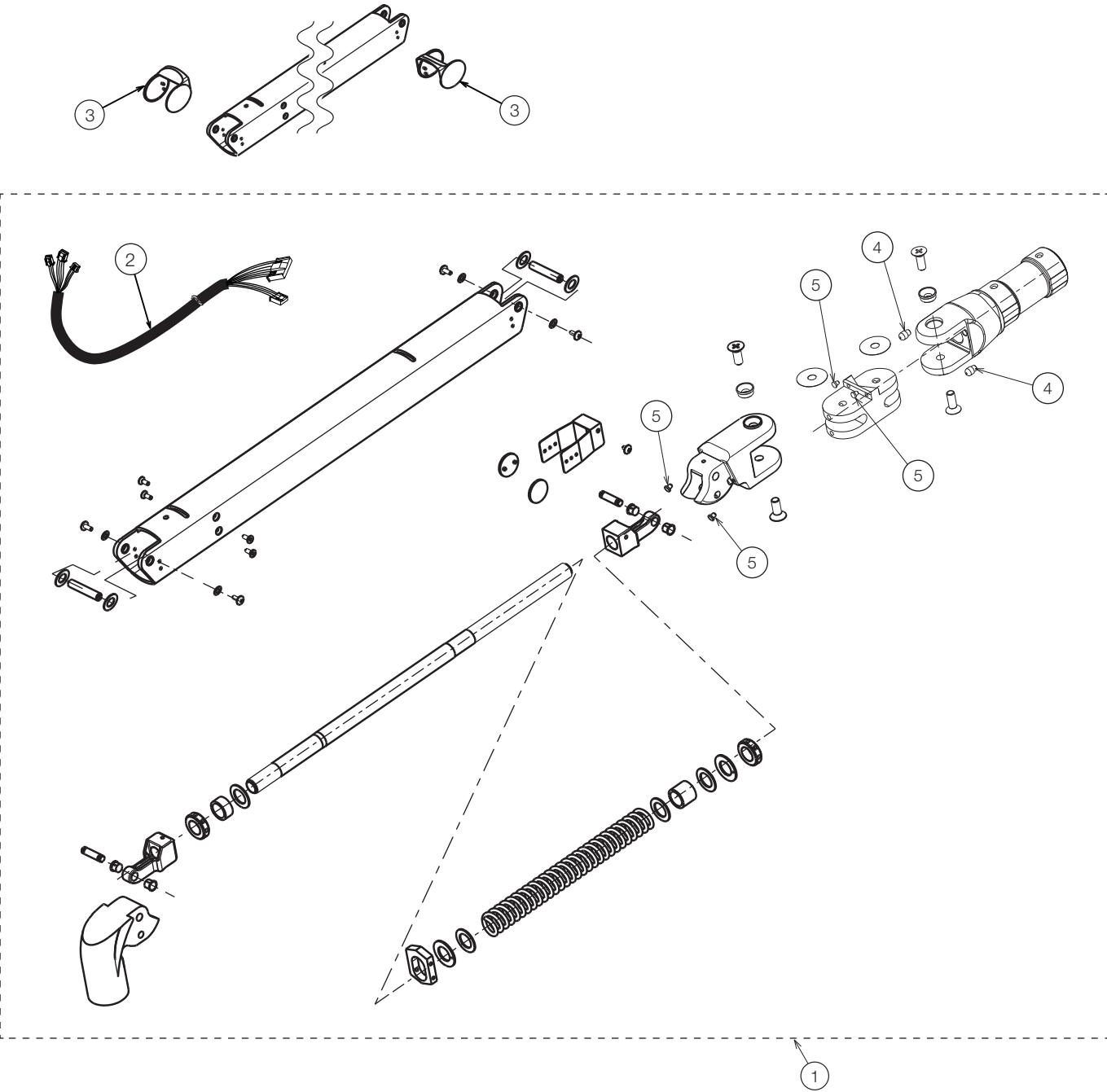




NO.	PARTS NO.	DESCRIPTION	QTY	APPLICABLE SERIAL NO.	NOTE
1	1P078BA0	BALANCE ARM ASSY FOR UNIT MOUNT TYPE D246G-QU (CN-80)	1		FOR QUOLIS
	1P078CA0	BALANCE ARM ASSY FOR UNIT MOUNT TYPE D246G-EV (CN-80)			FOR EVOGUE
2	1A086VA0	LIGHT CABLE IN ARM FOR D246G-QU	1		FOR QUOLIS
	1A089TB0	LIGHT CABLE IN BALANCE ARM FOR D246G-EV/ D249W-X,XN			FOR EVOGUE
3	CARA01A0	ROTATION PART COVER	2		
4	CATP10A0	STOPPER RL L=7.5 (6 NYLON)	2		
5	CAMP65A0	LARGE STOPPER (6 NYLON)	4		

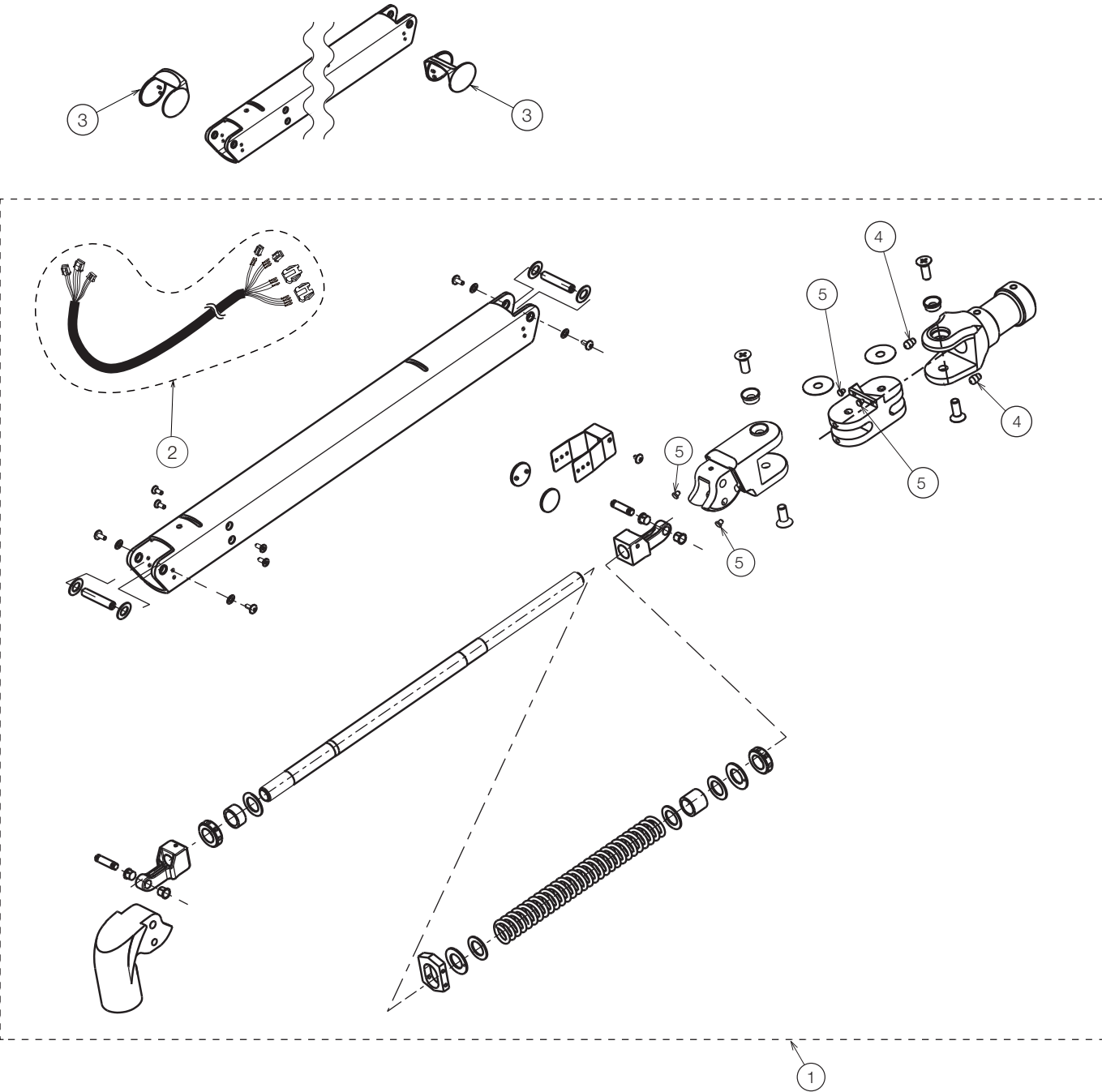


NO.	PARTS NO.	DESCRIPTION	QTY	APPLICABLE SERIAL NO.	NOTE
1	1P078DA0	BALANCE ARM ASSY FOR UNIT MOUNT TYPE (RAL9003)	1		FOR X-CALIBUR
2	1A089TB0	LIGHT CABLE IN BALANCE ARM FOR D246G-EV/ D249W-X,XN	1		
3	CARA01A0	ROTATION PART COVER	2		
4	CATP10A0	STOPPER RL L=7.5 6 NYLON	2		
5	CAMP65A0	LARGE STOPPER (6 NYLON)	4		

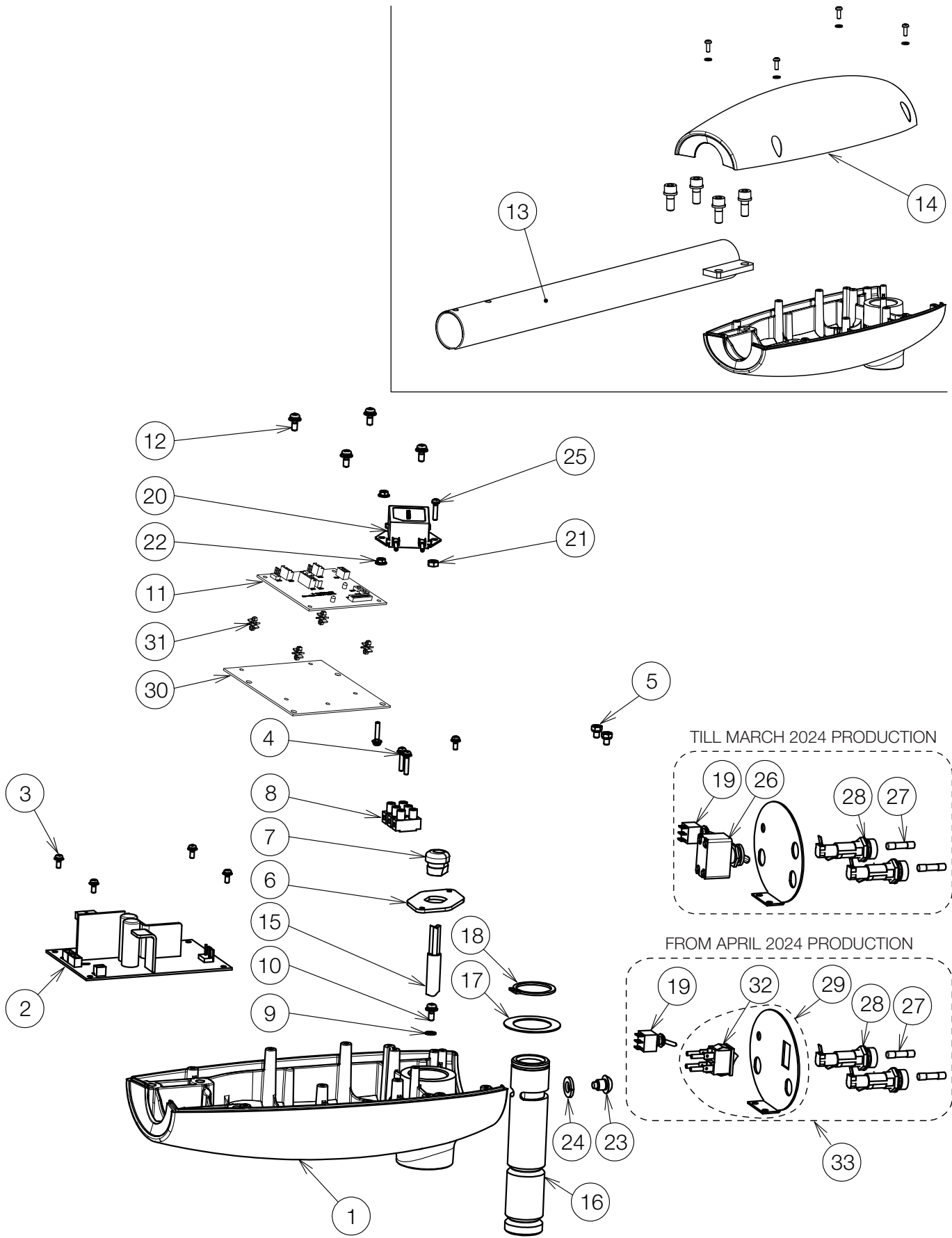


NO.	PARTS NO.	DESCRIPTION	QTY	APPLICABLE SERIAL NO.	NOTE
1	1A07PGA0	TRANSFORMER BOX (CN-80)	1		
	1A082HA0	TRANSFORMER BOX (RAL9003)			
2	1G08K8A0	SWITCHING POWER PCB	1		
3	ZA88JJPY	PAN HEAD SEMS SCREW 3P M3x8 CHROME	5		
4	ZA88JJUY	PAN HEAD SEMS SCREW 3P M3x16 CHROME	3		
5	1E00UKA0	UPSET HEAD BOLT 2P M4X6 CHROME	2		
6	CADQ35A0	CODE BUSH PANEL (70)	1		
7	CEGN46A0	CODE BUSH	1		
8	CAML90B0	CODE HOOK TERMINAL BLOCK	1		
9	ZB552LAY	EXTERNAL TOOTH WASHER M4 CHROME	1		
10	ZA88JLRL	PAN HEAD SEMS SCREW	1		
11	1G0803E0	LIGHT CONTROL PCB	1		
12	ZA88JLRY	PAN HEAD SEMS SCREW 3P M4x10 CHROME	4		
13	1A07PLA0	SWIVEL ARM (CN-80)	1		
14	1A07PHA0	TRANSFORMER BOX COVER (CN-80)	1		
	1A082JA0	TRANSFORMER BOX COVER (RAL9003)			
15	1A0XNAA0	POWER CODE	1		
16	1A07W1B0	ROTATION SHAFT	1		
17	CAND32A0	WASHER 43X29X0.8	1		
18	CENX94A0	C-RING	1		
19	1A0ZV0A0	SENSOR/MANUAL MODE SWITCH ASSY WITH WIRE	1		PREVIOUS CODE: 1A0STYB0
20	1G0A33A0	NOISE FILTER	1		TILL MARCH 2024 PRODUCTION
	1G0ASLA0	NOISE FILTER			FROM APRIL 2024 PRODUCTION
21	ZA812LAY	HEX NUT M4 CHROME	1		
22	ZA901JAY	FLANGE NUT M3 CHROME	2		
23	1A07W2A0	ROTATION STOPPER BOLT M8	1		
24	ZB511PAY	SPRING WASHER M8 CHROME	1		
25	ZA887JTY	PAN HEAD SEMS SCREW 2P M3x14 CHROME	1		
26	-	DISCONTINUED. FOR REPLACEMENT, PLEASE ORDER NO.29.	1		TILL MARCH 2024 PRODUCTION
27	CEEN94A0	FUSE 250V 0.8A (T)	2		
28	AEFR93B0	FUSE HOLDER 250V-10A	2		
29	1P080EA0	ROCKER SWITCH WITH PLATE AND HARNESS (D241/CN80)	1		TILL MARCH 2024 PRODUCTION
	1P080DA0	ROCKER SWITCH WITH PLATE AND HARNESS (D241/RAL9003)			
30	1A0XGTA0	BRACKET FOR CONTROL PCB	1		
31	CERN39A0	DOUBLE LOCKING SPACER	4		
32	1E09JTA0	ROCKER SWITCH	1		FROM APRIL 2024 PRODUCTION
33	1G09CRA0	BACK PLATE ASSY FOR ROCKER SWITCH SPEC. (D241) (CN80)	1		FROM APRIL 2024 PRODUCTION
	1G09CTA0	BACK PLATE ASSY FOR ROCKER SWITCH SPEC. (D241) (RAL9003)			

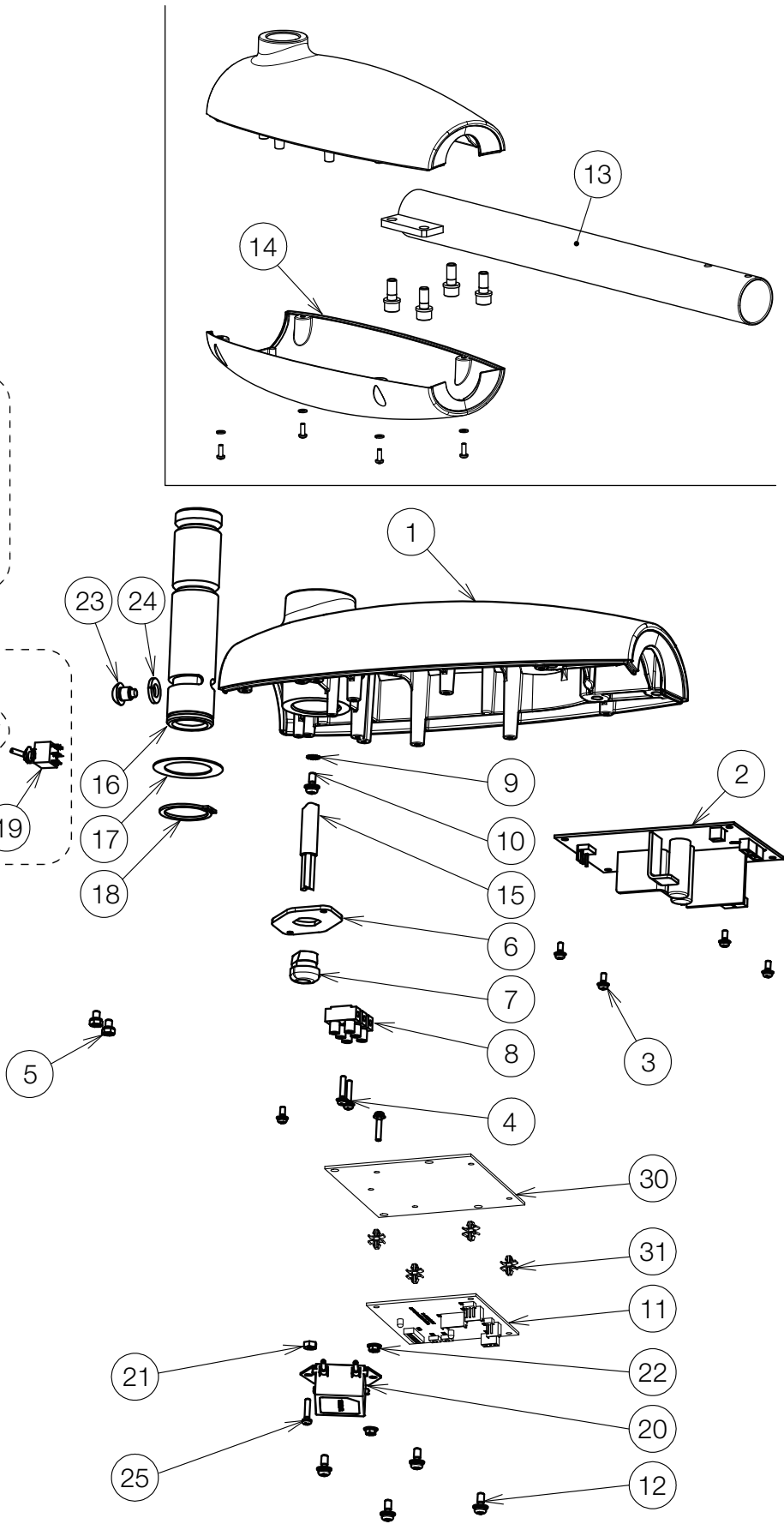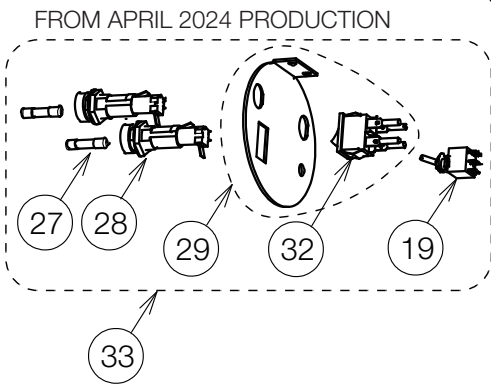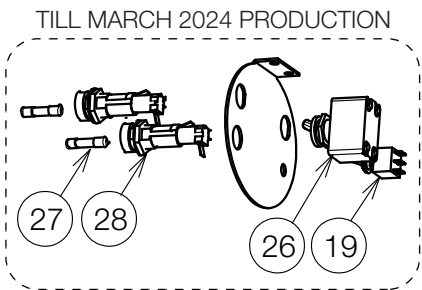


NO.	PARTS NO.	DESCRIPTION	QTY	APPLICABLE SERIAL NO.	NOTE
1	1A07PGA0	TRANSFORMER BOX (CN-80)	1		
2	1G08K8A0	SWITCHING POWER PCB	1		
3	ZA88JJPY	PAN HEAD SEMS SCREW 3P M3x8 CHROME	5		
4	ZA88JJUY	PAN HEAD SEMS SCREW 3P M3x16 CHROME	3		
5	1E00UKA0	UPSET HEAD BOLT 2P M4X6 CHROME	2		
6	CADQ35A0	CODE BUSH PANEL (70)	1		
7	CEGN46A0	CODE BUSH	1		
8	CAML90B0	CODE HOOK TERMINAL BLOCK	1		
9	ZB552LAY	EXTERNAL TOOTH WASHER M4 CHROME	1		
10	ZA88JLRL	PAN HEAD SEMS SCREW 3P M4x10 BS	1		
11	1G0803E0	LIGHT CONTROL PCB	1		
12	ZA88JLRY	PAN HEAD SEMS SCREW 3P M4x10 CHROME	4		
13	1A07PLA0	SWIVEL ARM (CN-80)	1		
14	1A07PHA0	TRANSFORMER BOX COVER (CN-80) *COATED	1		
15	1A0XNBA0	POWER CORD	1		
16	1A07W1B0	ROTATION SHAFT	1		
17	CAND32A0	WASHER 43X29X0.8	1		
18	CENX94A0	C-RING	1		
19	1A0ZV0A0	SENSOR/MANUAL MODE SWITCH ASSY WITH WIRE	1		PREVIOUS CODE: 1A0STYB0
20	1G0A33A0	NOISE FILTER	1		TILL MARCH 2024 PRODUCTION
	1G0ASLA0	NOISE FILTER			FROM APRIL 2024 PRODUCTION
21	ZA812LAY	HEX NUT M4X0.7	1		
22	ZA901JAY	FLANGE NUT M3 CHROME	2		
23	1A07W2A0	ROTATION STOPPER BOLT M8	1		
24	ZB511PAY	SPRING WASHER M8 CHROME	1		
25	ZA887JTY	PAN HEAD SEMS SCREW 2P M3x14 CHROME	1		
26	-	DISCONTINUED. FOR REPLACEMENT, PLEASE ORDER NO.29.	1		TILL MARCH 2024 PRODUCTION
27	CEEN94A0	FUSE 250V 0.8A (T)	2		
28	AEFR93B0	FUSE HOLDER 250V-10A	2		
29	1P080FA0	ROCKER SWITCH WITH PLATE AND HARNESS (D242/RAL9003)	1		TILL MARCH 2024 PRODUCTION
30	1A0XGTA0	BRACKET FOR CONTROL PCB	1		
31	CERN39A0	DOUBLE LOCKING SPACER	4		
32	1E09JTA0	ROCKER SWITCH	1		FROM APRIL 2024 PRODUCTION
33	1G09CWA0	BACK PLATE ASSY FOR ROCKER SWITCH SPEC. (D242) (CN80)	1		FROM APRIL 2024 PRODUCTION

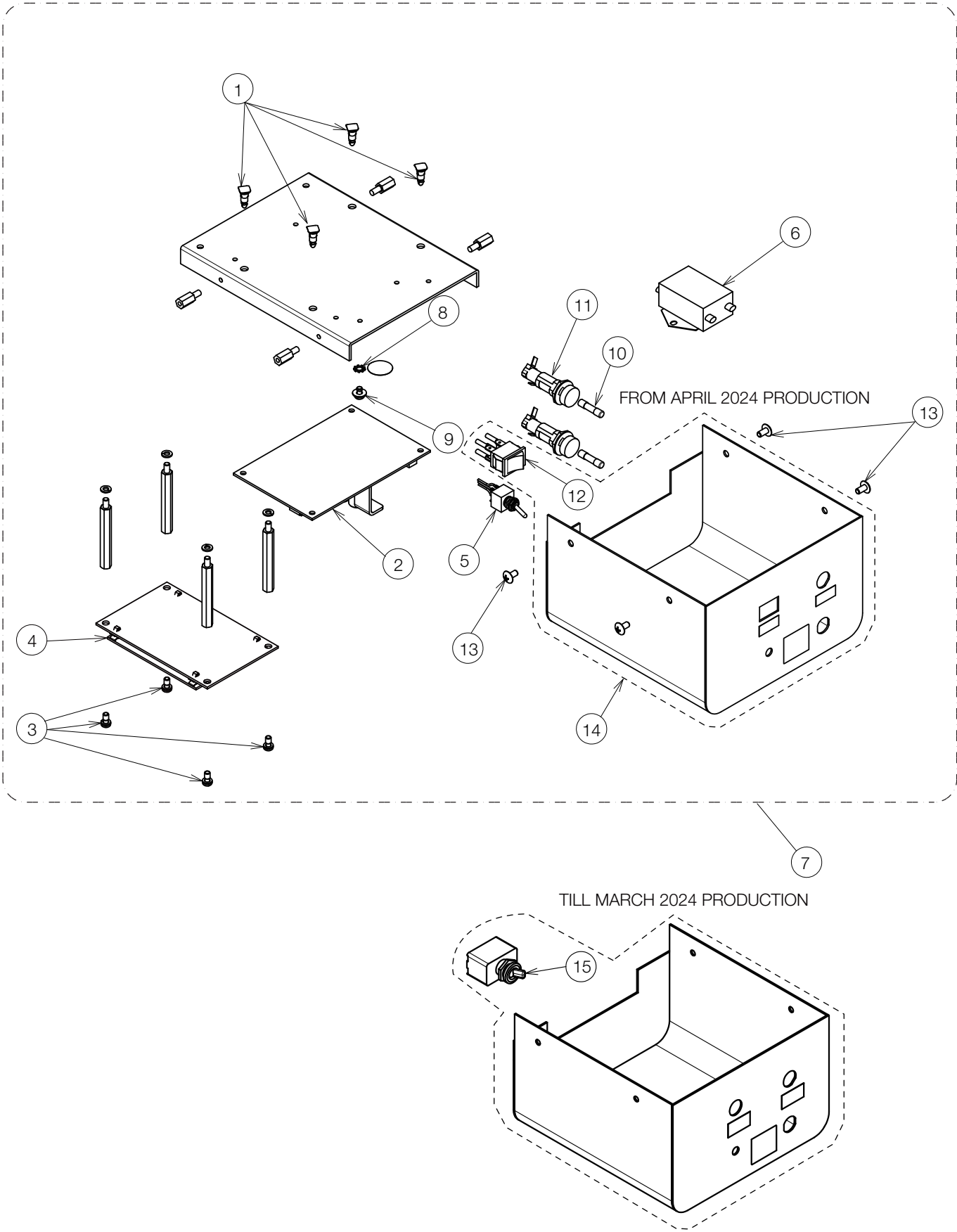




NO.	PARTS NO.	DESCRIPTION	QTY	APPLICABLE SERIAL NO.	NOTE
1	DESR05A0	SPACER	4		
2	1G08K8A0	SWITCHING POWER PCB	1		
3	ZA887LPY	PAN HEAD SEMS SCREW 2P M4x8 CHROME	8		
4	1G0803E0	LIGHT CONTROL PCB	1		
5	1A0ZV0A0	SENSOR/MANUAL MODE SWITCH ASSY WITH WIRE	1		PREVIOUS CODE: 1A0STYB0
6	1G0ASMA0	NOISE FILTER ASSY	1		
7	1G09CYA0	POWER SUPPLY BOX ASSY	1		FROM APRIL 2024 PRODUCTION
8	ZB552LAY	EXTERNAL TOOTH WASHER M4 CHROME	1		
9	ZA88JLNM	PAN HEAD SEMS SCREW 3P M4x6 BSNI	1		
10	CEEN94A0	FUSE 250V 0.8A (T)	2		
11	AEFR93B0	FUSE HOLDER 250V-10A	2		
12	1E09JTA0	ROCKER SWITCH	1		FROM APRIL 2024 PRODUCTION
13	1E01HHA0	TRUSS M4X8 RAL9003	4		
14	1P080GA0	ROCKER SWITCH WITH BOX AND HARNESS (D245)	1		TILL MARCH 2024 PRODUCTION
15	-	DISCONTINUED. FOR REPLACEMENT, PLEASE ORDER NO.14.	1		TILL MARCH 2024 PRODUCTION

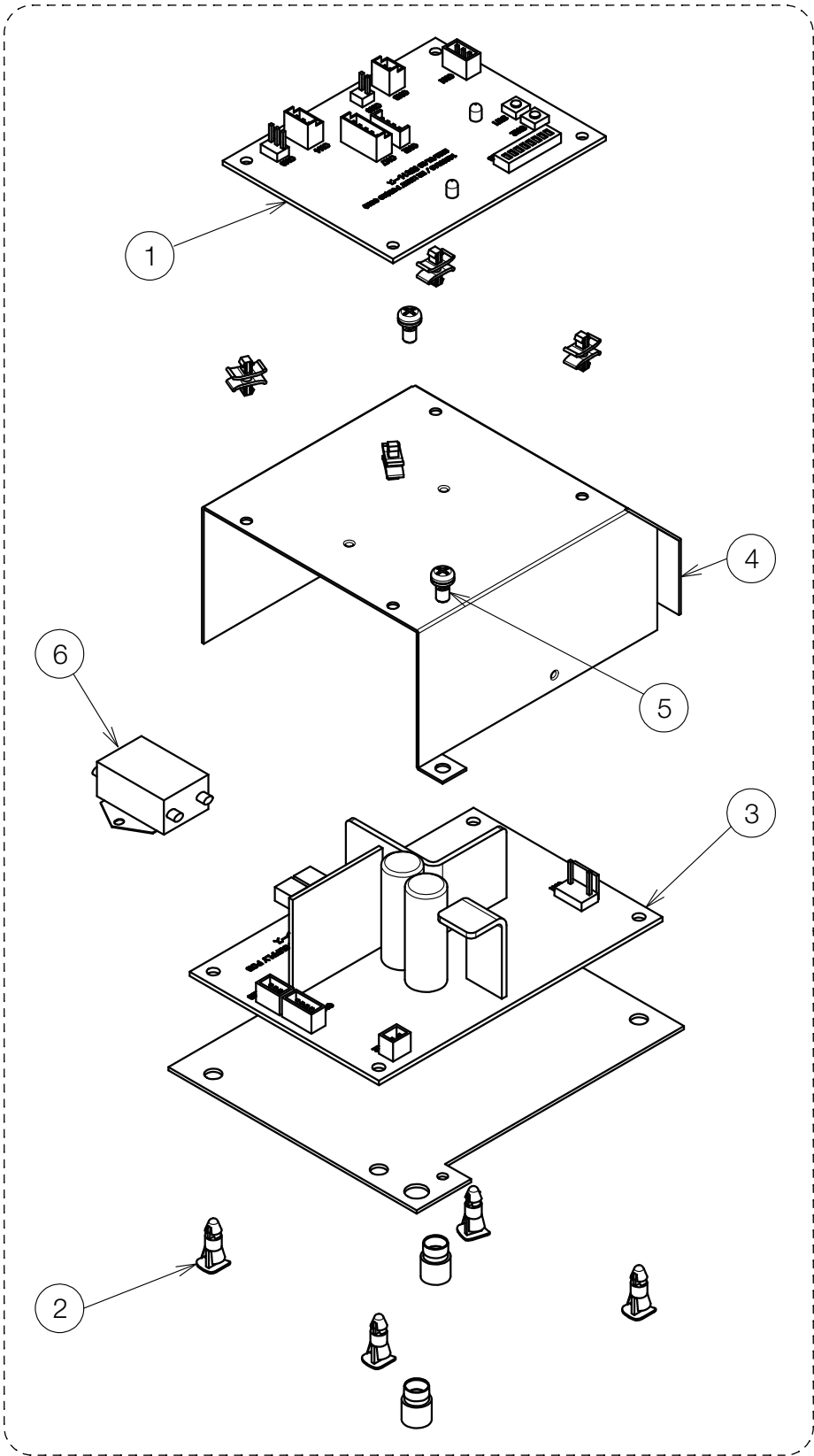


TILL DECEMBER 2024
PRODUCTION

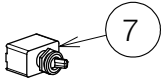


FROM JANUARY 2025
PRODUCTION

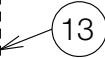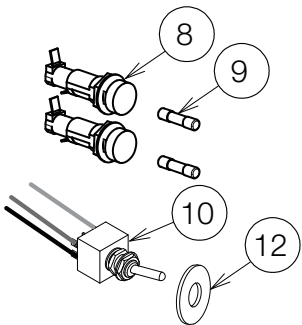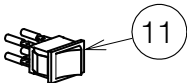


NO.	PARTS NO.	DESCRIPTION	QTY	APPLICABLE SERIAL NO.	NOTE
1	1G0803E0	LIGHT CONTROL PCB	1		
2	DESR05A0	SPACER	4		
3	1G08K8A0	SWITCHING POWER PCB	1		
4	1A07X1E0	COVER FOR POWER PCB	1		
5	ZA887LPY	PAN HEAD SEMS SCREW 2P M4X8 CHROME	2		
6	1G0ASNA0	NOISE FILTER ASSY WITH WIRE	1		PREVIOUS CODE: 1G0A3VA0
7	CENX56B0	MAIN SWITCH	1		TILL DECEMBER 2024 PRODUCTION
8	AEFR93B0	FUSE HOLDER	2		
9	CEEN94A0	FUSE 250V 0.8A (T)	2		
10	1A0SU4A0	SENSOR/MANUAL SWITCH ASSY	1		
11	1E09JTA0	ROCKER SWITCH	1		FROM JANUARY 2025 PRODUCTION
12	1A07PYC0	SPACER FOR SENSOR/MANUAL MODE SWITCH	1		
13	1G09RXA0	POWER PCB BOX (D247)	1		

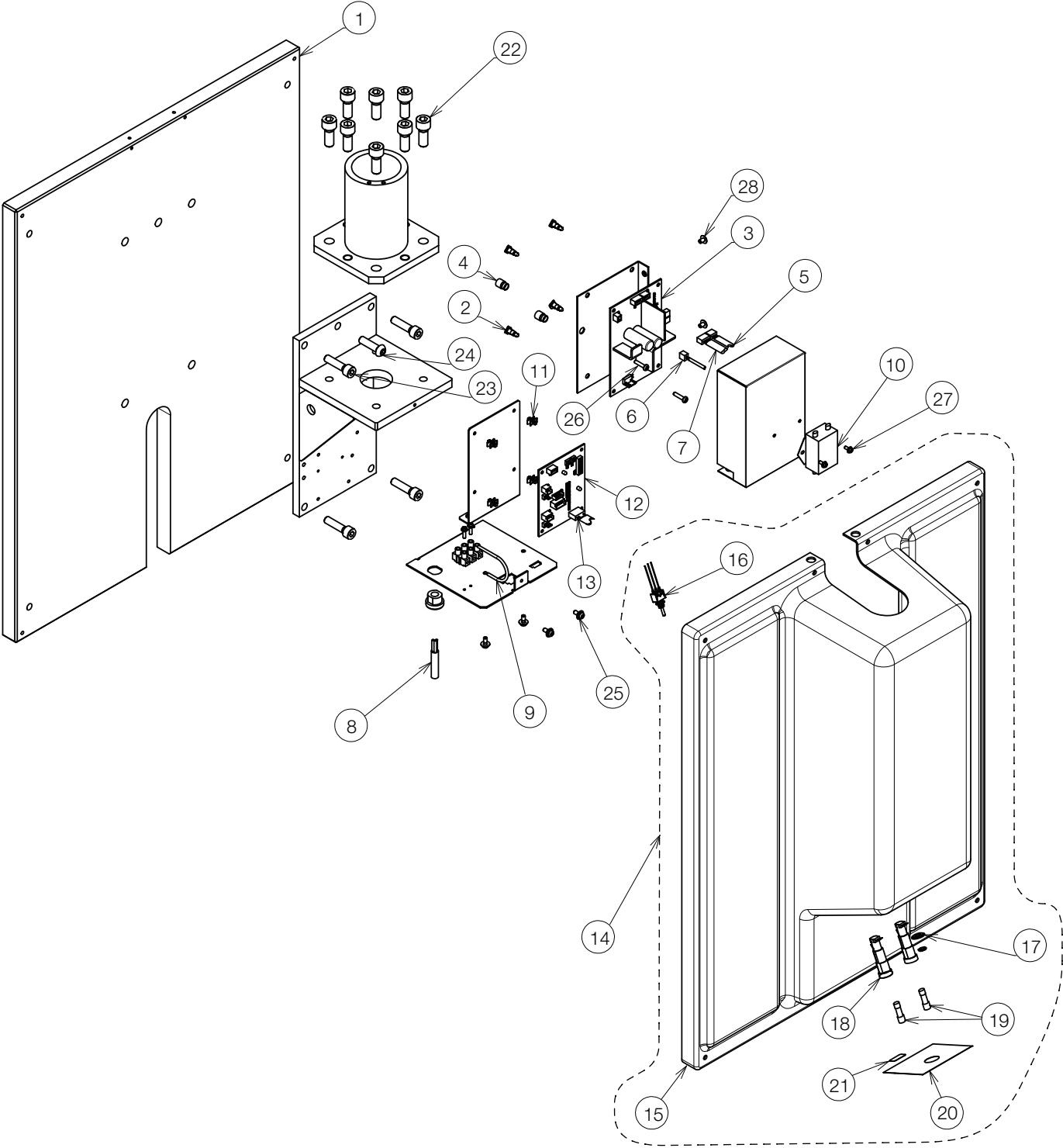


NO.	PARTS NO.	DESCRIPTION	QTY	APPLICABLE SERIAL NO.	NOTE
1	1A0QB3A0	WALL PLATE FOR EPA	1		
2	DESR05A0	SPACER	4		
3	1G08K8A0	SWITCHING POWER PCB	1		
4	1E02FBA0	SNAP COLOR BUSH	2		
5	1A0STXA0	TRUNK LINE FOR ADJUST PULSE	1		
6	1A07MPA0	TRUNK LINE A FOR CONTROL PCB	1		
7	1A07MRA0	TRUNK LINE FOR LED MODULE	1		
8	1A04CZA0	CT CODE 2000L	1		
9	1A04CXA0	EARTH WIRE 200L	1		
10	1G0A3DA0	NOISE FILTER ASSY	1		
11	CERN39A0	SPACER	4		
12	1G0803E0	LIGHT CONTROL PCB	1		
13	1A0SU8A0	TRUNK LINE A FOR SENSOR/MANUAL SWITCH	1		
14	1G0827A0	FRONT COVER SET (CN80)	1		
15	1A0ALYA0	FRONT COVER (CN80)	1		
16	1A0SU9A0	SENSOR/MANUAL MODE SWITCH ASSY WITH WIRE	1		
17	1A07PYC0	SPACER FOR SENSOR/MANUAL SWITCH	1		
18	AEFR93B0	FUSE HOLDER 250V-10A	2		
19	CEEN94A0	FUSE 250V 0.8A (T)	2		
20	CATP11C0	LABEL FOR SENSOR/MANUAL MODE SWITCH	1		
21	CAEP84B0	FUSE LABEL 0.8 A (T)	1		
22	ZA761QYU	HEX SOCKET HEAD SCREW M10x25 SUS	8		
23	CETN56A0	HEX SOCKET HEAD SCREW M8x32 BLACK CHROME	4		
24	ZSA11PZG	HEX BUTTON HEAD SCREW M8x30 BLACK	1		
25	ZA88JLRY	PAN HEAD SEMS SCREW 3P M4x10 CHROME	4		
26	ZA111LUC	PAN HEAD SCREW M4x16 CHROME	2		
27	ZA88JJPY	PAN HEAD SEMS SCREW 3P M3x8 CHROME	2		
28	ZA114LNY	TRUSS M4x6 CHROME	2		

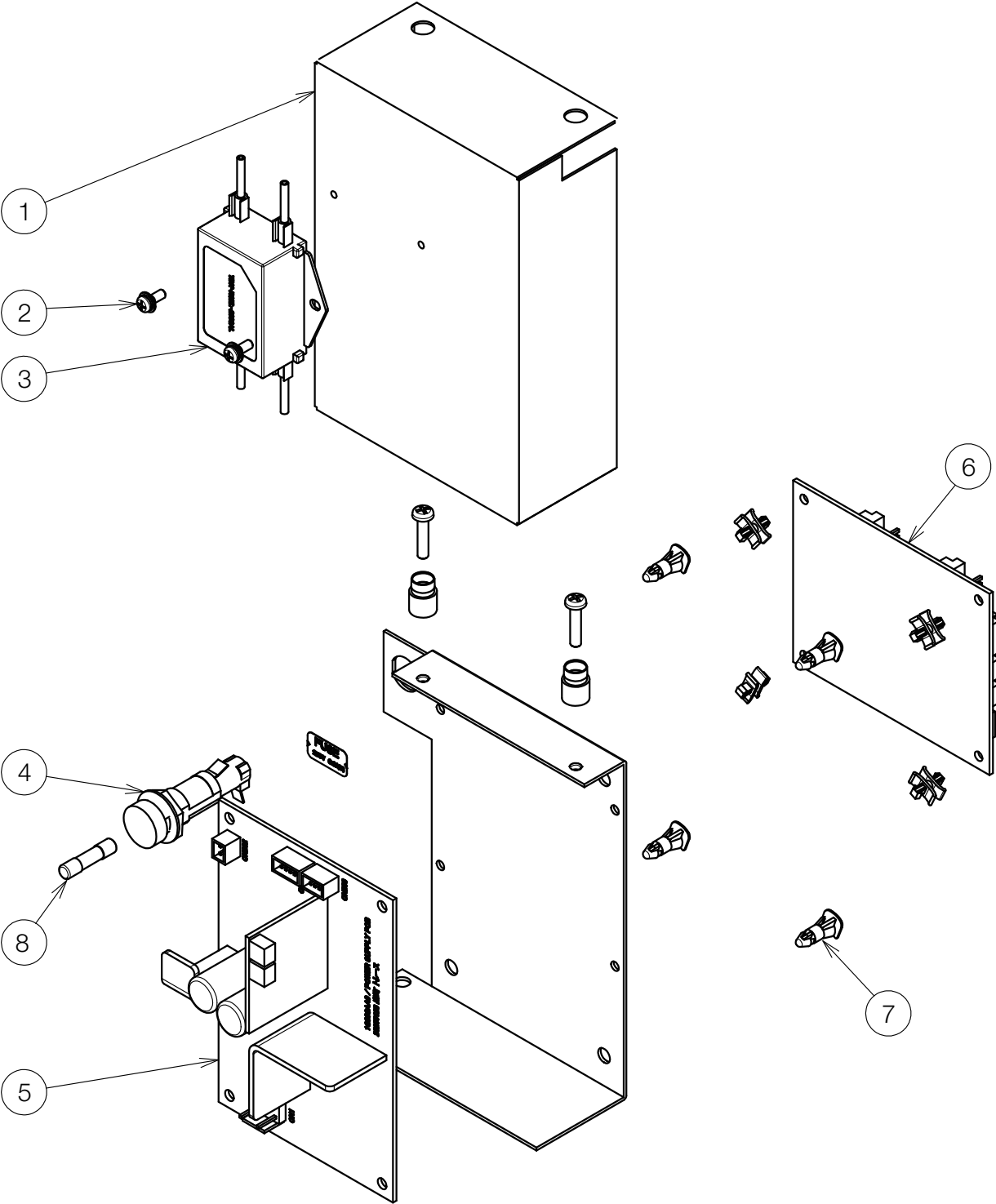


NO.	PARTS NO.	DESCRIPTION	QTY	APPLICABLE SERIAL NO.	NOTE
1	1A07Z9B0	POWER PCB BOX	1		
2	ZA88JJPY	PAN HEAD SEMS SCREW 3P M3x8 CHROME	2		
3	1A08AGC0	NOISE FILTER ASSY	1		
4	AEFR93B0	FUSE HOLDER 250V-10A	1		
5	1G08K8A0	SWITCHING POWER PCB	1		
6	1G07PBE0	LIGHT CONTROL PCB FOR QUOLIS	1		
7	DESR05A0	SPACER	4		
8	CEEN94A0	FUSE 250V 0.8A (T)	1		

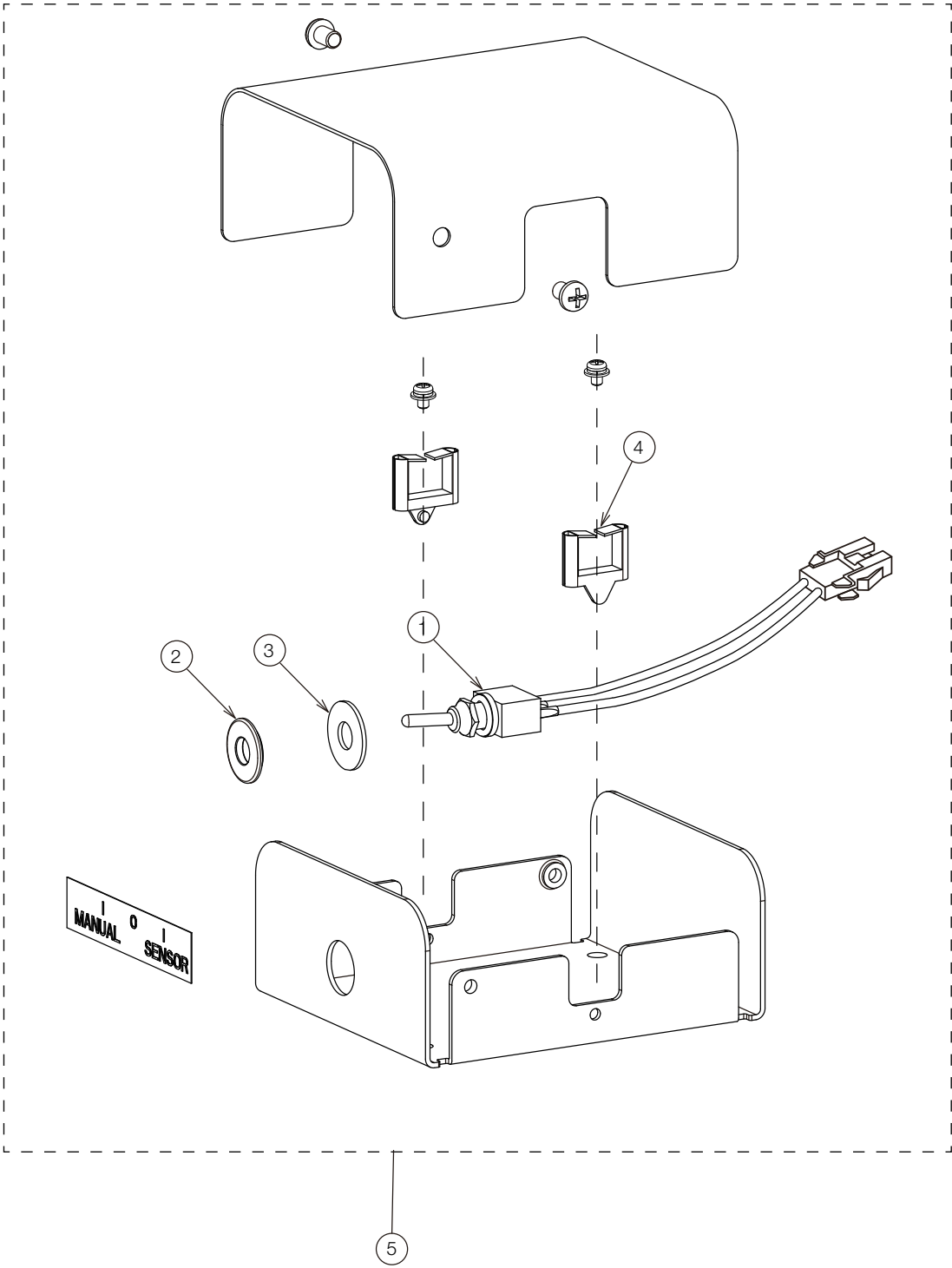


NO.	PARTS NO.	DESCRIPTION	QTY	APPLICABLE SERIAL NO.	NOTE
1	1A086ZA0	SENSOR/MANUAL MODE SWITCH ASSY WITH WIRE	1		
2	1A07PYC0	SPACER FOR SENSOR/MANUAL SWITCH	1		
3	1E00V3A0	WASHER M6x16x1.0 CHROME	1		
4	CEGN75A0	EDGE SADDLE	2		
5	1P081KA0	SWITCH BOX ASSY FOR D246G-QU			

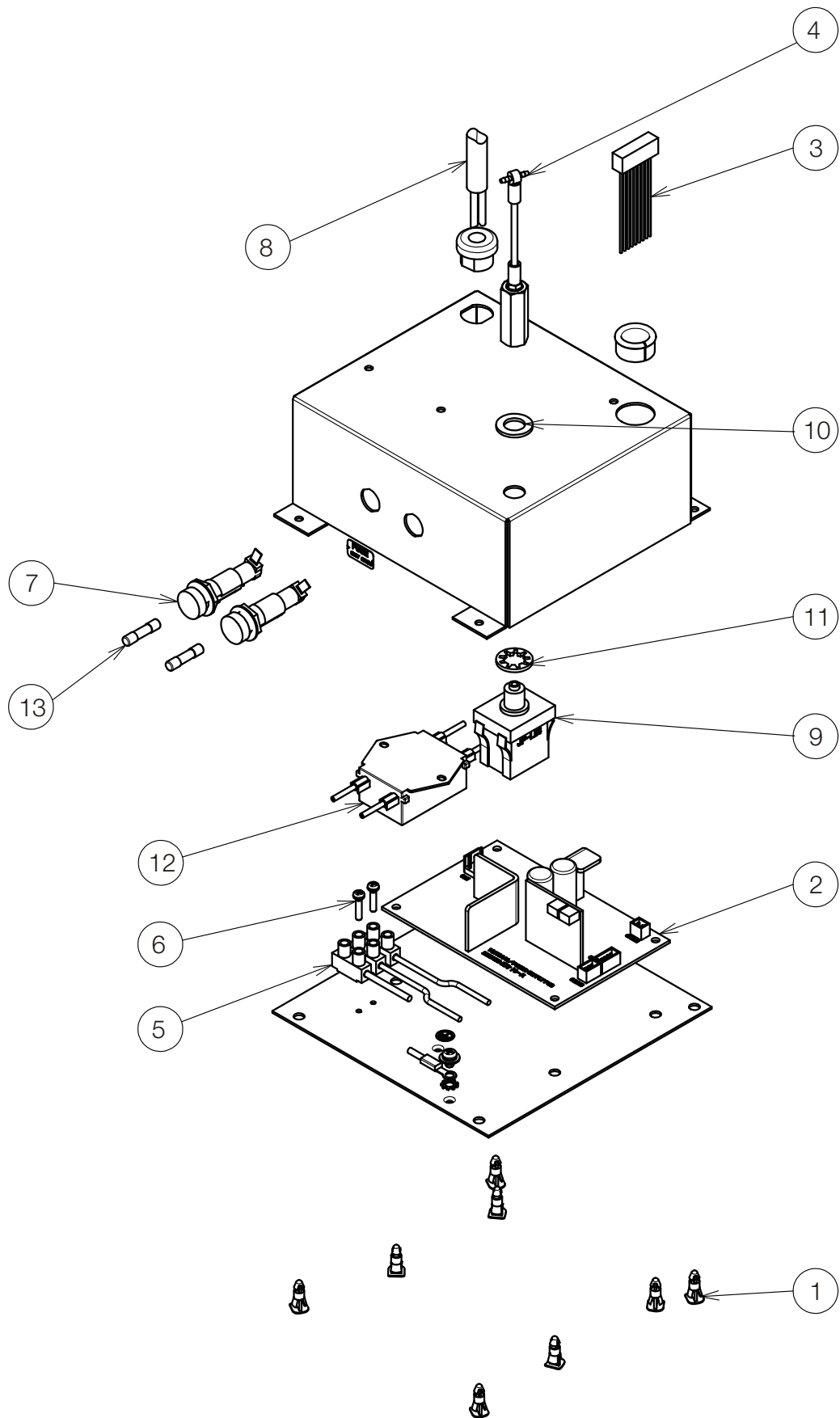


NO.	PARTS NO.	DESCRIPTION	QTY	APPLICABLE SERIAL NO.	NOTE
1	DESR05A0	SPACER	8		
2	1G08K8A0	SWITCHING POWER PCB	1		
3	1A0SUFA0	TRUNK LINE FOR POWER PCB	1		
4	1G02MBA0	AIR SWITCH CYLINDER ASSY WITH TUBING & FITTING	1		
5	1G0A3WA0	TERMINAL BLOCK ASSY	1		
6	ZA887JTY	PAN HEAD SEMS SCREW 2P M4x8 CHROME	2		
7	AEFR93B0	FUSE HOLDER 250V-10A	2		
8	1A10AXA0	CT CODE 1000L	1		PREVIOUS CODE: 1A04D6A0
9	1A0H4TB0	AIR SWITCH	1		
10	1E00A7A0	WASHER M10X22X2.4 CHROME	1		
11	ZB551QAY	INTERNAL TOOTH WASHER M10 CHROME	1		
12	1G0A3UA0	NOISE FILTER ASSY	1		
13	CEEN94A0	FUSE 250V 0.8A (T)	2		

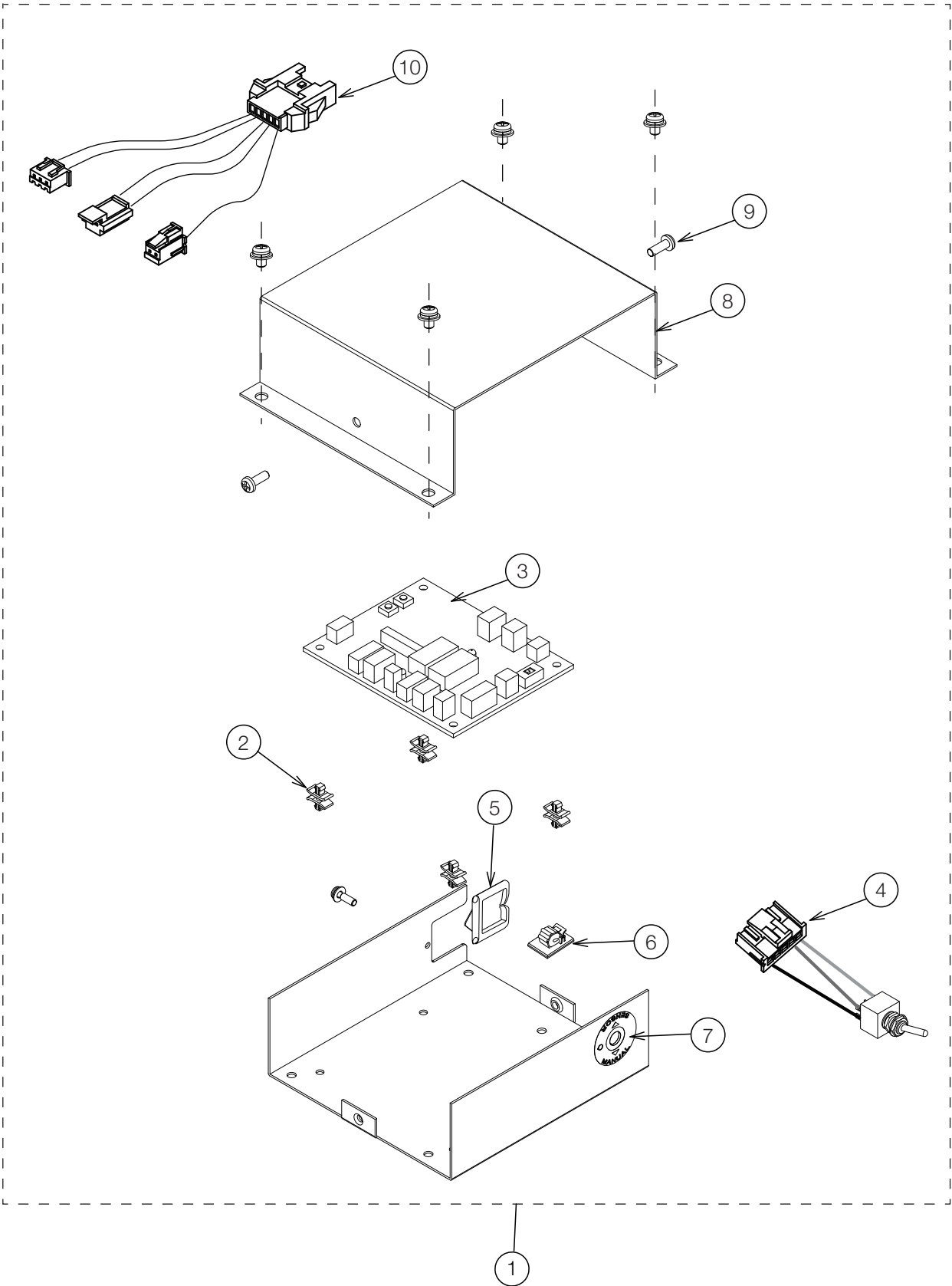


NO.	PARTS NO.	DESCRIPTION	QTY	APPLICABLE SERIAL NO.	NOTE
1	1P081MA0	SWITCH BOX ASSY (RAL9003) FOR D249W-X	1		
	1P081PA0	SWITCH BOX ASSY (CN80) FOR D246G-EV			
2	CERN39A0	DOUBLE LOCKING SPACER	4		
3	1G0803E0	LIGHT CONTROL PCB	1		
4	1A0SUJA0	SENSOR/MANUAL MODE SWITCH ASSY WITH WIRE	1		
5	CETN14A0	EDGE SADDLE FOR CABLE PROTECTION	1		
6	CETN13A0	CLAMP WITH SPONGE	1		
7	1A0HMYA0	LABEL FOR S/M MODE SWITCH	1		
8	CATN46CR	SWITCH BOX COVER (RAL9003)	1		
	1A0HWWA0	SWITCH BOX COVER (CN80)			
9	1E00VTA0	PAN HEAD SEMS SCREW M4x10 RAL9003	2		
	1A033FA0	PAN HEAD SCREW M4x10 CN80			
10	1A0SUHA0	TRUNK LINE FOR ADJUST PULSE	1		

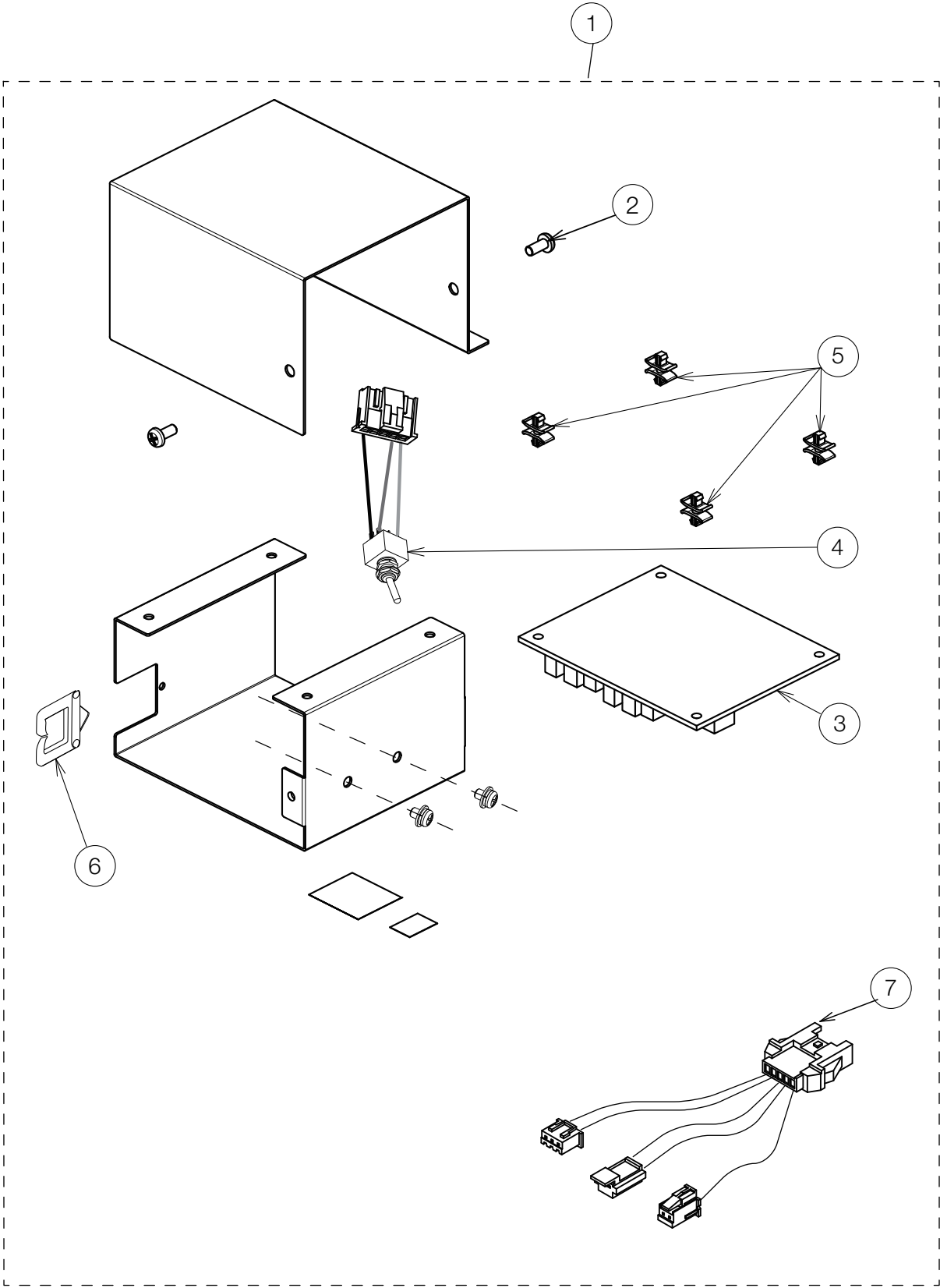


NO.	PARTS NO.	DESCRIPTION	QTY	APPLICABLE SERIAL NO.	NOTE
1	1P081RA0	SWITCH BOX ASSY FOR D249W-XN	1		
2	ZA887LPU	PAN HEAD SEMS SCREW 2P M4x8 SUS	2		
3	1G0803E0	LIGHT CONTROL PCB	1		
4	1A0SUJA0	SENSOR/MANUAL MODE SWITCH ASSY WITH WIRE	1		
5	CERN39A0	DOUBLE LOCKING SPACER	4		
6	CETN14A0	EDGE SADDLE FOR CABLE PROTECTION	1		
7	1A0SUHA0	TRUNK LINE FOR ADJUST PULSE	1		

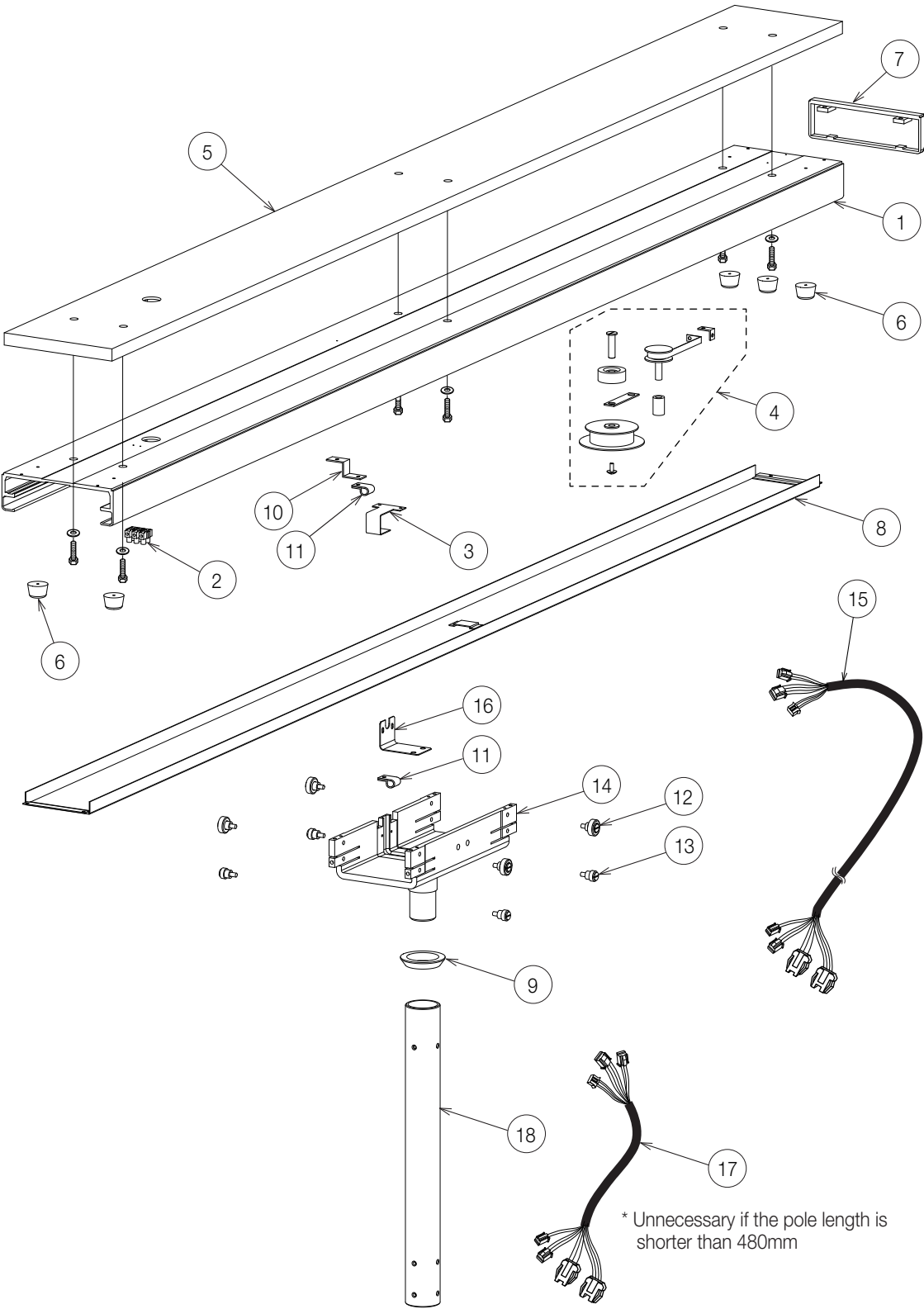



NO.	PARTS NO.	DESCRIPTION	QTY	APPLICABLE SERIAL NO.	NOTE
1	1A0824A0	RAIL (HAMMER NET COATED, CN-80)	1		
2	CAML90B0	CODE HOOK TERMINAL BLOCK	1		
3	CAMG90A0	CENTRE COVER HOOK	1		
4	1G0133A0	PULLEY ASSY WITH STICKER	1		
5	1A0QSDA0	PALLET	1		
6	ABNL23A0	STOPPER	5		
7	1G02EEA0	END COVER ASSY (CN-80)	1		
8	1P03CYA0	CENTER COVER ASSY (CN-80)	1		
9	CAMG95A0	TROLLEY COLLAR (COATED)	1		
10	CAMV63A0	LEAD WIRE GUIDE	1		
11	CENZ15A0	NYLON CLAMP	2		
12	1P08PDA0	ROLLER UPPER ASSY	4		
13	1P08PEA0	ROLLER LOWER ASSY	4		
14	1A0823D0	TROLLEY (HAMMER NET COATED, CN-80)	1		
15	1A087RA0	YOKE RELAY CODE A	1		
16	CAMP73B0	LEAD WIRE COVER	1		
17	1A087TA0	YOKE RELAY CODE C	1		
18	1G02U2A0	T POLE SET 380L (CN-80)	1		
	1G02U3A0	T POLE SET 530L (CN-80)			
	1G02U4A0	T POLE SET 680L (CN-80)			
	1G02U5A0	T POLE SET 840L (CN-80)			
	1G02U6A0	T POLE SET 1000L (CN-80)			
-	1G009XA0	PALLET SCREW SET	1		
-	CGRN39B0	TRACK SCREW SET	1		

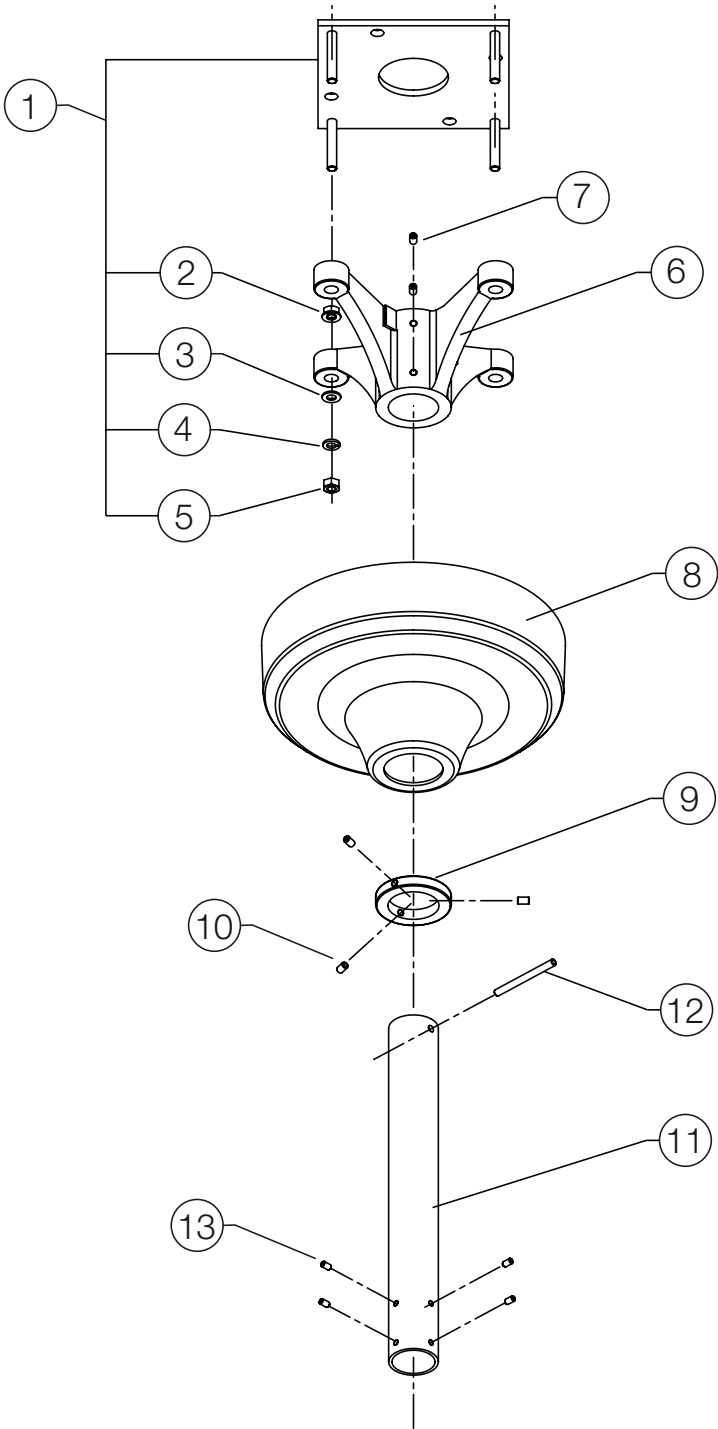



NO.	PARTS NO.	DESCRIPTION	QTY	APPLICABLE SERIAL NO.	NOTE
1	1P0211A0	MOUNTING PLATE WITH NUT (NO.2-5)	1		
2	ZA901PAY	FLANGE NUT M8 CHROME (LEVELING NUT)	4		
3	CENE00B0	WASHER M8X22X1.63 CHROME	4		
4	ZB511PAY	SPRING WASHER M8 CHROME	4		
5	1E00ABA0	U NUT M8 CHROME	4		
6	TCPA01A0	CEILING FLANGE	1		
7	ZA774OPU	HEX SOCKET SET SCREW M6X8 (CUP POINT) SUS	4		
8	1A0827A0	CEILING COVER (CN-80)	1		
9	CAMG76B0	STOPPER RING	1		
10	CENB35B0	HEX SOCKET SET SCREW M6X8 (CUP POINT) (RED MEC THREAD LOCK)	3		
11	1G09C4A0	CEILING POLE SET 380MM (CN-80) 8FT	1		
	1G09C6A0	CEILING POLE SET 530MM (CN-80) 8.5FT			
	1G02TNA0	CEILING POLE SET 680MM (CN-80) 9FT			
	1G02TPA0	CEILING POLE SET 840MM (CN-80) 9.5FT			
	1G02TQA0	CEILING POLE SET 1000MM (CN-80) 10FT			
12	CAMK40A0	ROLL PIN 6X70	1		
13	ZA772NRY	HEX SOCKET SET SCREW M5X10 (TRUNCATED CONE POINT) CHROME	4		
-	CGRN38B0	CEILING SCREW SET	1		
-	CGQO13A0	CEILING FLANGE SET (ID NO, 1-7 AND 12)	1		

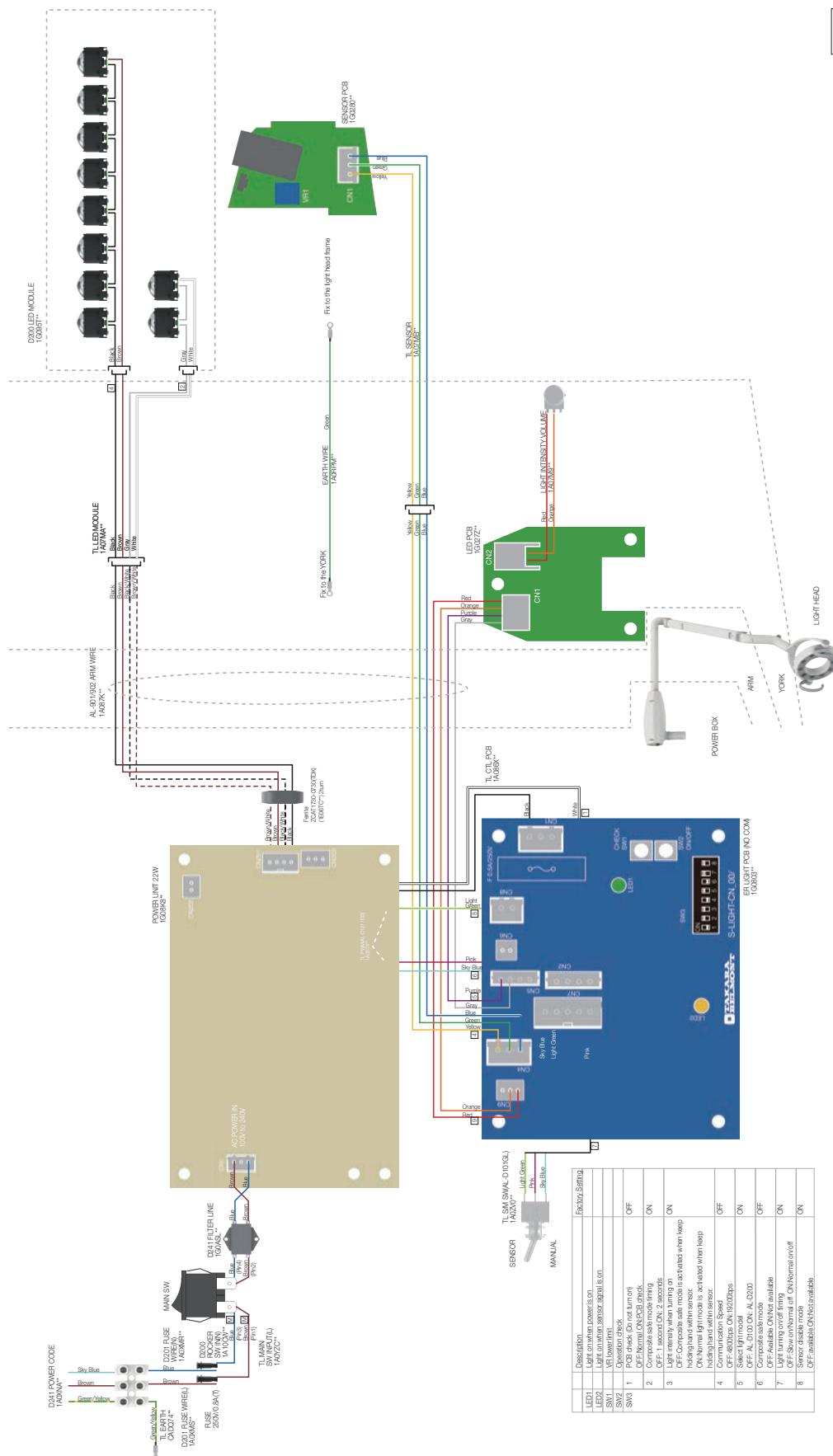


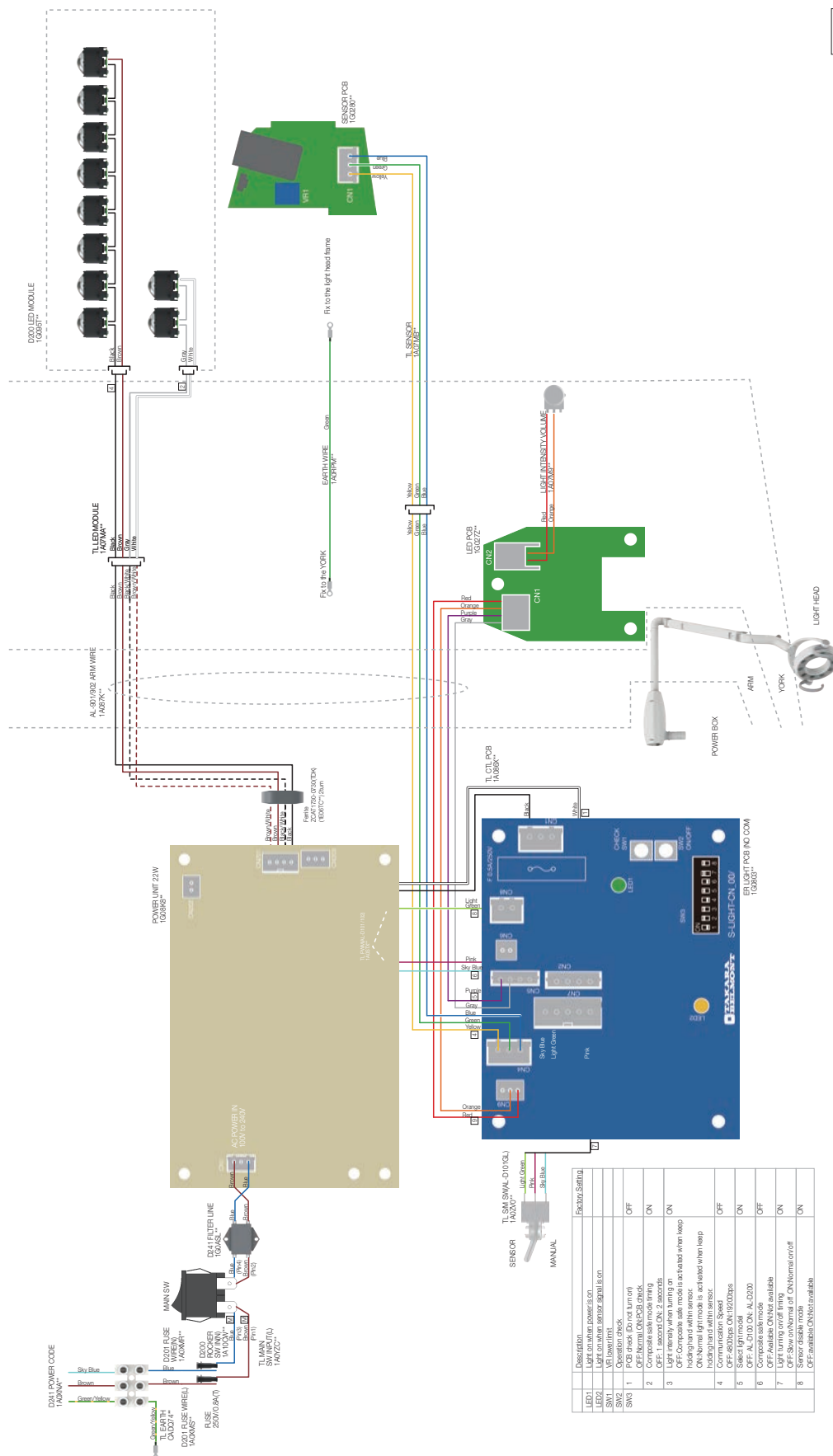

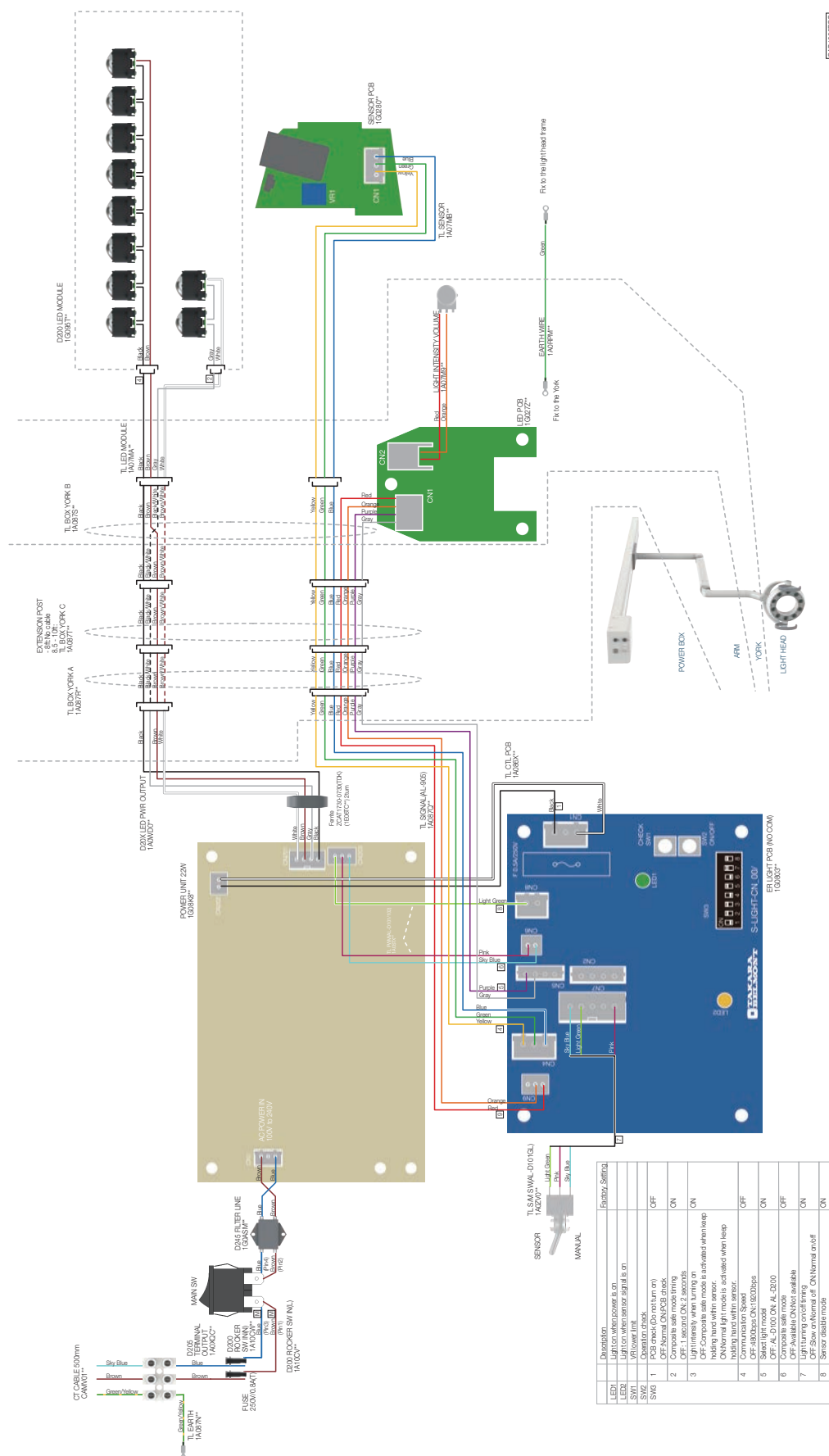

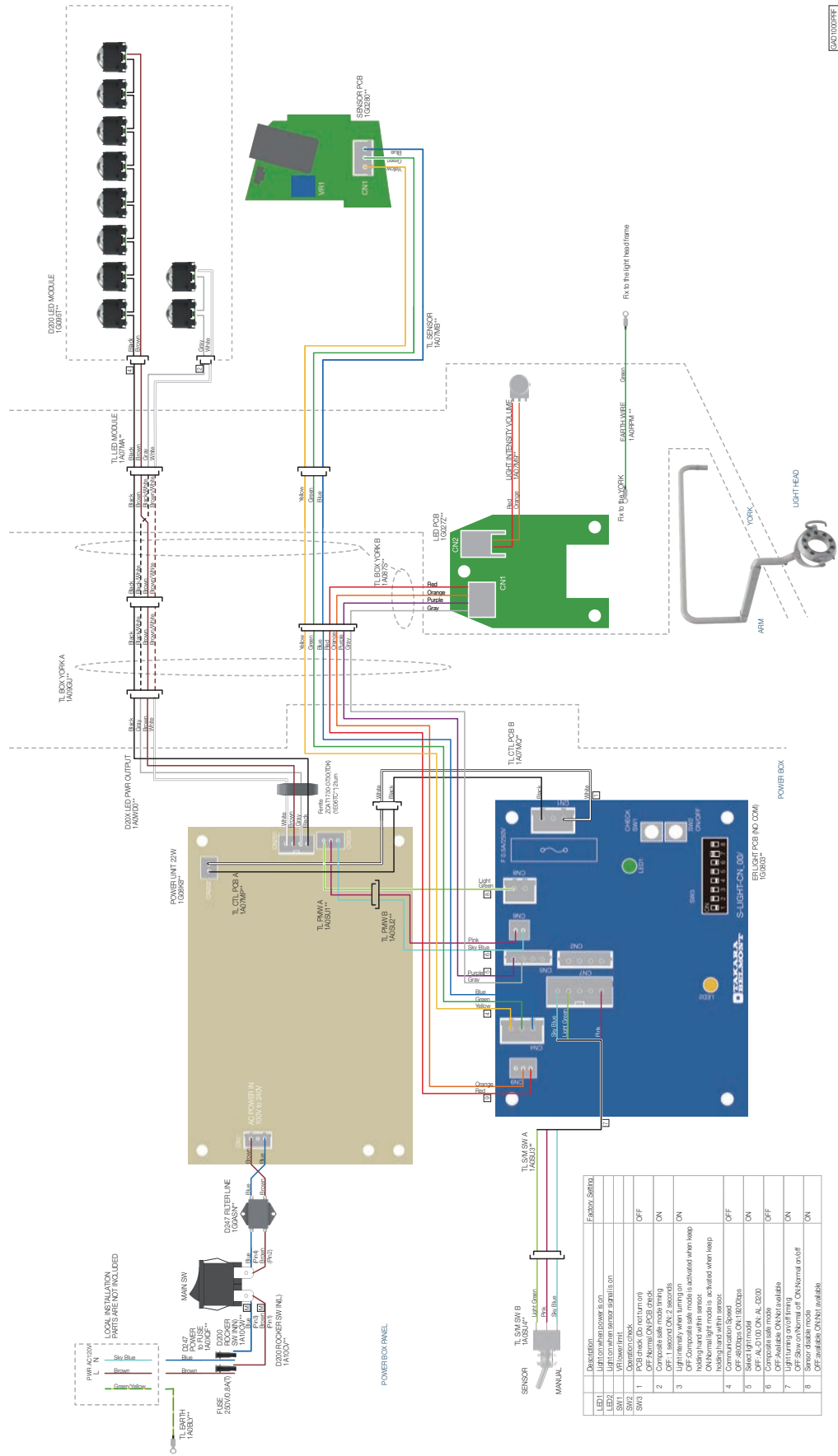


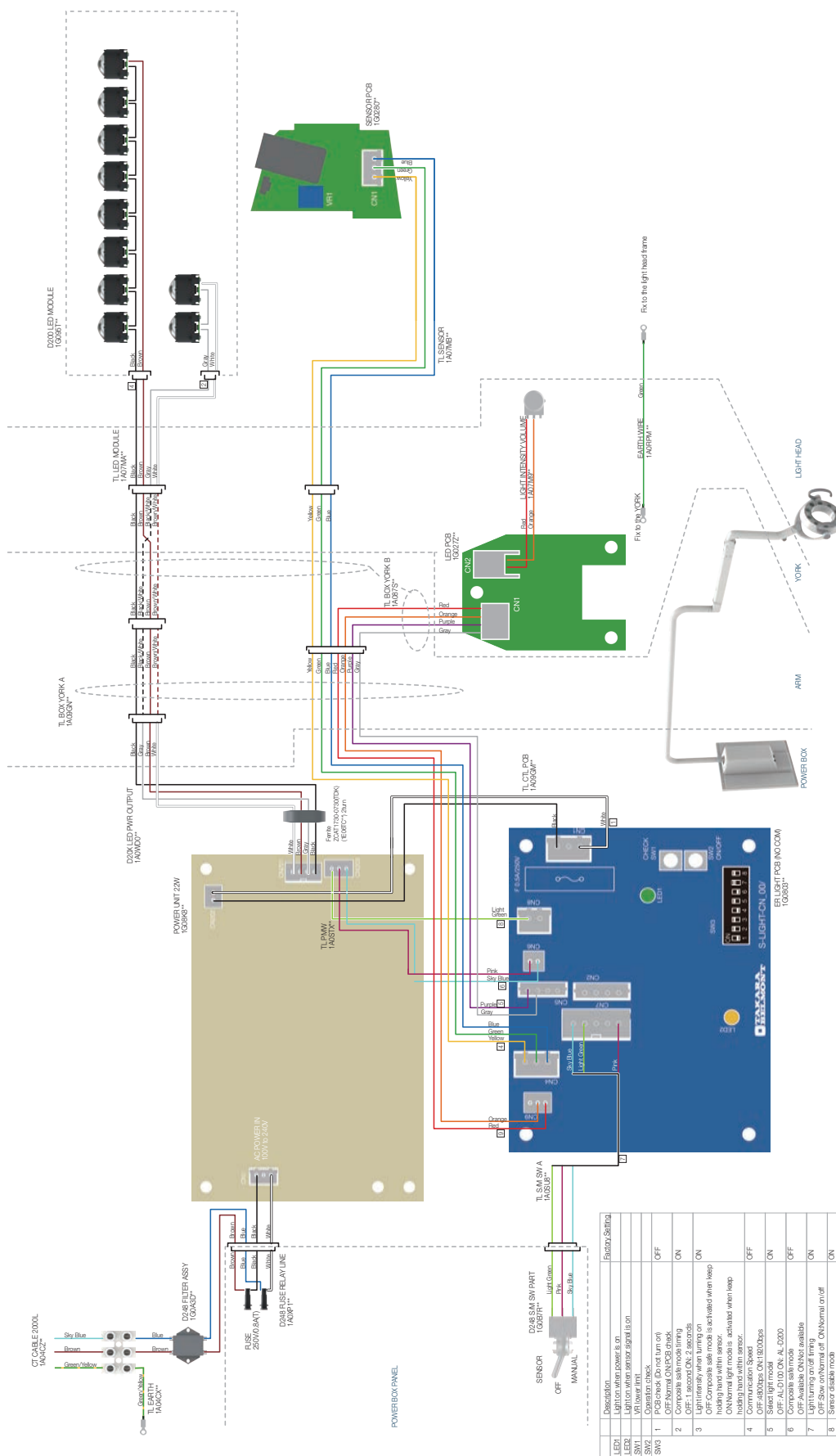


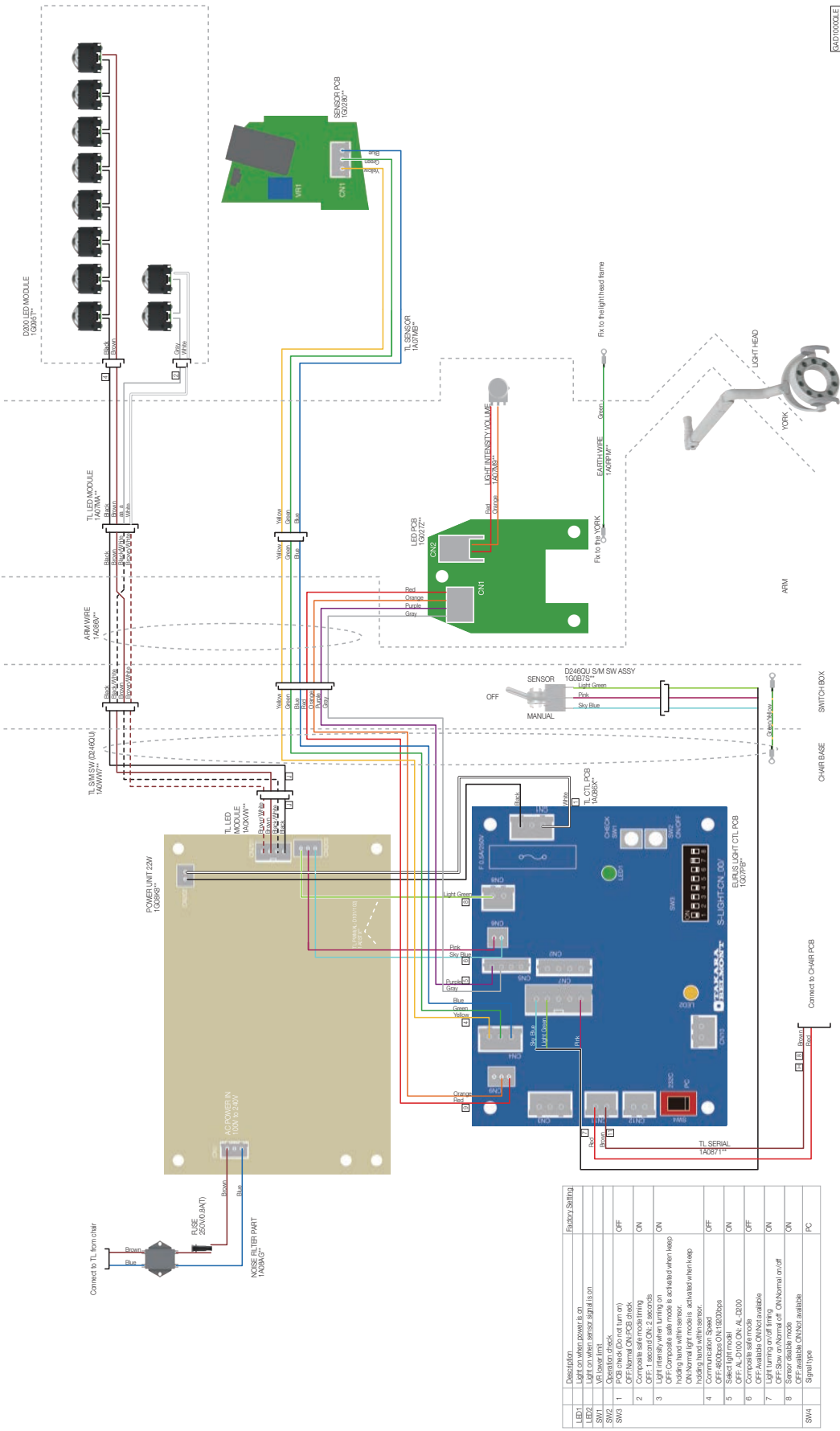

(EVG OTP & Swing Unit on Q3300/Q5500, EVG Swing Unit on B50N, X-Calibur OTP & Swing Unit on B50N/Q3300/Q5500)

520100002

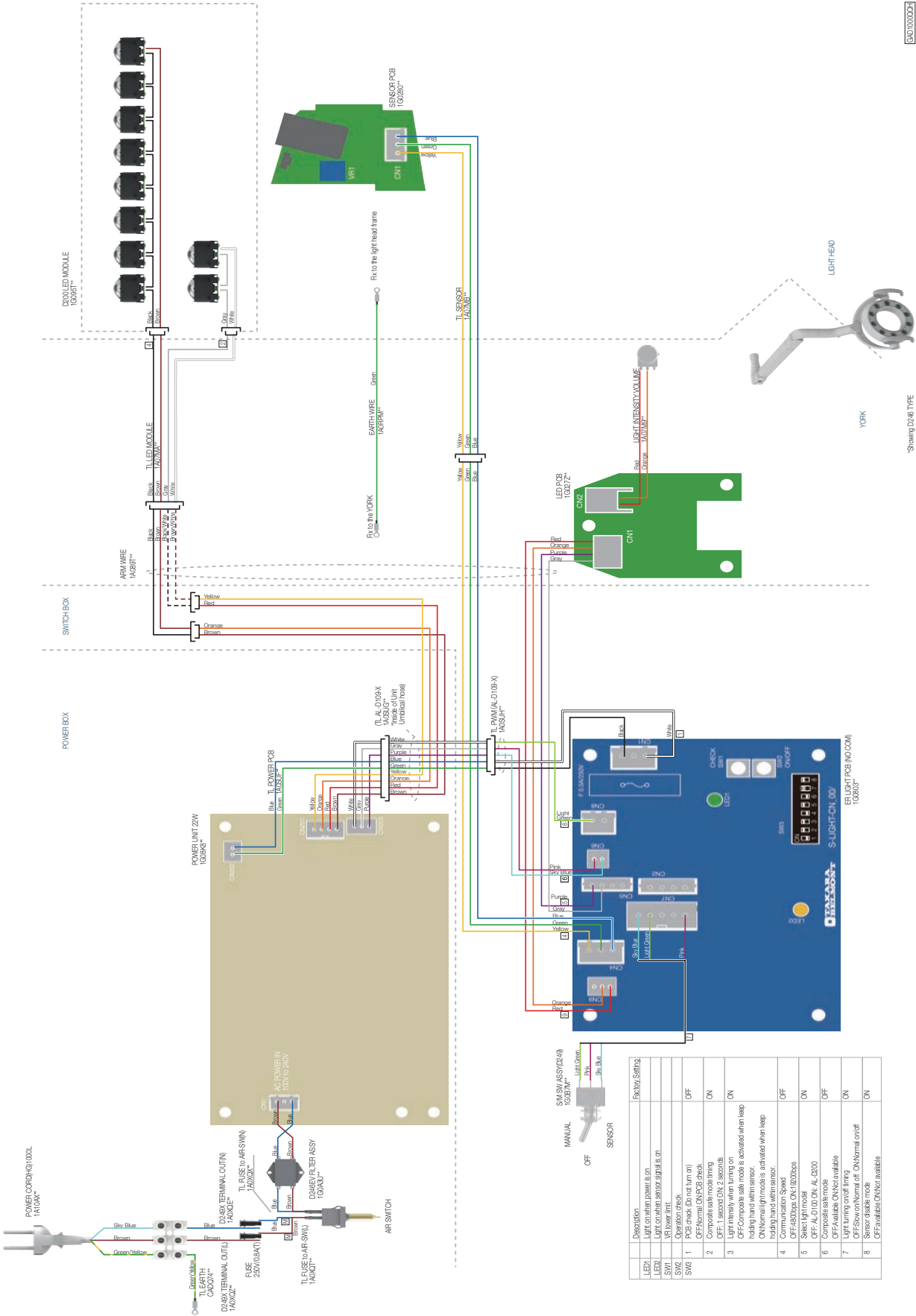

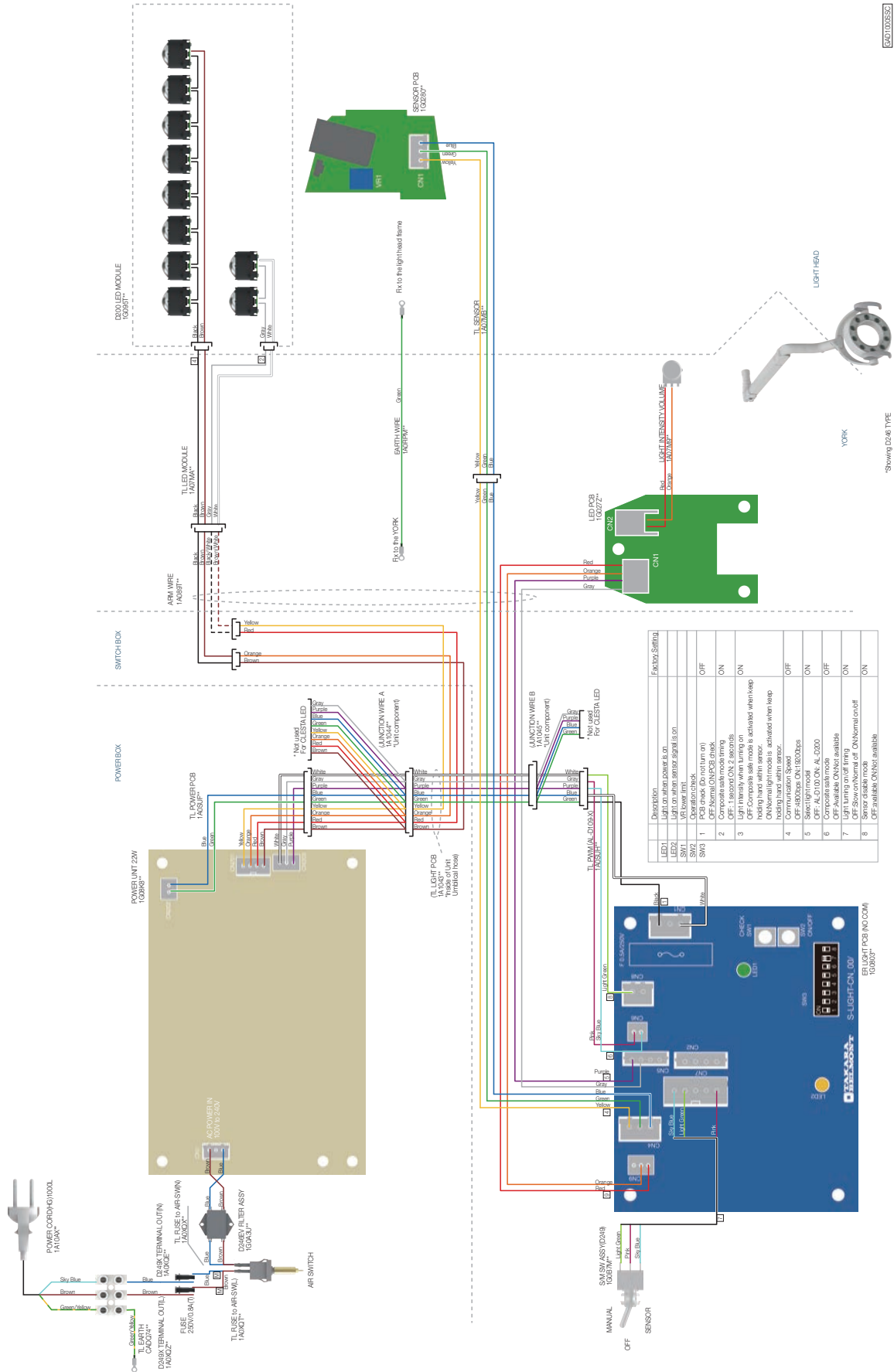

54100352

*Showing D246 TYPE

REVISION RECORD

DATE		CONTENT
22-Sep-22	1st	Publish
24-Feb-23	2nd	Modify the control PCB for D241, D242, and D245
22-Apr-24	3rd	Regularly update and modify table of contetns and main switch for D241/D242/D245.
20-Feb-25	4th	Regularly update



Importer in U.S.A.
BELMONT EQUIPMENT,
Division of Takara Belmont USA, Inc.

101 Belmont Drive, Somerset, New Jersey 08873 U.S.A.

TEL : (732) 469-5000 / (800) 223-1192

FAX : (732) 356-1035

Importer in Canada
TAKARA COMPANY, CANADA, LTD.

2455 Meadowvale Blvd. Mississauga, Ontario L5N 5S2, Canada

TEL : (905) 816-8965

www.belmont.ca

TAKARA BELMONT CORPORATION

2-1-1, Higashishinsaibashi, Chuo-ku, Osaka, 542-0083, Japan

TEL : +81-6-6213-5945

FAX : +81-6-6212-3680

