

Bel-Nova DENTAL LIGHT PARTS LIST


TABLE OF CONTENTS

	PAGE
1 . LIGHT SECTION -----	4
2 . LIGHT HEAD -----	6
3 . YORK / SENSOR ASSY-----	8
4 . POWER SUPPLY BOX (D101) -----	10
5 . POWER SUPPLY BOX (D102) -----	12
6 . POWER SUPPLY BOX (D105) -----	14
7 . POWER SUPPLY BOX (D106G-QU) -----	16
8 . SWITCH BOX (D106G-QU) -----	18
9 . POWER SUPPLY BOX (D106G-EV, D109) -----	20
10 . SWITCH BOX (D106G-EV, D109W-X) -----	22
11 . SWITCH BOX (D109W-XN) -----	24
12 . POWER SUPPLY BOX (D107) -----	26
13 . POWER SUPPLY BOX (D108) -----	28
14 . CEILING POLE (D102) -----	30
15 . TRACK (D105) -----	32
16 . SWIVEL ARM (D107) -----	34
17 . SWIVEL ARM (D108) -----	36

NOTE:"LEAD TIME" means below


A < B < C
Short ⇔ Long

LIGHT SECTION

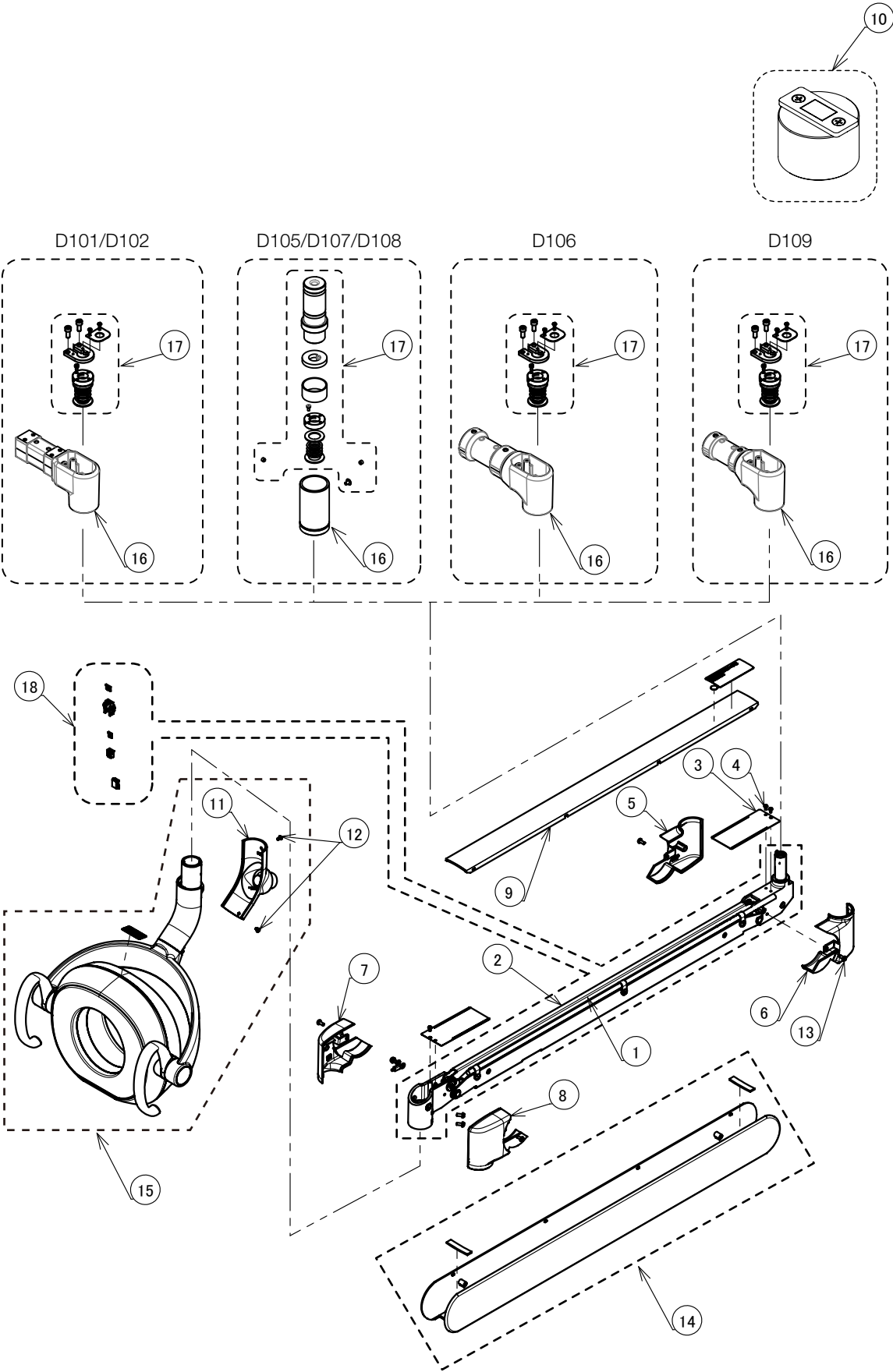


NO.	ID	PARTS NO.	DESCRIPTION	QTY	APPLICABLE SERIAL NO.	LEAD TIME	NOTE
1	G12 - 1A0STWA0	CABLE FOR LIGHT IN BALANCE ARM (D101/ D102)	1		C		
	G12 - 1A0SU0A0	CABLE FOR LIGHT IN BALANCE ARM (D105 / D107 / D108)			C		
	G12 - 1A0SUKA0	CABLE FOR LIGHT IN BALANCE ARM (D106G-EV / D109)			C		
	G12 - 1A0SUQA0	CABLE FOR LIGHT IN BALANCE ARM (D106G-QU)			C		
2	G12 - 1G0717A0	BALANCE ARM ASSY WITHOUT HARNESS	1		C		
3	G12 - 1A0PLCB0	PROTECTION COVER FOR BALANCE ARM	2		B		
4	G12 - ZA111JLY	PAN HEAD SCREW M3x5 CHROME	5		A		
5	G12 - 1A0PLAA0	BRACKET COVER (L) FOR SWING ARM (RAL9003)	1		C		
	G12 - 1A0STBA0	BRACKET COVER (L) FOR SWING ARM (CN80)			C		
6	G12 - 1A0PLBA0	BRACKET COVER (R) FOR SWING ARM (RAL9003)	1		C		
	G12 - 1A0STCA0	BRACKET COVER (R) FOR SWING ARM (CN80)			C		
7	G12 - 1A0PLDA0	BRACKET COVER (L) FOR LIGHT (RAL9003)	1		C		
	G12 - 1A0STDA0	BRACKET COVER (L) FOR LIGHT (CN80)			C		
8	G12 - 1A0PLEA0	BRACKET COVER (R) FOR LIGHT (RAL9003)	1		C		
	G12 - 1A0STEA0	BRACKET COVER (R) FOR LIGHT (CN80)			C		
9	G12 - 1A0PL9A0	UPPER COVER FOR BALANCE ARM (RAL9003)	1		C		
	G12 - 1A0STGA0	UPPER COVER FOR BALANCE ARM (CN80)			C		
10	G12 - 1P05HBA0	JIG FOR BALANCE ARM	1		B		
11	G12 - 1G03JAA0	REAR COVER FOR YORK (RAL9003)	1		C		
	G12 - 1G059EA0	REAR COVER FOR YORK (CN80)			C		
12	G12 - CENB09AR	FLAT HEAD SCREW M3x6 (RAL9003)	2		A		
	G12 - CEGN77A0	FLAT HEAD SCREW M3x6 (CN80)			B		
13	G12 - ZA111LPY	PAN HEAD SCREW M4x8 CHROME	6		A		
14	G12 - 1P05L9A0	LOWER COVER ASSY FOR BALANCE ARM (RAL9003)	1		C		
	G12 - 1P063XA0	LOWER COVER ASSY FOR BALANCE ARM (CN80)			C		
15	G12 - 1P05LAA0	LIGHT HEAD ASSY (RAL9003)	1		C		
	G12 - 1P063YA0	LIGHT HEAD ASSY (CN80)			C		
16	G12 - 1P063ZA0	SWIVEL ARM KNUCKLE (D101 RAL9003)	1		C		
	G12 - 1P0640A0	SWIVEL ARM KNUCKLE (D101 / D102 CN80)			C		
	G12 - 1G07WXA0	SWIVEL ARM KNUCKLE (D106 CN80)			C		
	G12 - 1G080BA0	POLE JOINT (D105 / D107 / D108 CN80)			C		
	G12 - 1G07WWA0	SWIVEL ARM KNUCKLE (D109 RAL9003)			C		
17	G12 - 1P05LCA0	JOINT KIT (D101 / D102 / D106 / D109)	1		C		
	G12 - 1P0641A0	JOINT KIT (D105 / D107 / D108)			C		
18	G12 - 1P0642A0	BALANCE ARM ASSY WITH HARNESS (D101/ D102)	1		C		
	G12 - 1P0643A0	BALANCE ARM ASSY WITH HARNESS (D105 / D107 / D108)			C		
	G12 - 1P0644A0	BALANCE ARM ASSY WITH HARNESS (D106G-EV / D109)			C		
	G12 - 1P0645A0	BALANCE ARM ASSY WITH HARNESS (D106G-QU)			C		

LIGHT HEAD

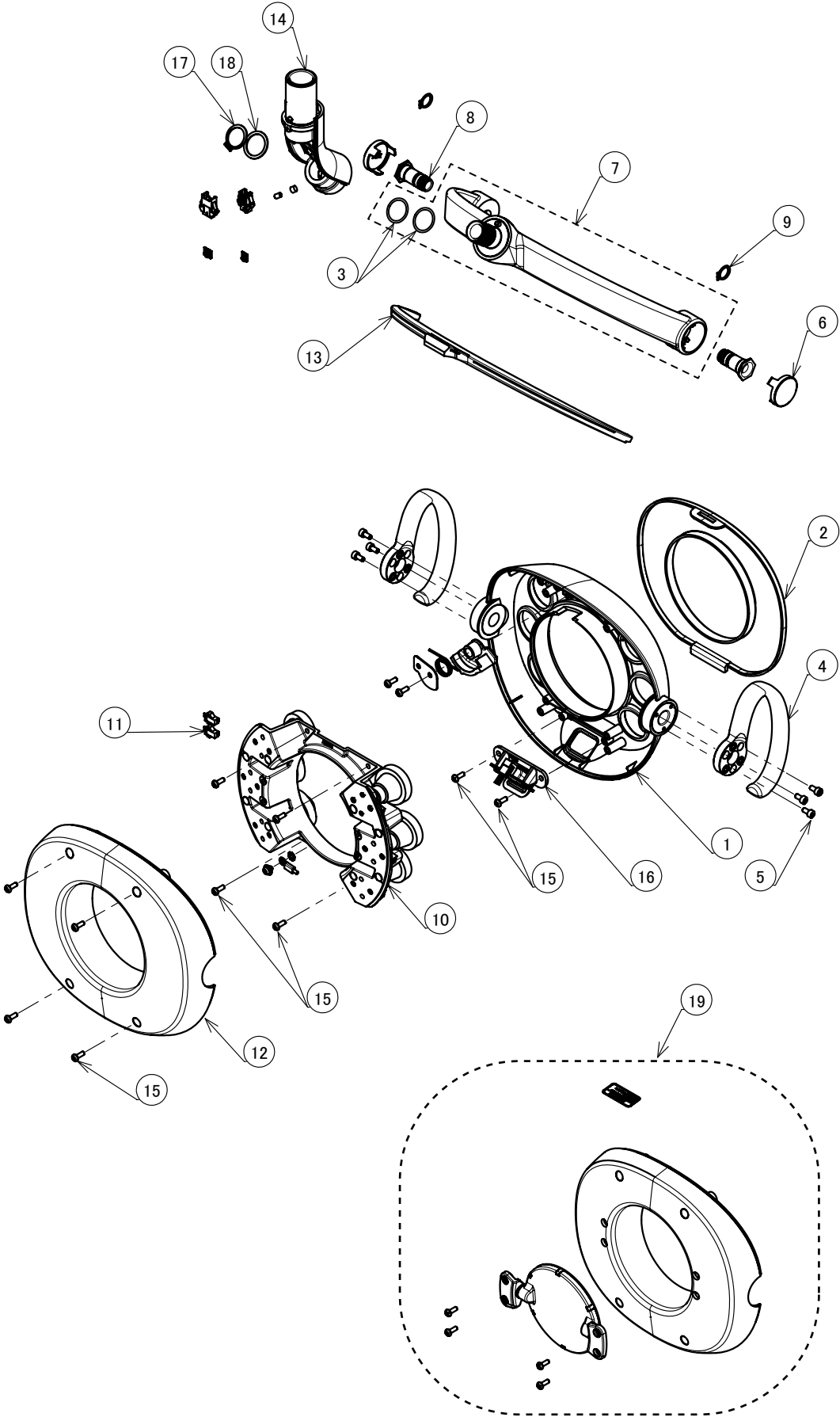


NO.	ID	PARTS NO.	DESCRIPTION	QTY	APPLICABLE SERIAL NO.	LEAD TIME	NOTE
1	G12 - 1A0PKPB0	FRONT COVER FOR HEAD (RAL9003)		1		C	
	G12 - 1A0ST8B0	FRONT COVER FOR HEAD (CN80)				C	
2	G12 - 1A0PKRA0	COVER FOR LENS		1		C	
3	G12 - FJMA83A0	O-RING		2		A	
4	G12 - 1A07L6B0	HANDLE FOR LIGHT		2		C	
5	G12 - ZA761LPU	HEX SOCKET HEAD SCREW M4x8 SUS		6		A	
6	G12 - 1A0B9FA0	CAP FOR YORK		2		C	
7	G12 - 1G071AA0	YORK (RAL9003)		1		C	
	G12 - 1G07X2A0	YORK (CN80)				C	
8	G12 - 1G03SBA0	SHAFT FOR HEAD ROTATE		2		C	
9	G12 - ZG041RAG	C-RING M12 BLACK		2		A	
10	G12 - 1G06Y9B0	LED MODULE ASSY		1		C	
11	G12 - 1A07MAB0	WIRE FOR LED MODULE		1		B	
12	G12 - 1A0PKLA0	REAR COVER FOR HEAD (RAL9003)		1		C	
	G12 - 1A0STAA0	REAR COVER FOR HEAD (CN80)				C	
13	G12 - 1A0PKJA0	COVER FOR YORK		1		B	
14	G12 - 1G0725A0	YORK FOR BASE (RAL9003)		1		C	
	G12 - 1G07X1A0	YORK FOR BASE (CN80)				C	
15	G12 - AFFX58A0	PAN HEAD SCREW TYPE-P M4x10 CHROME		12		A	
16	G12 - 1G071BA0	SENSOR ASSY		1		C	
17	G12 - ZG041VAG	C-RING		1		B	
18	G12 - 1A0B9RA0	WASHER FOR YORK		1		C	
19	G12 - 1P05U9A0	PATIENT MIRROR SET (RAL9003)		1		C	
	G12 - 1P05XXA0	PATIENT MIRROR SET (CN80)				C	

14 YORK

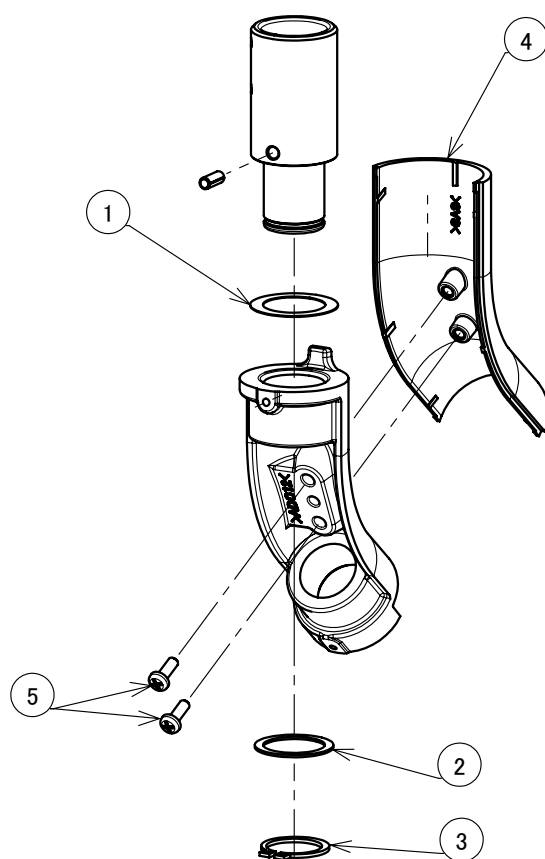


16 SENSOR ASSY

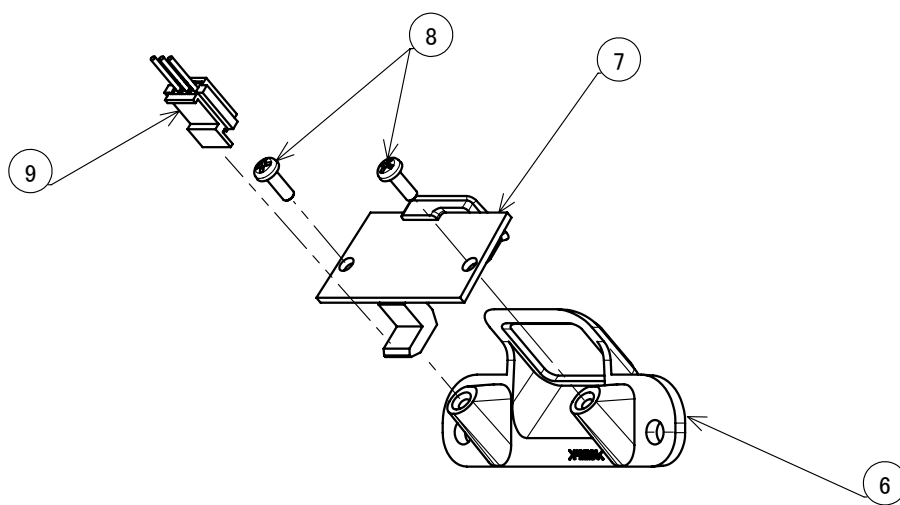


NO.	ID	PARTS NO.	DESCRIPTION	QTY	APPLICABLE SERIAL NO.	LEAD TIME	NOTE
1	G12 - 1E00UUA0		WAVE WASHER	1		B	
2	G12 - 1A0B9RA0		WASHER FOR YORK	1		C	
3	G12 - ZG041VAG		C-RING	1		B	
4	G12 - 1A0B9BA0		FRONT COVER FOR YORK (RAL9003)	1		C	
	G12 - 1A0HVKA0		FRONT COVER FOR YORK (CN80)			C	
5	G12 - AFFX58A0		PAN HEAD SCREW TYPE-P M4x10 CHROME	2		A	
6	G12 - 1A0PKNA0		COVER FOR SENSOR	1		C	
7	G12 - 1G03GLA0		SENSOR PCB	1		B	
8	G12 - CECN46A0		PAN HEAD TAPPING SCREW M3x8 FOR PLASTIC	2		A	
9	G12 - 1A0P3SB0		WIRE FOR SENSOR	1		B	

POWER SUPPLY BOX (D101)

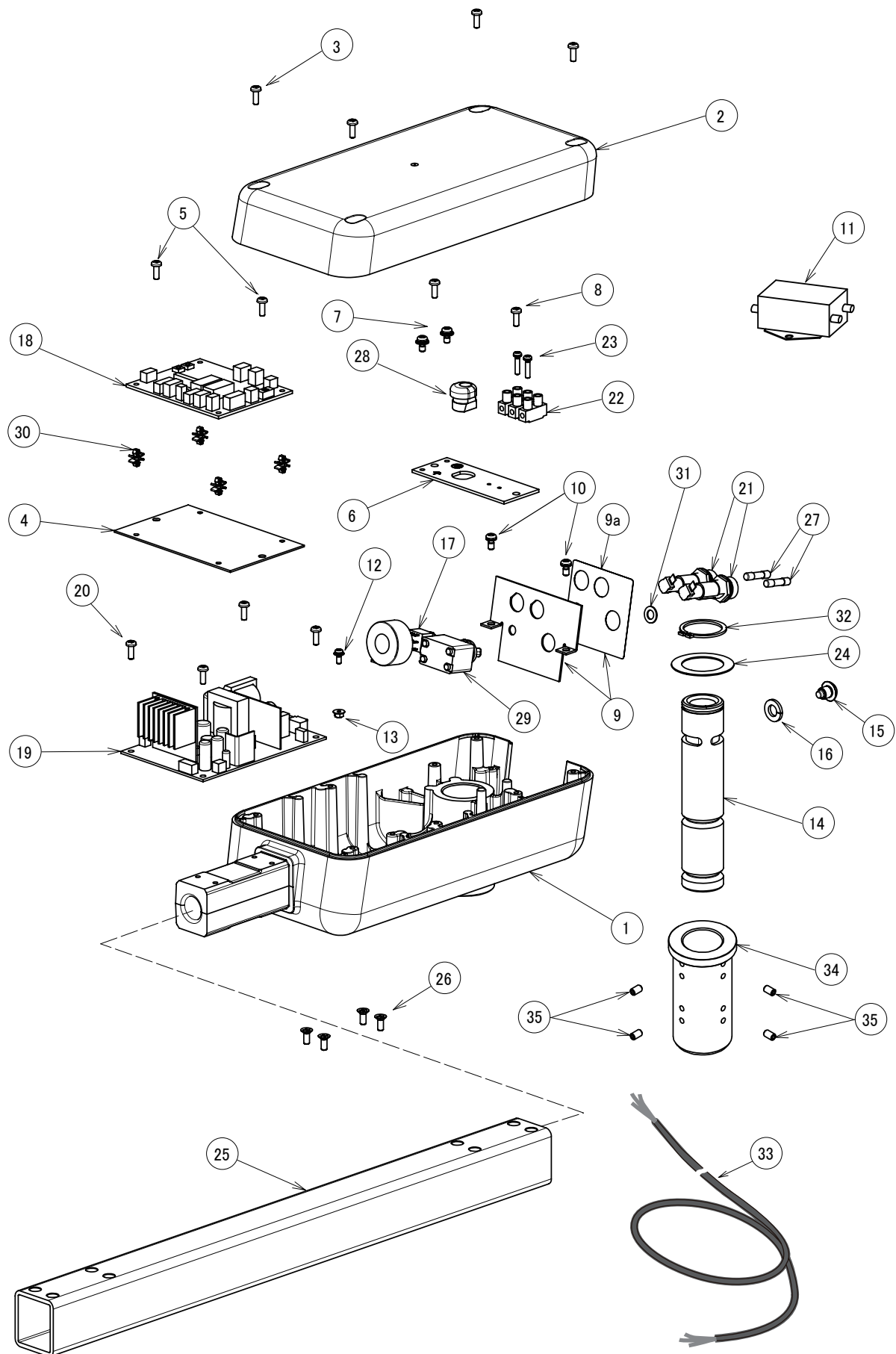


NO.	ID	PARTS NO.	DESCRIPTION	QTY	APPLICABLE SERIAL NO.	LEAD TIME	NOTE
1	G12 - 1A0BFMA0	HOUSING (RAL9003)	1		B		
	G12 - 1A0HVNA0	HOUSING (CN80)			B		
2	G12 - 1A0BFNA0	HOUSING COVER (RAL9003)	1		C		
	G12 - 1A0HVJA0	HOUSING COVER (CN80)			C		
3	G12 - ZA111LPY	PAN HEAD SCREW M4x8 CHROME	4		A		
4	G12 - 1A0BFPA0	PLATE FOR CONTROL PCB	1		C		
5	G12 - ZA111LPY	PAN HEAD SCREW M4x8 CHROME	2		A		
6	G12 - 1A0BFQB0	ELECTRIC PARTS FIXING PLATE	1		C		
7	G12 - ZA88JLRL	PAN HEAD SEMS SCREW 3P M4x10 BRASS	2		A		
8	G12 - ZA111LPY	PAN HEAD SCREW M4x8 CHROME	2		A		
9a	G12 - 1A0EPCA0	SWITCH LABEL FOR 301 (RAL9003)	1		B		
	G12 - 1A0HW0A0	SWITCH LABEL FOR 301 (CN80)			B		
9	G12 - 1P049XA0	BACK PLATE W/ SWITCH LABEL (RAL9003)	1		C		
	G12 - 1P04A7A0	BACK PLATE W/ SWITCH LABEL (CN80)			C		
10	G12 - ZA111LPY	PAN HEAD SCREW M4x8 CHROME	2		A		
11	G12 - 1A08AHB0	NOISE FILTER WITH WIRE	1		C		
12	G12 - ZA88JJPY	PAN HEAD SEMS SCREW 3P M3x8 CHROME	1		A		
13	G12 - ZA901JAY	FLANGE NUT M3 CHROME	1		A		
14	G12 - 1A07W1B0	ROTATION SHAFT	1		B		
15	G12 - 1A07W2A0	ROTATION STOPPER BOLT M8	1		A		
16	G12 - ZB511PAY	SPRING WASHER M8 CHROME	1		A		
17	G12 - 1A0STYB0	SENSOR/MANUAL MODE SWITCH ASSY WITH WIRE	1		C		
18	G12 - 1G0803A0	LIGHT CONTROL PCB	1		C		
19	G12 - 1G06SAA0	POWER PCB	1		C		
20	G12 - ZA111LPY	PAN HEAD SCREW M4x8 CHROME	4		A		
21	G12 - AEFR93B0	FUSE HOLDER F-400-01A2	2		C		
22	G12 - CAML90B0	TERMINAL BLOCK	1		C		
23	G12 - ZA88JJUY	PAN HEAD SEMS SCREW 3P M3x16 CHROME	2		A		
24	G12 - CAND32A0	WASHER 43x29x0.8	1		B		
25	G12 - CAO445CR	SWIVEL ARM 485MM (4 HOLES) (RAL9003)	1		C		
	G12 - 1A0HVSA0	SWIVEL ARM 485MM (4 HOLES) (CN80)			B		
26	G12 - ZA112LRY	FLAT HEAD SCREW M4x10	4		A		
27	G12 - CEEN94A0	FUSE 250V 0.8A (T)	2		C		
28	G12 - CEGN46A0	CORD BUSH	1		B		
29	G12 - CENX56B0	MAIN SWITCH	1		C		
30	G12 - CERN39A0	DOUBLE LOCKING SPACER	4		B		
31	G12 - ZB561OAM	WASHER M6x11.5x0.8 BRASS NICKEL	1		A		
32	G12 - CENX94A0	C-RING	1		B		
33	G12 - 1A0BMMA0	CT CODE 5000MM	1		B		
34	G12 - 1A0GPFB0	LIGHT ADAPTOR	1		B		
35	G12 - CENB34B0	HEX SET SCREW M5x8 CHROME	4		A		

POWER SUPPLY BOX (D102)

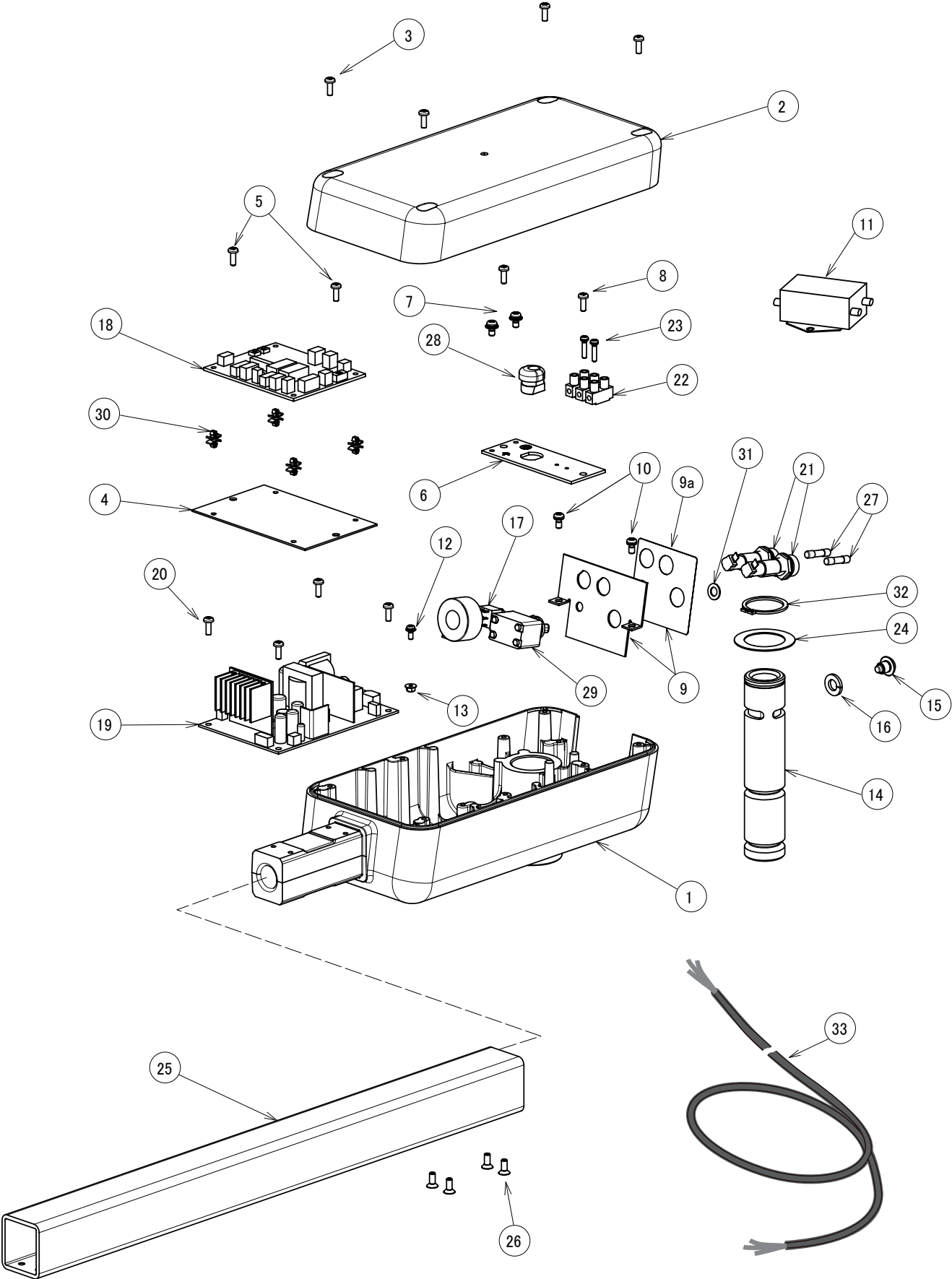



NO.	ID	PARTS NO.	DESCRIPTION	QTY	APPLICABLE SERIAL NO.	LEAD TIME	NOTE
1	G12 - 1A0BFMA0	HOUSING (RAL9003)	1		B		
	G12 - 1A0HVNA0	HOUSING (CN80)			B		
2	G12 - 1A0BFNA0	HOUSING COVER (RAL9003)	1		C		
	G12 - 1A0HVJA0	HOUSING COVER (CN80)			C		
3	G12 - ZA111LPY	PAN HEAD SCREW M4x8 CHROME	4		A		
4	G12 - 1A0BFPA0	CONTROL PCB FIXING PLATE	1		C		
5	G12 - ZA111LPY	PAN HEAD SCREW M4x8 CHROME	2		A		
6	G12 - 1A0BFQB0	ELECTRIC PARTS FIXING PLATE	1		C		
7	G12 - ZA88JLRL	PAN HEAD SEMS SCREW P3 M4x10 BRASS	2		A		
8	G12 - ZA111LPY	PAN HEAD SCREW M4x8 CHROME	2		A		
9a	G12 - 1A0EPDA0	SWITCH LABEL FOR 302 (RAL9003)	1		C		
	G12 - 1A0J2GA0	SWITCH LABEL FOR 302 (CN80)			B		
9	G12 - 1P049YA0	BACK PLATE WITH SWITCH LABEL (RAL9003)	1		C		
	G12 - 1P04A8A0	BACK PLATE WITH SWITCH LABEL (CN80)			C		
10	G12 - ZA111LPY	PAN HEAD SCREW M4x8 CHROME	2		A		
11	G12 - 1A08AHB0	NOISE FILTER WITH WIRE	1		C		
12	G12 - ZA88JJPY	PAN HEAD SEMS SCREW 3P M3x8 CHROME	1		A		
13	G12 - ZA901JAY	FLANGE NUT M3 CHROME	1		A		
14	G12 - 1A07W1B0	ROTATION SHAFT	1		B		
15	G12 - 1A07W2A0	ROTATION STOPPER BOLT M8	1		A		
16	G12 - ZB511PAY	SPRING WASHER M8 CHROME	1		A		
17	G12 - 1A0STYB0	S/M MODE SWITCH ASSY WITH WIRE	1		C		
18	G12 - 1G0803A0	LIGHT CONTROL PCB	1		C		
19	G12 - 1G06SAA0	POWER PCB	1		C		
20	G12 - ZA111LPY	PAN HEAD SCREW M4x8 CHROME	4		A		
21	G12 - AEFR93B0	FUSE HOLDER F-400-01A2	2		C		
22	G12 - CAML90B0	TERMINAL BLOCK	1		C		
23	G12 - ZA88JJUY	PAN HEAD SEMS SCREW 3P M3x16 CHROME	2		A		
24	G12 - CAND32A0	WASHER 43x29x0.8	1		B		
25	G12 - CAO445CR	SWIVEL ARM 485MM (4 HOLES) (RAL9003)	1		C		
	G12 - 1A0HVSA0	SWIVEL ARM 485MM (4 HOLES) (CN80)			B		
26	G12 - ZA112LRY	FLAT HEAD SCREW M4x10 CHROME	4		A		
27	G12 - CEEN94A0	FUSE 250V 0.8A (T)	2		C		
28	G12 - CEGN46A0	CORD BUSH	1		B		
29	G12 - CENX56B0	MAIN SWITCH	1		C		
30	G12 - CERN39A0	DOUBLE LOCKING SPACER	4		B		
31	G12 - ZB561OAM	WASHER M6x11.5x0.8 BRASS NICKEL	1		A		
32	G12 - CENX94A0	C-RING STW-29	1		B		
33	G12 - CADQ71B0	POWER CABLE 3200MM (SJT)	1		B		

POWER SUPPLY BOX (D105)

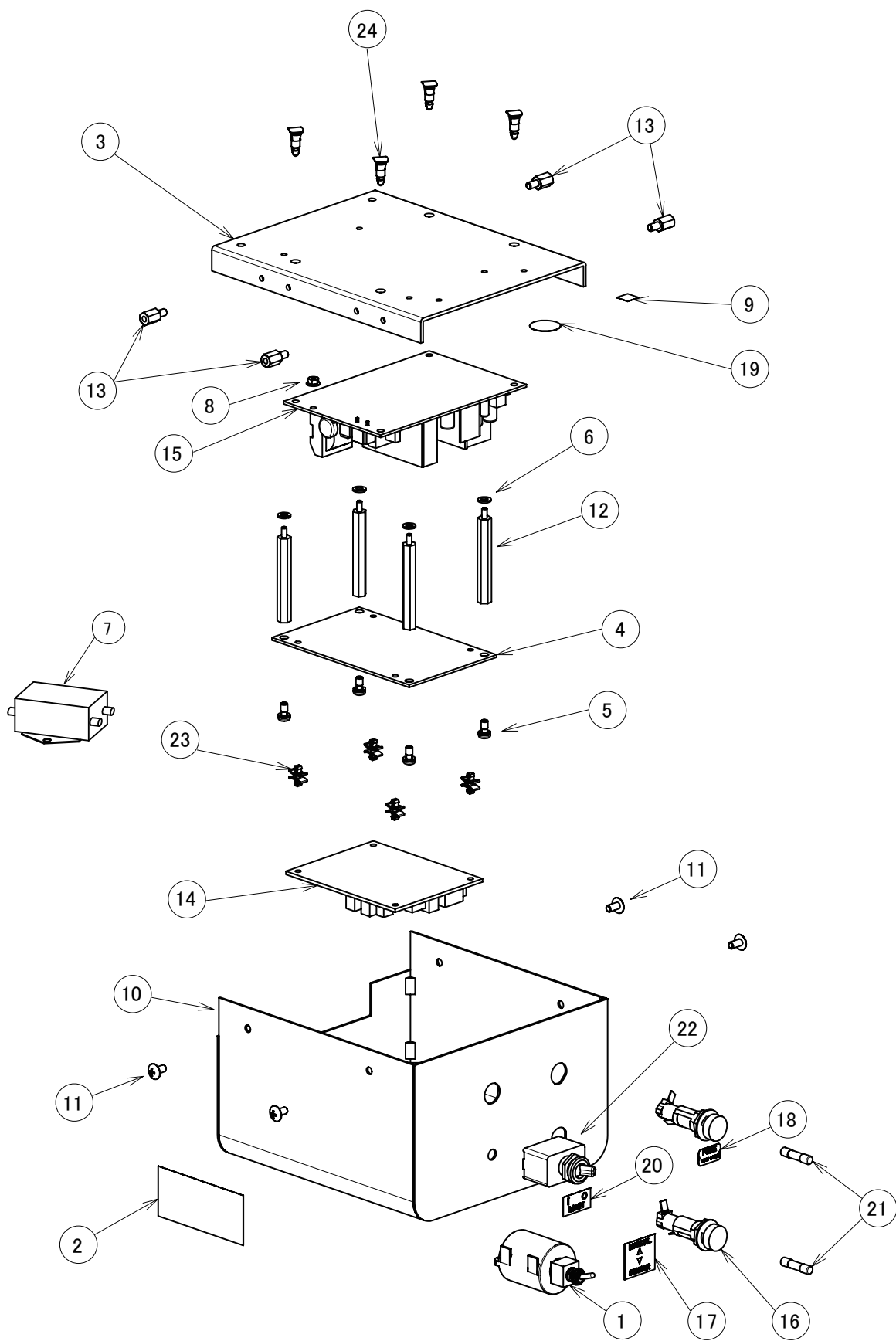


NO.	ID	PARTS NO.	DESCRIPTION	QTY	APPLICABLE SERIAL NO.	LEAD TIME	NOTE
1	G12 - 1A0STYB0		S/M MODE SWITCH ASSY (WITH WIRE)	1		C	
2	G12 - 1A0HQZA0		CAUTION LABEL (ENGLISH AND FRENCH)	1		A	
3	G12 - 1A07M5B0		POWER PCB FIXING PLATE	1		B	
4	G12 - 1A07PKA0		SENSOR CONTROL PCB FIXING PLATE	1		C	
5	G12 - ZA887LPY		PAN HEAD SEMS SCREW 2P M4x8 CHROME	4		A	
6	G12 - ZB511LAY		SPRING WASHER M4 CHROME	1		A	
7	G12 - 1A08AJB0		NOISE FILTER WITH WIRE	1		B	
8	G12 - ZA901JAY		FLANGE NUT M3 CHROME	1		A	
9	G12 - 1A07YUA0		EARTH LABEL	1		A	
10	G12 - 1A08BAA0		POWER SUPPLY BOX COVER (RAL9003)	1		B	
	G12 - 1A07M8A0		POWER SUPPLY BOX COVER (CN80)			B	
11	G12 - 1E01HHA0		TRUSS M4x8 RAL9003	4		A	
	G12 - 1E02HPA0		TRUSS M4x8 CN80			A	
12	G12 - 1E02E9A0		METAL SPACER	4		A	
13	G12 - 1E04S0A0		HEX METAL SPACER	4		A	
14	G12 - 1G0803A0		LIGHT CONTROL PCB	1		C	
15	G12 - 1G06SAA0		POWER PCB	1		C	
16	G12 - AEFR93B0		FUSE HOLDER F-400-01A2	2		C	
17	G12 - CADR07A0		LABEL FOR S/M MODE SWITCH	1		B	
18	G12 - CAEP84B0		FUSE LABEL 0.8A	1		A	
19	G12 - CAMT05A0		EARTH MARK (N-7)	1		B	
20	G12 - CARO04A0		LABEL FOR MAIN SWITCH	1		B	
21	G12 - CEEN94A0		FUSE 250V 0.8A (T)	2		C	
22	G12 - CENX56B0		MAIN SWITCH	1		C	
23	G12 - CERN39A0		DOUBLE LOCKING SPACER	4		B	
24	G12 - DESR05A0		SPACER	4		A	

POWER SUPPLY BOX (D106G-QU)

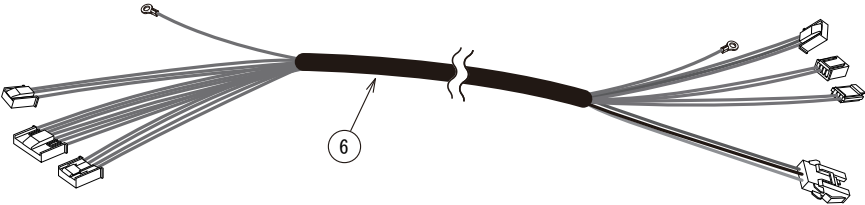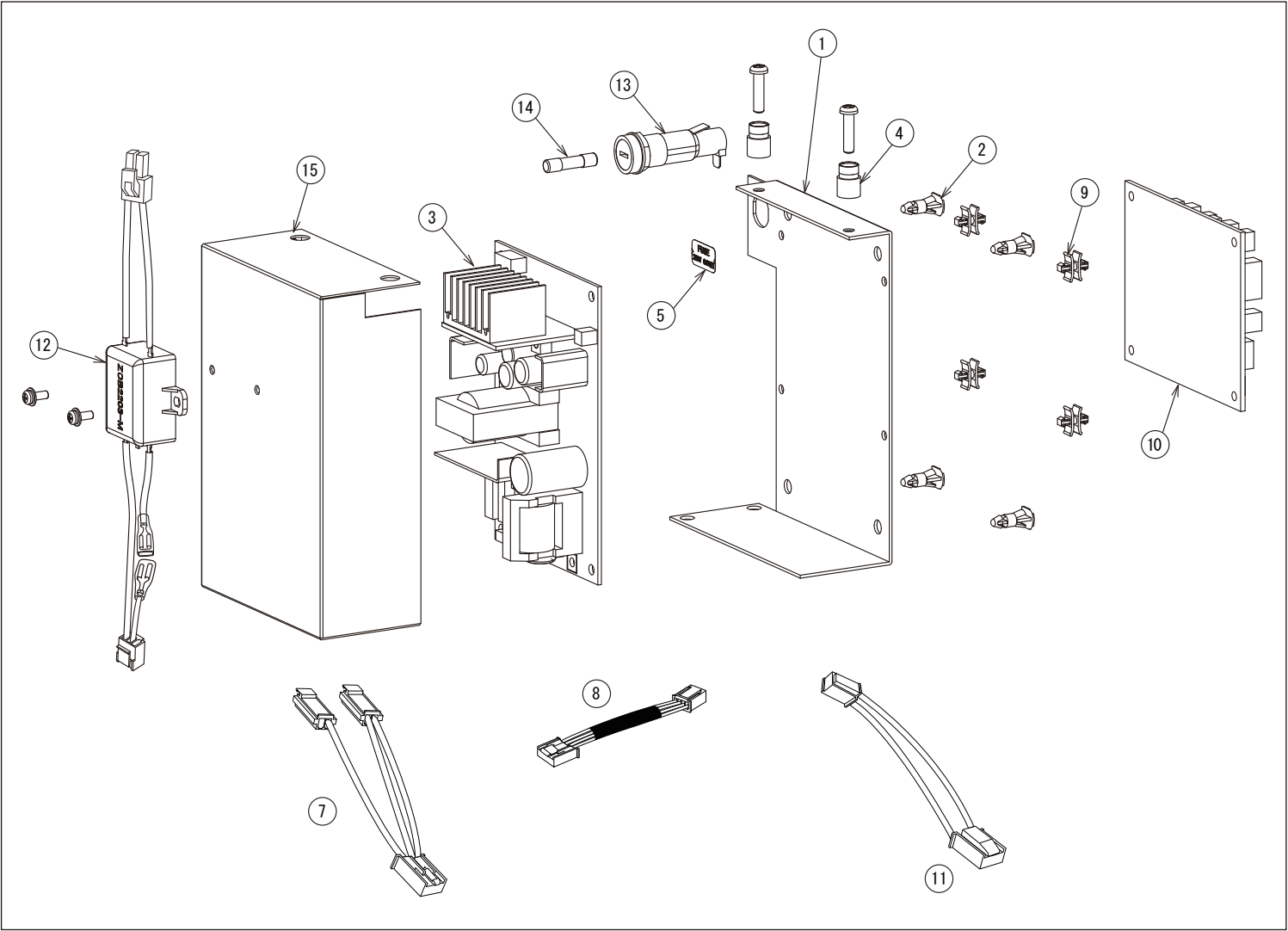


NO.	ID	PARTS NO.	DESCRIPTION	QTY	APPLICABLE SERIAL NO.	LEAD TIME	NOTE
1	G12 - 1A07Z8A0		PLATE FOR POWER PCB	1		B	
2	G12 - DESR05A0		SPACER	4		A	
3	G12 - 1G06SAA0		POWER PCB	1		C	
4	G12 - 1E02FBA0		SNAP COLOR BUSH	2		B	
5	G12 - CAEP84B0		FUSE LABEL 0.8 A (T)	1		A	
6	G12 - 1A0SUNA0		TRUNK LINE FOR S/M SWITCH (D106Q)	1		C	
7	G12 - 1A0STXA0		TRUNK LINE FOR ADJUST PULSE (AL-D101/102)	1		C	
8	G12 - 1A07MRA0		LED MODULE RELAY CODE (POWER SOURCE SIDE)	1		B	
9	G12 - CERN39A0		DOUBLE LOCKING SPACER WLS-04-0	4		B	
10	G12 - 1G07PBA0		SENSOR CONTROL PCB	1		C	
11	G12 - 1A0871A0		SERIAL COMMUNICATION CODE	1		B	
12	G12 - 1A08AGB0		NOISE FILTER WITH WIRE	1		C	
13	G12 - AEFR93B0		FUSE HOLDER F-400-01A2	1		C	
14	G12 - CEEN94A0		FUSE 250V 0.8A(T)	1		C	
15	G12 - 1A07Z9B0		POWER PCB COVER (QUOLIS)	1		B	

SWITCH BOX (D106G-QU)

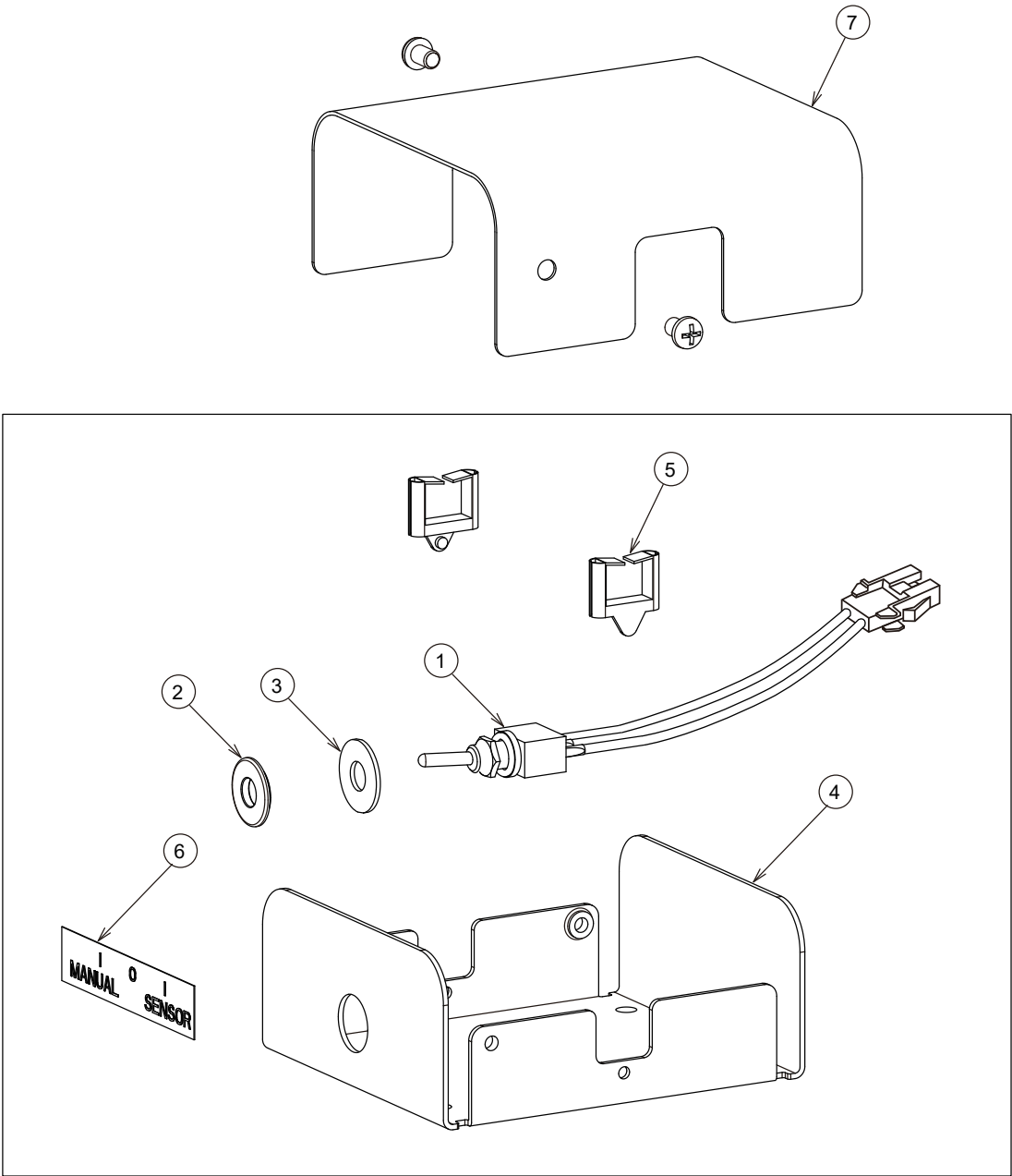


NO.	ID	PARTS NO.	DESCRIPTION	QTY	APPLICABLE SERIAL NO.	LEAD TIME	NOTE
1	G12 - 1A086ZA0		S/M CHANGING SWITCH	1		B	
2	G12 - 1A07PYC0		S/M CHANGING SWITCH SPACER	1		B	
3	G12 - 1E00V3A0		WASHER M6x16x1.0 CHROME	1		A	
4	G12 - CAGP86A1		SWITCH BOX FLAME	1		C	
5	G12 - CEGN75A0		EDGE SADDLE	2		B	
6	G12 - CAJC01A0		POWER SWITCH LABEL (NDL/ UK)	1		B	
7	G12 - CAGP87A1		SW BOX COVER	1		C	

POWER SUPPLY BOX (D106G-EV , D109)

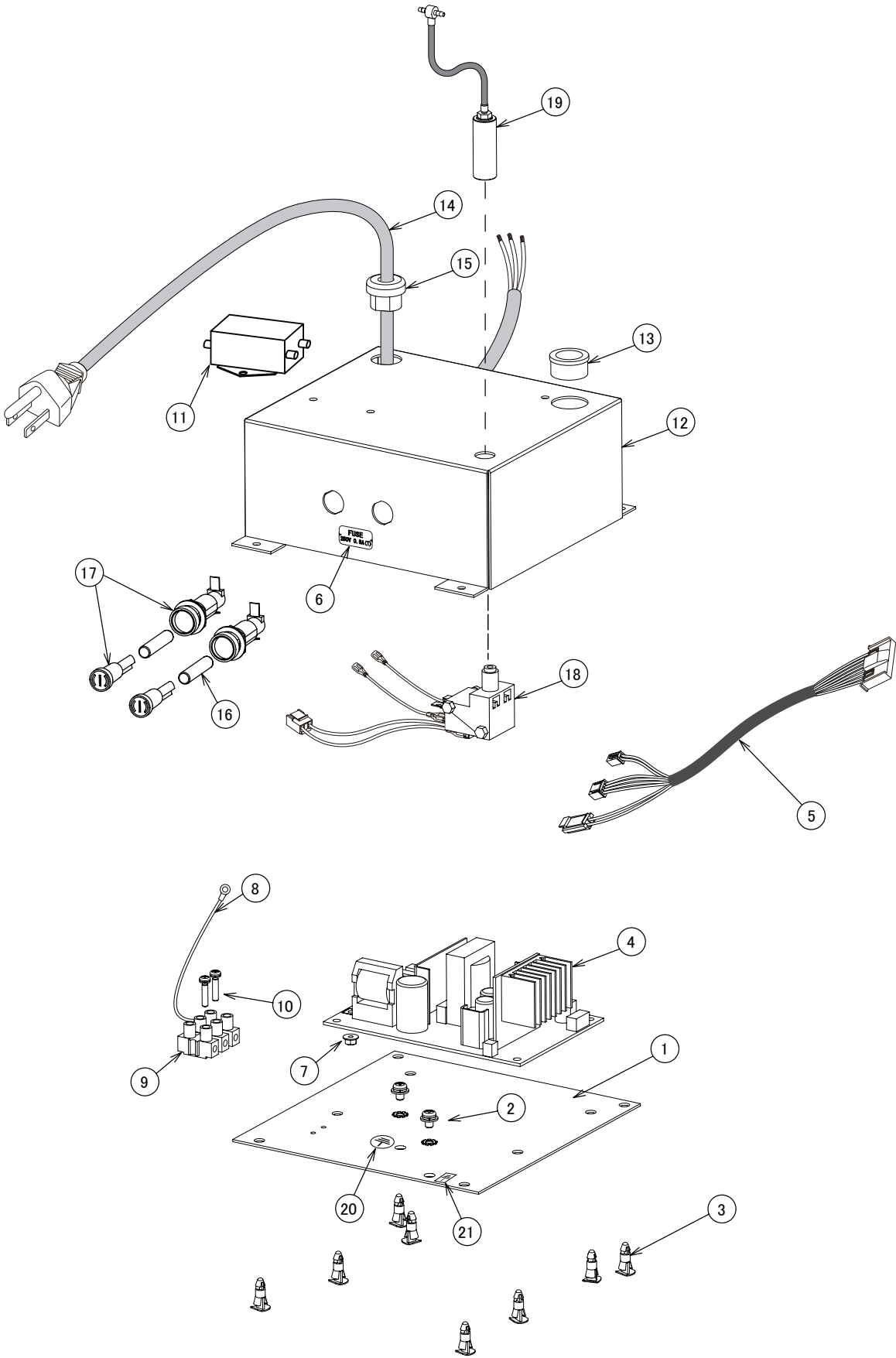


NO.	ID	PARTS NO.	DESCRIPTION	QTY	APPLICABLE SERIAL NO.	LEAD TIME	NOTE
1	G12 - 1A08BWA0		POWER SUPPLY PCB BOX FIXING PLATE	1		B	
2	G12 - ZA88JLNM		PAN HEAD SEMS SCREW 3P M4x6 NICKEL	2		A	
3	G12 - DESR05A0		SPACER	8		A	
4	G12 - 1G06SAA0		POWER PCB	1		C	
5	G12 - 1A0SUFA0		TRUNK LINE FOR POWER PCB(D109)	1		C	
6	G12 - CAEP84B0		FUSE LABEL 0.8A	1		A	
7	G12 - ZA901JAY		FLANGE NUT M3 CHROME	1		A	
8	G12 - CADQ74A0		EARTH CABLE	1		C	
9	G12 - CAML90B0		TERMINAL BLOCK	1		C	
10	G12 - ZA887JTY		PAN HEAD SEMS SCREW 2P M3x14 CHROME	2		A	
11	G12 - 1A08AJB0		NOISE FILTER WITH WIRE	1		B	
12	G12 - 1A08BVB0		POWER SUPPLY BOX COVER	1		C	
13	G12 - AFFX96A0		CODE BUSH	1		B	
14	G12 - 1A04D6A0		POWER SUPPLY CODE 1000L	1		C	
15	G12 - CESN07A0		RELIEF BUSHING SR-6N3-4	1		C	
16	G12 - CEEN94A0		FUSE 250V 0.8A (T)	2		C	
17	G12 - AEFR93B0		FUSE HOLDER F-400-01A2	2		C	
18	G12 - 1G02LKA0		AIR SWITCH BODY ASSY	1		C	
19	G12 - 1G02MBA0		AIR SWITCH CYLINDER ASSY WITH TUBING & FITTING	1		C	
20	G12 - CAMT05A0		EARTH MARK	1		B	
21	G12 - 1A07YUA0		EARTH LABEL	1		A	
1 to 21	G12 - 1G082UA0		POWER SUPPLY BOX ASSY	1		C	

SWITCH BOX (D106G-EV , D109W-X)

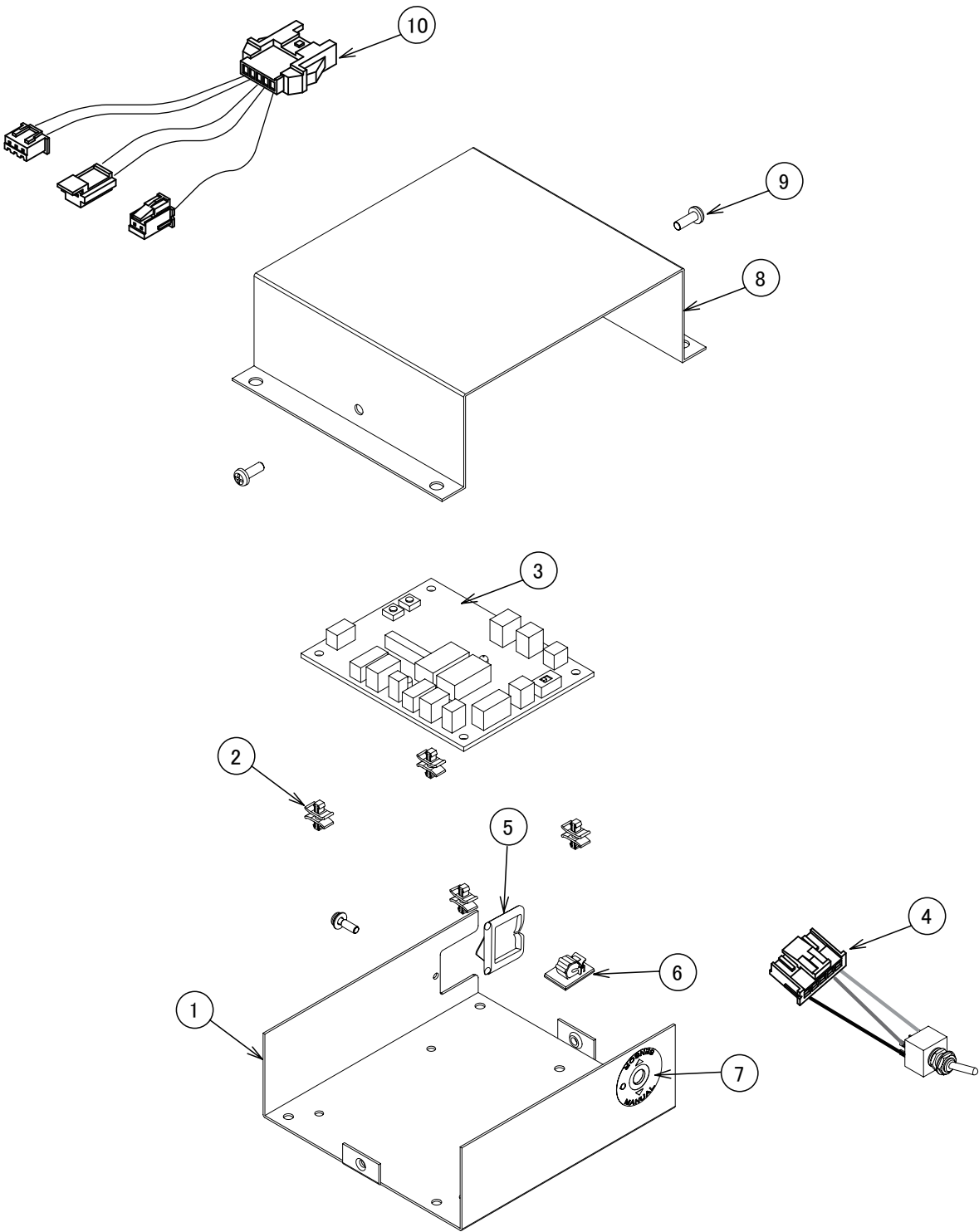


NO.	ID	PARTS NO.	DESCRIPTION	QTY	APPLICABLE SERIAL NO.	LEAD TIME	NOTE
1	G12 - 1A08CTA0	SWITCH BOX (RAL9003)		1		B	
	G12 - 1A0HVMA0	SWITCH BOX (CN80)				B	
2	G12 - CERN39A0	DOUBLE LOCKING SPACER (WLS-04-0)		4		B	
3	G12 - 1G0803A0	LIGHT CONTROL PCB		1		C	
4	G12 - 1A0SUJA0	S/M MODE SWITCH ASSY		1		C	
5	G12 - CETN14A0	EDGE SADDLE FOR CABLE PROTECTION		1		B	
6	G12 - CETN13A0	CLAMP WITH SPONGE		1		B	
7	G12 - 1A0HMYA0	LABEL FOR S/M MODE SWITCH		1		B	
8	G12 - CATN46CR	SWITCH BOX COVER (RAL9003)		1		C	
	G12 - 1A0HWWA0	SWITCH BOX COVER (CN80)				B	
9	G12 - 1E00VTA0	PAN HEAD SEMS SCREW M4x10 RAL9003		2		A	
	G12 - 1A033FA0	PAN HEAD SCREW M4x10 CN80				A	
10	G12 - 1A0SUHA0	TRUNK LINE FOR ADJUST PULSE(AL-D109-X)		1		C	

SWITCH BOX (D109W-XN)

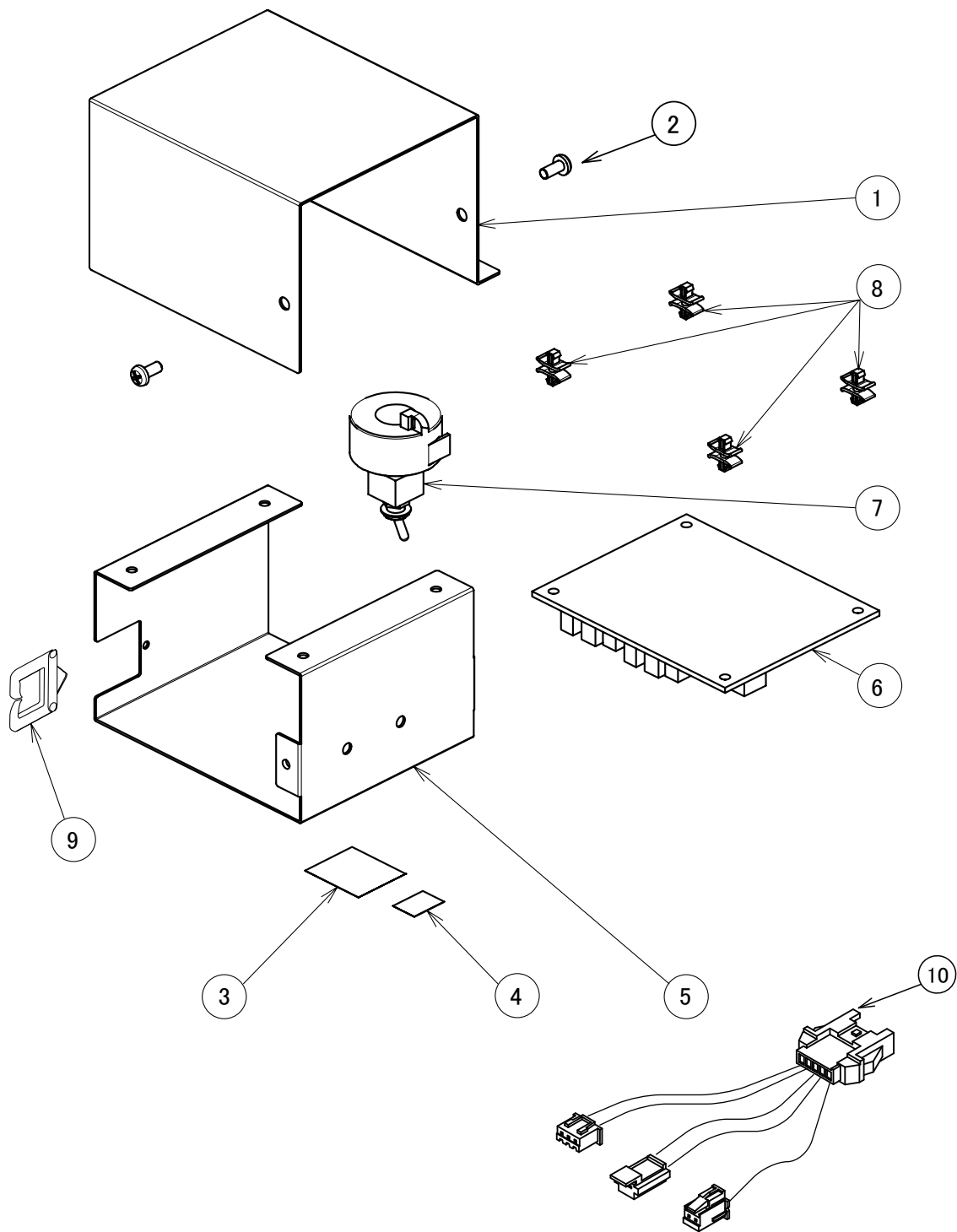


NO.	ID	PARTS NO.	DESCRIPTION	QTY	APPLICABLE SERIAL NO.	LEAD TIME	NOTE
1	G12 - 1A0AFRA0		SWITCH BOX COVER	1		C	
2	G12 - ZA887LPU		PAN HEAD SEMS SCREW 2P M4x8 SUS	2		A	
3	G12 - CADR07A0		LABEL FOR S/M MODE SWITCH	1		B	
4	G12 - U-31860		LABEL FOR POWER SUPPLY	1		A	
5	G12 - 1A0B6SA0		SWITCH BOX	1		C	
6	G12 - 1G0803A0		LIGHT CONTROL PCB	1		C	
7	G12 - 1A0SUJA0		S/M MODE SWITCH ASSY	1		C	
8	G12 - CERN39A0		DOUBLE LOCKING SPACER (WLS-04-0)	4		B	
9	G12 - CETN14A0		EDGE SADDLE FOR CABLE PROTECTION	1		B	
10	G12 - 1A0SUHA0		TRUNK LINE FOR ADJUST PULSE(AL-D109-X)	1		C	

POWER SUPPLY BOX (D107)

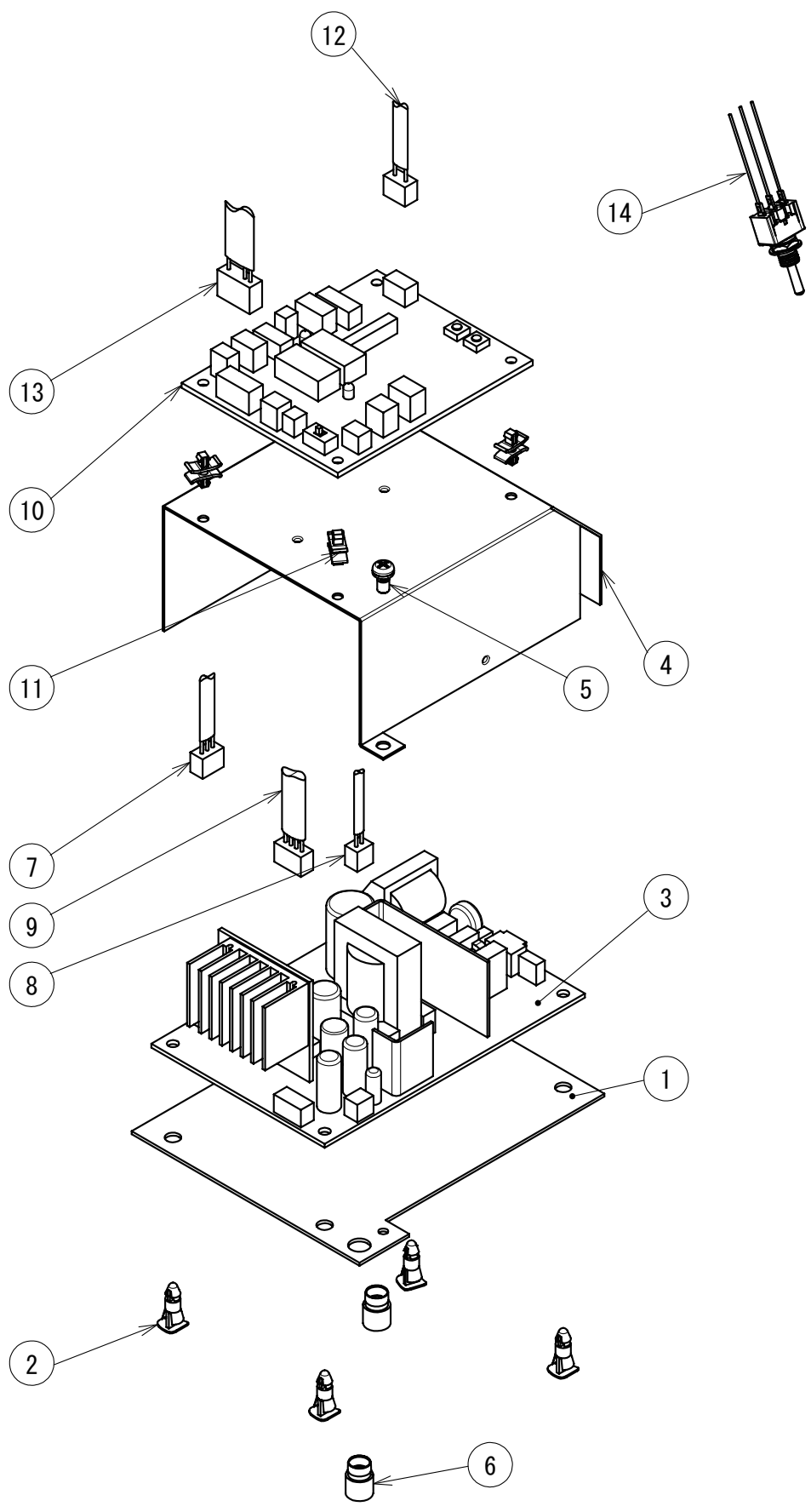


NO.	ID	PARTS NO.	DESCRIPTION	QTY	APPLICABLE SERIAL NO.	LEAD TIME	NOTE
1	G12 - 1A07NUA0		PLATE FOR POWER PCB	1		C	
2	G12 - DESR05A0		SPACER	4		A	
3	G12 - 1G06SAA0		POWER PCB	1		C	
4	G12 - 1A07X1D0		COVER FOR POWER PCB	1		B	
5	G12 - ZA887LPY		PAN HEAD SEMS SCREW 2P M4x8 CHROME	2		A	
6	G12 - 1E02FBA0		SNAP COLOR BUSH	2		B	
7	G12 - 1A0SU1A0		TRUNK LINE A FOR ADJUST PULSE	1		B	
8	G12 - 1A07MPA0		TRUNK LINE A FOR CONTROL PCB	1		B	
9	G12 - 1A07MRA0		TRUNK LINE FOR LED MODULE	1		B	
10	G12 - 1G0803A0		LIGHT CONTROL PCB	1		C	
11	G12 - CERN39A0		SPACER	4		B	
12	G12 - 1A07MQA0		TRUNK LINE B FOR CONTROL PCB	1		B	
13	G12 - 1A0SU3A0		TRUNK LINE FOR SENSOR/MANUAL SWITCH	1		B	
14	G12 - 1A0SU4A0		SENSOR/MANUAL SWITCH ASSY	1		C	

POWER SUPPLY BOX (D108)

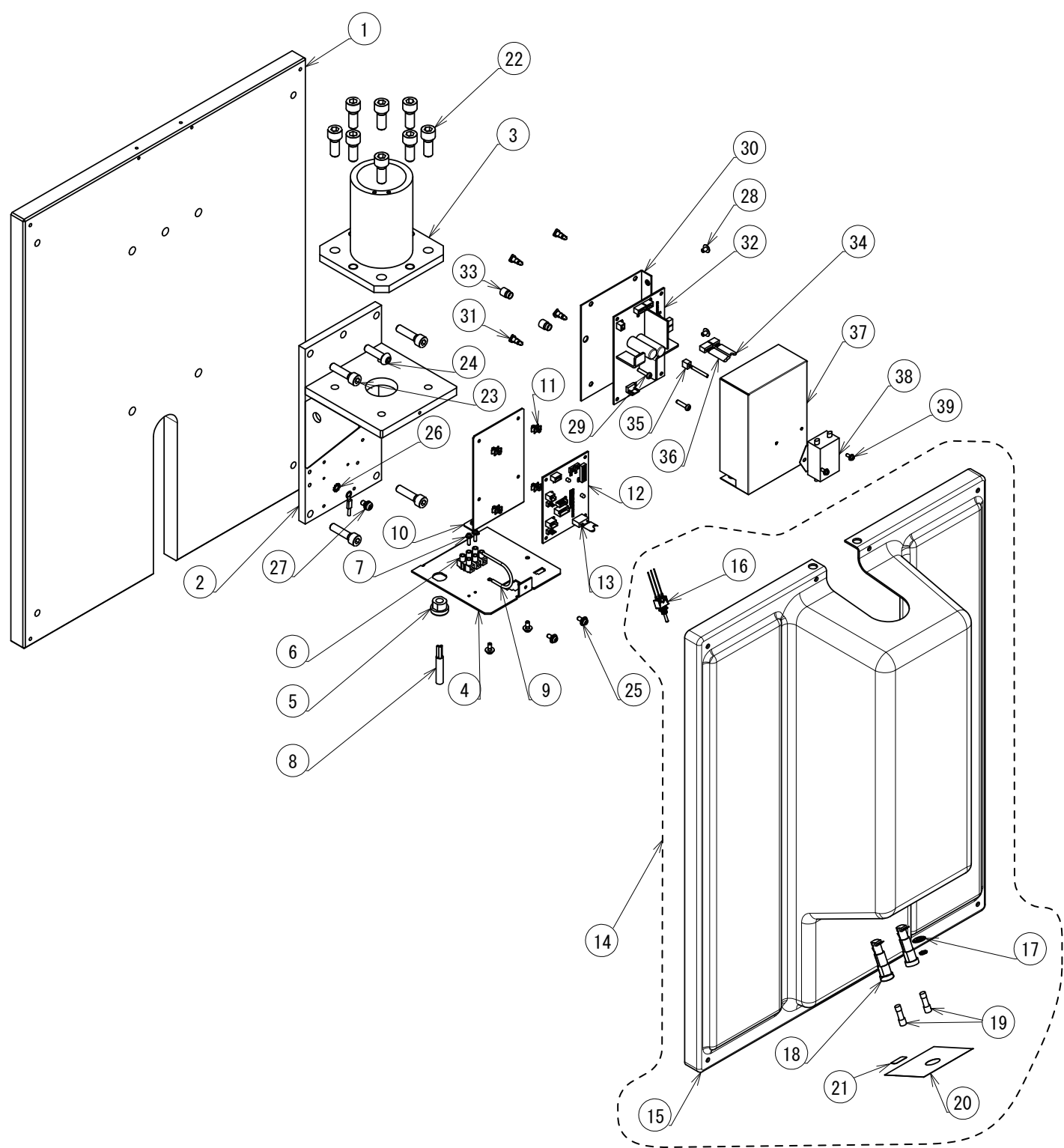

NO.	ID	PARTS NO.	DESCRIPTION	QTY	APPLICABLE SERIAL NO.	LEAD TIME	NOTE
1	G12 - 1A0QB3A0		WALL PLATE FOR EPA	1		C	
2	G12 - CATN01D0		BRACKET FOR WALL	1		C	
3	G12 - CATN02B0		BRACKET FOR LIGHT POLE	1		C	
4	G12 - 1A0ALWA0		BOTTOM PLATE	1		C	
5	G12 - CESN07A0		BUSH	1		C	
6	G12 - CAML90B0		TERMINAL BLOCK	1		C	
7	G12 - ZA88MJSY		PAN HEAD SEMS SCREW 4P M3x12 CHROME	2		A	
8	G12 - 1A04CZA0		CT CODE 2000L	1		C	
9	G12 - 1A04CXA0		EARTH WIRE 200L	1		A	
10	G12 - 1A0ALXA0		PLATE FOR PCB	1		B	
11	G12 - CERN39A0		SPACER	4		B	
12	G12 - 1G0803A0		LIGHT CONTROL PCB	1		C	
13	G12 - 1A0SU8A0		TRUNK LINE A FOR SENSOR/MANUAL SWITCH	1		B	
14	G12 - 1G0827A0		FRONT COVER SET	1		C	
15	G12 - 1A0ALYA0		FRONT COVER CN80	1		B	
16	G12 - 1A0SU9A0		SENSOR/MANUAL SWITCH ASSY	1		C	
17	G12 - 1A07PYC0		SPACER FOR SENSOR/MANUAL SWITCH	1		B	
18	G12 - AEFR93B0		FUSE HOLDER	2		C	
19	G12 - CEEN94A0		FUSE 250V 0.8A (T)	2		C	
20	G12 - CATP11B0		LABEL FOR SENSOR/MANUAL	1		B	
21	G12 - CAEP84B0		FUSE LABEL 0.8 A (T)	1		B	
22	G12 - ZA761QYU		HEX SOCKET HEAD SCREW M10x25 SUS	8		A	
23	G12 - CETN56A0		HEX SOCKET HEAD SCREW M8x32 BLACK CHROME	4		A	
24	G12 - ZSA11PZG		HEX BUTTON HEAD SCREW M8x30 BLACK	1		A	
25	G12 - ZA88JLRY		PAN HEAD SEMS SCREW 3P M4x10 CHROME	4		A	
26	G12 - ZB552NAM		EXTERNAL TOOTH WASHER M5 BRASS NICKEL	1		A	
27	G12 - ZA88JNRL		PAN HEAD SEMS SCREW 3P M5x10 BRASS	1		A	
28	G12 - ZA114LNY		TRUSS M4x6 CHROME	2		A	
29	G12 - ZA111LUC		PAN HEAD SCREW M4x16 CHROME	2		A	
30	G12 - 1A07X7A0		BRACKET FOR POWER PCB	1		B	
31	G12 - DESR05A0		SPACER	4		A	
32	G12 - 1G06SAA0		POWER SUPPLY PCB	1		C	
33	G12 - 1E02FBA0		SNAP COLOR BUSH	2		B	
34	G12 - 1A0STXA0		TRUNK LINE FOR ADJUST PULSE	1		C	
35	G12 - 1A07MPA0		TRUNK LINE A FOR CONTROL PCB	1		B	
36	G12 - 1A07MRA0		TRUNK LINE FOR LED MODULE	1		B	
37	G12 - 1A07Z9B0		POWER PCB BOX	1		B	
38	G12 - 1A09GHB0		NOISE FILTER WITH WIRE	1		B	
39	G12 - ZA88JJPY		PAN HEAD SEMS SCREW 3P M3x8 CHROME	2		A	

CEILING POLE (D102)

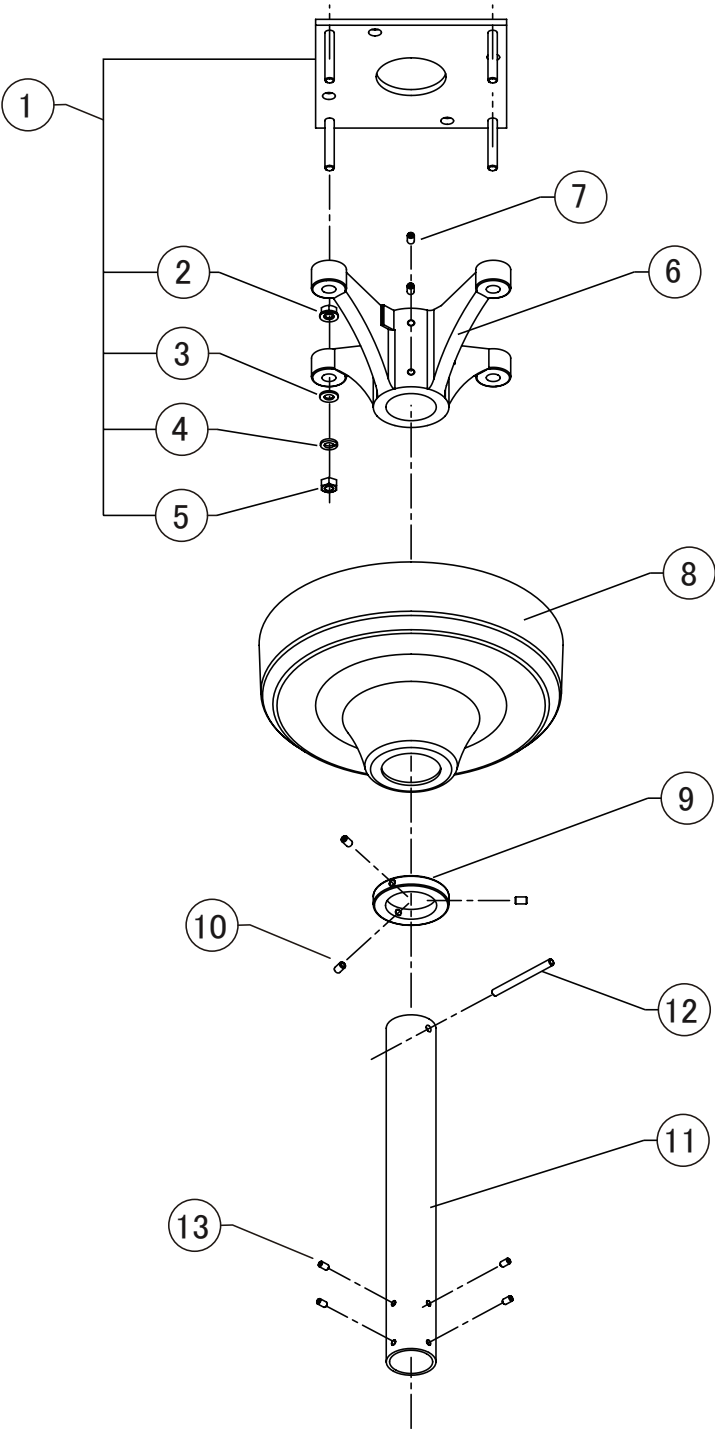


NO.	ID	PARTS NO.	DESCRIPTION	QTY	APPLICABLE SERIAL NO.	LEAD TIME	NOTE
1 to 5	G12 - 1P0211A0		MOUNTING PLATE WITH NUT	1		C	
2	G12 - ZA901PAY		FLANGE NUT M8 CHROME	4		A	FOR ADJUST LEVEL
3	G12 - CENE00B0		WASHER M8x22x1.63	4		A	
4	G12 - ZB511PAY		SPRING WASHER M8 CHROME	4		A	
5	G12 - 1E00ABA0		U NUT M8	4		A	
6	G12 - TCPA01A0		CEILING FLANGE	1		B	
7	G12 - ZA7740PU		HEX SOCKET SET SCREW (CUP POINT) M6x8 SUS	4		A	
8	G12 - 1A0827A0		FLANGE COVER (CN80)	1		C	
9	G12 - CAMG76A0		COVER RING	1		C	
10	G12 - CENB35B0		HEX SOCKET SET SCREW M6x8 (CUP POINT) (RED MEC THREAD LOCK)	3		C	
11	G12 - 1G02TLA0		SUSPENSION TUBE SET 340MM (CN80) 8FT	1		C	
	G12 - 1G02TMA0		SUSPENSION TUBE SET 510MM (CN80) 8.5FT			C	
	G12 - 1G02TNA0		SUSPENSION TUBE SET 680MM (CN80) 9FT			C	
	G12 - 1G02TPA0		SUSPENSION TUBE SET 840MM (CN80) 9.5FT			C	
	G12 - 1G02TQA0		SUSPENSION TUBE SET 1000MM (CN80) 10FT			C	
12	G12 - CAMK40A0		ROLL PIN 6x70	1		A	
13	G12 - ZA772NRY		HEX SOCKET SET SCREW M5x10 (TRUNCATED CONE POINT)	4		B	
-	G12 - CGRN38B0		CEILING SCREW SET	1		C	

* Unnecessary if the pole length is shorter than 480mm


NO.	ID	PARTS NO.	DESCRIPTION	QTY	APPLICABLE SERIAL NO.	LEAD TIME	NOTE
1	G12 - 1A0824A0		RAIL (HAMMER TONE COATING CN80)	1		C	
2	G12 - CAML90B0		TERMINAL BLOCK	1		C	
3	G12 - CAMG90A0		CENTER COVER HOOK (OP/NDL)	1		C	
4	G12 - CAMG88B0		DRUM SHAFT	1		C	
5	G12 - CAMG91A0		CONSTANT FORCE SPRING PLATE	1		C	
6	G12 - CAMP70B0		PULLEY (OP/NDL) DURACON	1		B	
7	G12 - CAMP71C0		DRUM A DURACON	1		C	
8	G12 - CAMP72C0		DRUM B DURACON	1		C	
9	G12 - CAMU63A0		CONSTANT FORCE SPRING	1		C	
10	G12 - CAMV62A0		GUIDE (OP/NDL)	1		C	
11	G12 - CAQN96B0		DRUM COLLAR (WHITE POM)	1		C	
12	G12 - 1G0133A0		PULLEY ASSY	1		C	
13	G12 - ABNL23A0		STOPPER	5		B	
14	G12 - 1G02EEA0		END CAP ASSY (CN80)	1		C	
15	G12 - 1P03CYA0		BOTTOM COVER ASSY (CN80)	1		C	
16	G12 - CAMG95A0		TROLLEY COLLAR	1		C	
17	G12 - CAMV63A0		LEAD WIRE GUIDE (OP/NDL)	1		C	
18	G12 - CENZ15A0		NYLON CLAMP NK-4N	2		A	
19	G12 - CGAN76A0		ROLLER UPPER (OPNDL) ASSY	4		B	
20	G12 - CGAN77A0		ROLLER LOWER (OPNDL) ASSY	4		B	
21	G12 - 1A0823B0		TROLLEY (HAMMER TONE COATING, CN80)	1		C	
22	G12 - 1A087RA0		TRUNK LINE A (POWER SUPPLY BOX TO YOKE)	1		C	
23	G12 - CAMP73B0		LEAD WIRE COVER	1		C	
24	G12 - 1A087TA0		TRUNK LINE C (POWER SUPPLY BOX TO YOKE)	1		C	
25	G12 - 1G02U2A0		T POLE SET 380MM (CN80) 8FT	1		C	
	G12 - 1G02U3A0		T POLE SET 530MM (CN80) 8.5FT			C	
	G12 - 1G02U4A0		T POLE SET 680MM (CN80) 9FT			C	
	G12 - 1G02U5A0		T POLE SET 840MM (CN80) 9.5FT			C	
	G12 - 1G02U6A0		T POLE SET 1000MM (CN80) 10FT			C	
26	G12 - CAPN00D0		PALLET (UL TYPE)	1		C	
-	G12 - 1G009XA0		PALLET SCREW SET	1		C	
-	G12 - CGRN39B0		TRACK SCREW SET	1		C	

SWIVEL ARM (D107)

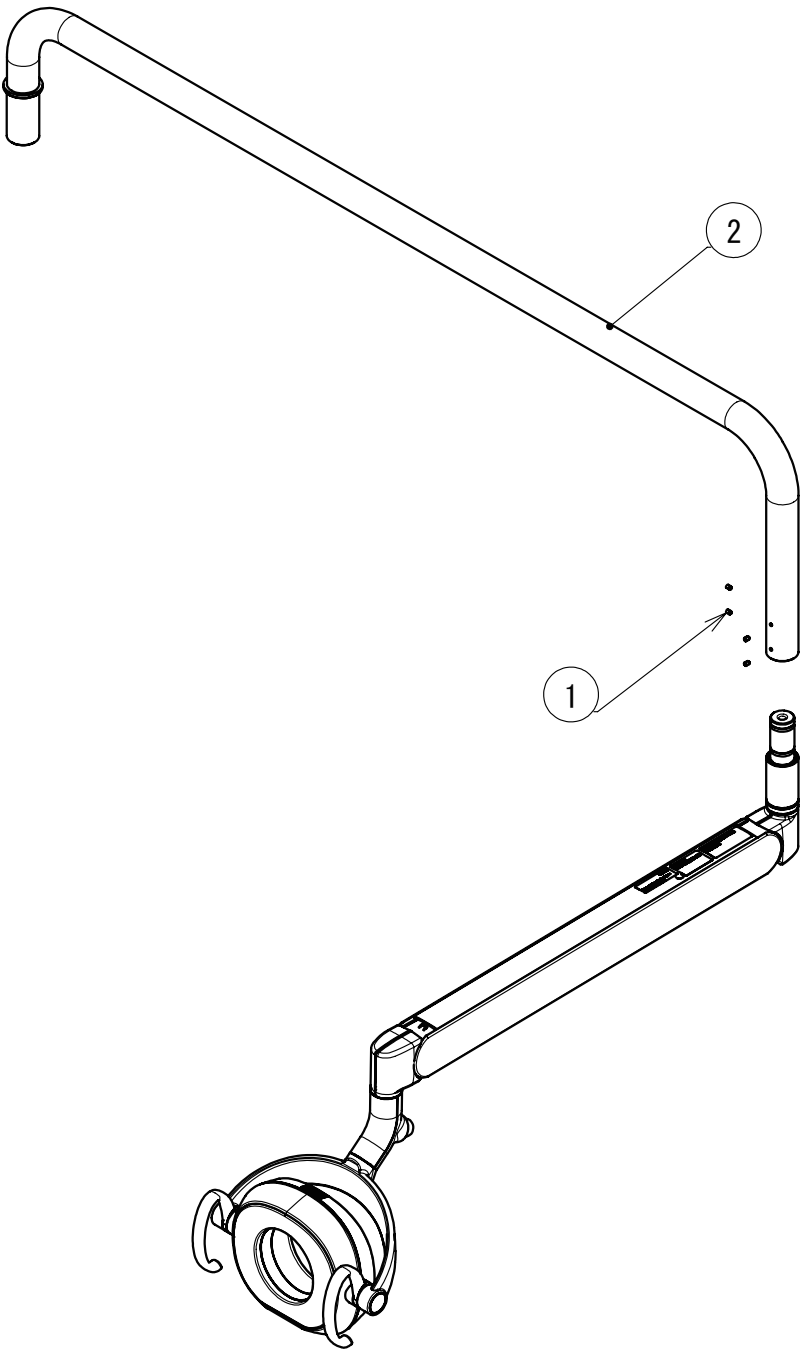

NO.	ID	PARTS NO.	DESCRIPTION	QTY	APPLICABLE SERIAL NO.	LEAD TIME	NOTE
1	G12 - ZA772NPY		HEX SOCKET SET SCREW M5x8 CHROME	4		B	
2	G12 - 1A0AH6A0		LIGHT POLE (CN80)	1		C	

SWIVEL ARM (D108)

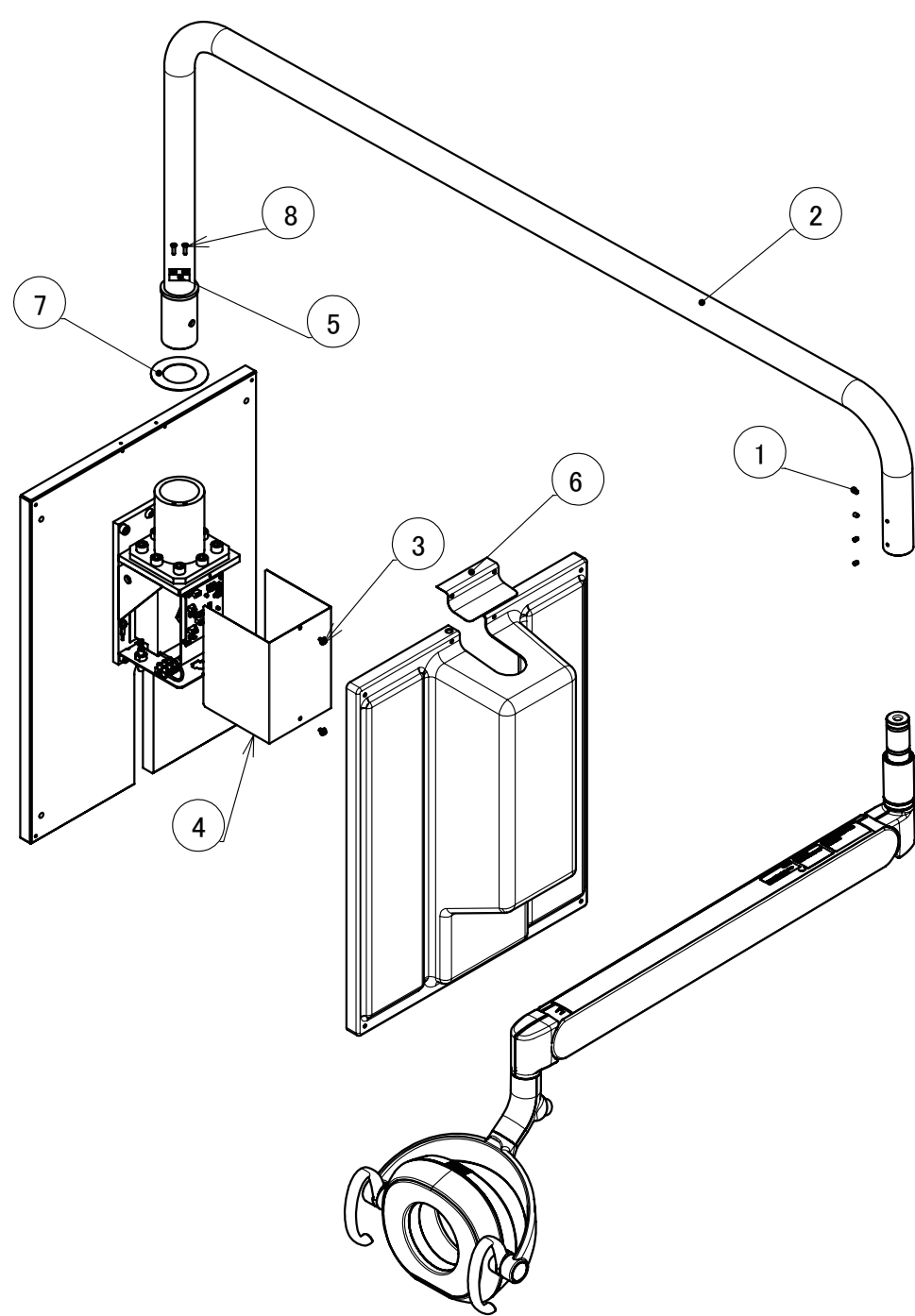

NO.	ID	PARTS NO.	DESCRIPTION	QTY	APPLICABLE SERIAL NO.	LEAD TIME	NOTE
1	G12 -	ZA772NPY	HEX SOCKET SET SCREW M5x8 CHROME	4		B	
2	G12 -	1A0ALVA0	LIGHT POLE (CN80)	1		C	
3	G12 -	ZA88JLPY	PAN HEAD SEMS SCREW 3P M4x8 CHROME	2		A	
4	G12 -	CATP03B0	COVER FOR BRACKET	1		B	
5	G12 -	CATP04A0	RETAINER METAL	1		C	
6	G12 -	1A0ALZA0	FRONT TOP COVER	1		C	
7	G12 -	CATN92A0	POLE DISK	1		B	
8	G12 -	ZA112L#U	FLAT HEAD SCREW M4x15 SUS	2		A	

Bel-Nova DENTAL LIGHT PARTS LIST REVISION RECORD

DATE		CONTENT
23-Mar-20	1st	Publish
14-Apr-21	2nd	Regularly update and add light adaptor



TAKARA BELMONT CORPORATION

2-1-1, Higashishinsaibashi, Chuo-ku, Osaka, 542-0083, Japan

TEL : +81-6-6213-5945

FAX : +81-6-6212-3680

BOOK No.1E07USB0

