

QUOLIS CHAIR (MODEL Q5500)
EVOGUE CHAIR (MODEL Q3300)
MAINTENANCE AND
ADJUSTMENT MANUAL

Chair SV Flow Adjustment

- Loosen Chair Up Speed
- Tighten Chair Up Speed



Video




Adjustment
Pass: Belmont

Video



Replacement
Pass: Belmont

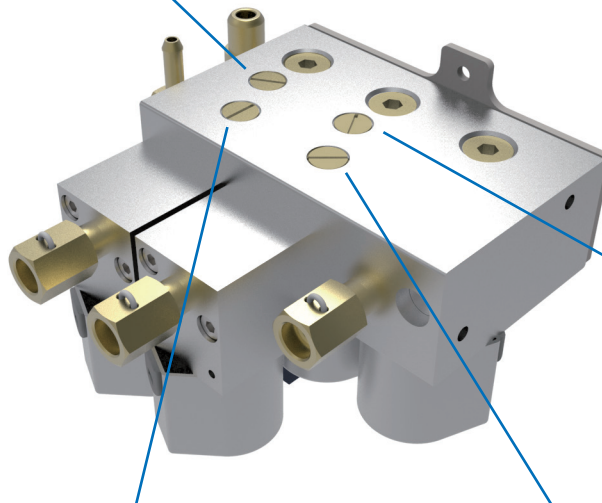

- Loosen Backrest Raise Speed
- Tighten Backrest Raise Speed

- Loosen Chair Down Speed
- Tighten Chair Down Speed

- Loosen Backrest Recline Speed
- Tighten Backrest Recline Speed



Oil Type




Oil: Mobil DTE 24
ISO VG32



Chair Limit Setting




Video
Pass: Belmont

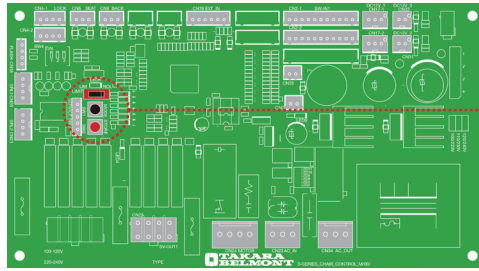

CHAIR PCB

LIM NOL
LIMIT SWITCH

STORE SWITCH

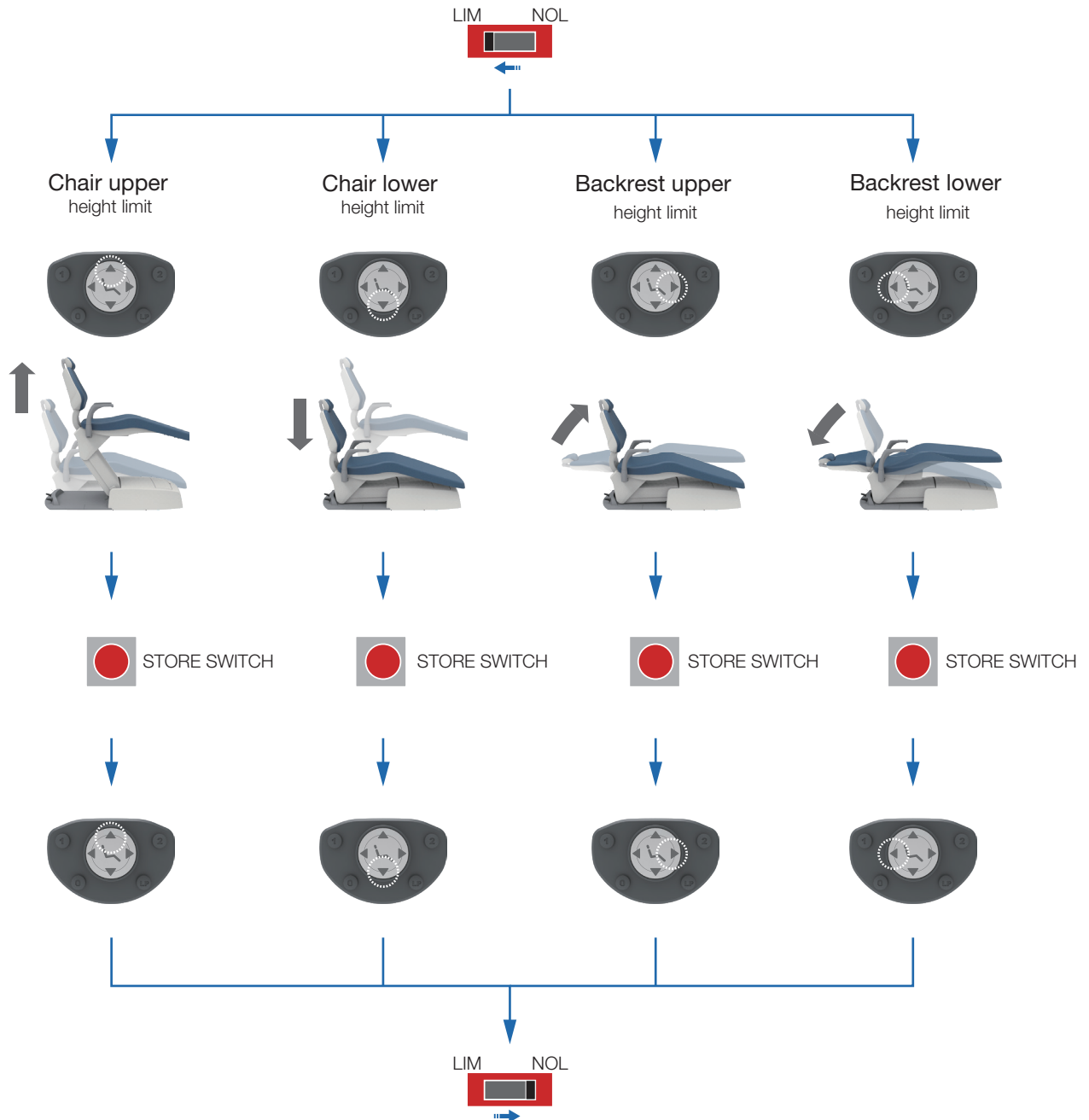



Importer in U.S.A.
BELMONT EQUIPMENT,
Division of Takara Belmont USA, Inc.

101 Belmont Drive, Somerset, New Jersey 08873 U.S.A.

TEL.: (732) 469-5000 / (800) 223-1192

FAX.: (732) 356-1035

Importer in Canada
TAKARA COMPANY, CANADA, LTD.

2455 Meadowvale Blvd. Mississauga, Ontario L5N 5S2, Canada

TEL : (905) 816-8965

www.belmont.ca

TAKARA BELMONT CORPORATION

2-1-1, Higashishinsaibashi, Chuo-ku, Osaka, 542-0083, Japan

TEL : +81-6-6213-5945

FAX : +81-6-6212-3680

REF DTD-007

