









Refer to Characteristic Drawing of Emission for Cathode

1 4 . Supplied Voltage of Primary Side for 50-100kV Output

About 150 Vp (PWM)

1 5 . Weight of X-ray Generator

About 7.13 kg



3.7kgf.












MAXIMUM RATING CHARTS (ABSOLUTE MAXIMUM RATING CHARTS)

DC


FOCAL SPOT : 0.5 mm

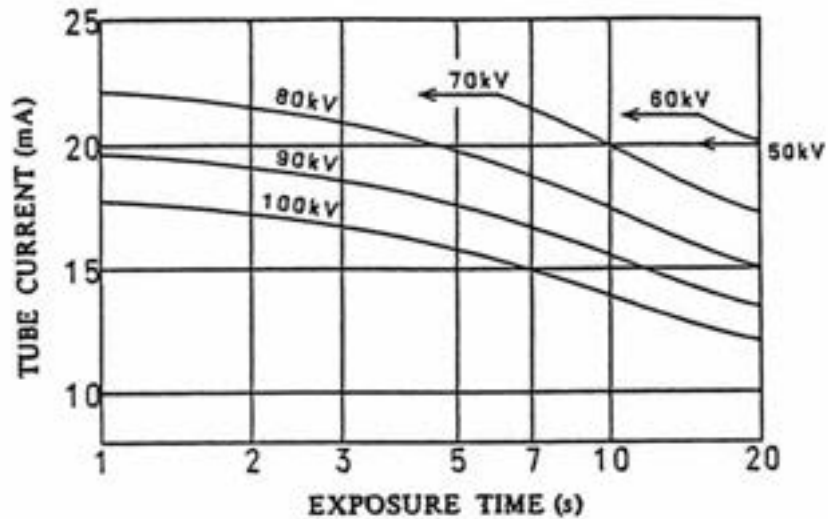


EMISSION & FILAMENT CHARACTERISTICS

DC

FOCAL SPOT : 0.5 mm

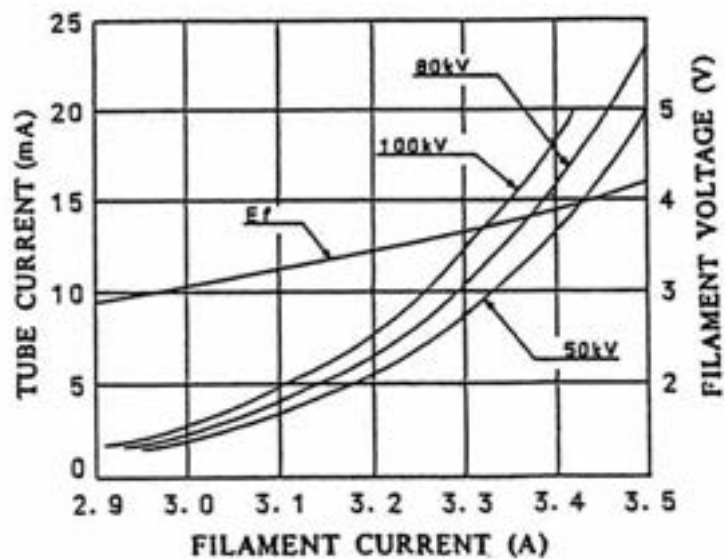

ANODE THERMAL CHARACTERISTICS

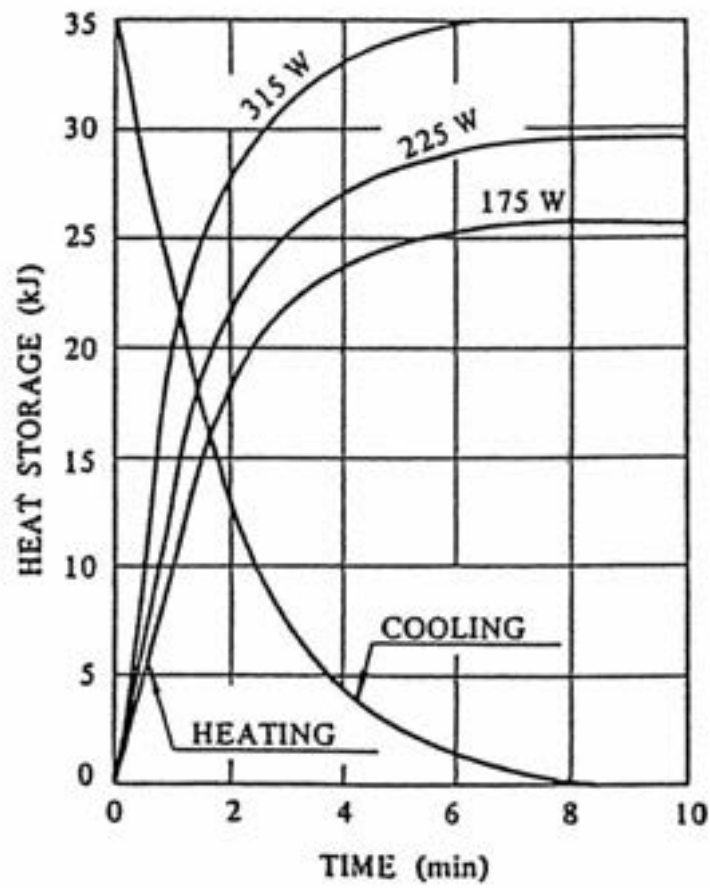












COLLIMATOR OF ANA-BEL CM

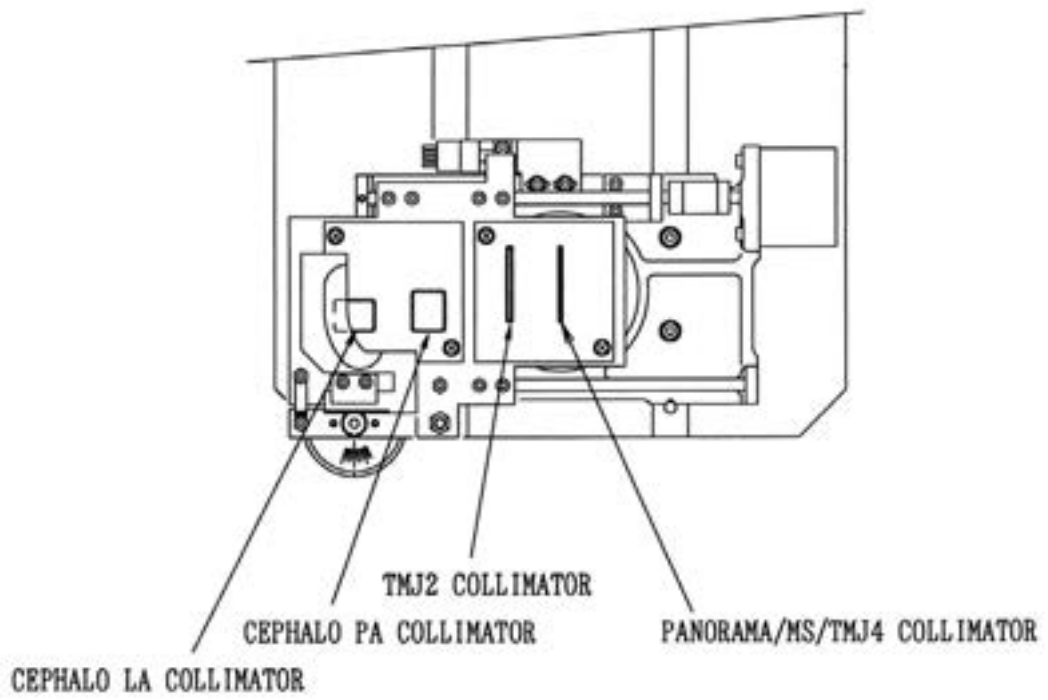

Shape of COLLIMATOR	Exposure on Film	Width of X-RAY Beam on Film
<div data-bbox="217 627 424 828" style="border: 1px solid black; width: 130px; height: 90px; margin: 10px auto; display: flex; justify-content: space-around; align-items: center;"> <div style="width: 10px; height: 30px; background-color: black;"></div> <div style="width: 10px; height: 30px; background-color: black;"></div> </div> <div data-bbox="475 331 574 425" style="margin-left: 10px;"> PANORAMA MS TMJ LA </div> <div data-bbox="507 519 641 707" style="margin-left: 10px;"> </div>	<div data-bbox="740 349 1066 568" style="margin-bottom: 20px;"> </div> <div data-bbox="740 649 1066 869" style="margin-bottom: 20px;"> </div> <div data-bbox="740 949 1066 1169" style="margin-bottom: 20px;"> </div>	<div data-bbox="1177 488 1324 752" style="margin-bottom: 20px;"> </div> <div data-bbox="1177 936 1324 1178" style="margin-bottom: 20px;"> </div>
<div data-bbox="217 1402 424 1603" style="border: 1px solid black; width: 130px; height: 90px; margin: 10px auto; display: flex; justify-content: space-around; align-items: center;"> <div style="width: 20px; height: 15px; background-color: black;"></div> <div style="width: 20px; height: 15px; background-color: black;"></div> </div> <div data-bbox="475 1227 596 1254" style="margin-left: 10px;"> CEPHALO PA </div> <div data-bbox="485 1285 647 1447" style="margin-left: 10px;"> </div> <div data-bbox="475 1523 596 1550" style="margin-left: 10px;"> CEPHALO LA </div> <div data-bbox="485 1594 625 1783" style="margin-left: 10px;"> </div>	<div data-bbox="912 1223 1190 1491" style="margin-bottom: 20px;"> </div> <div data-bbox="912 1523 1206 1792" style="margin-bottom: 20px;"> </div>	<div data-bbox="1177 936 1324 1178" style="margin-bottom: 20px;"> </div>

(mm)



