

DENTAL LIGHT

D-Lux

Model AL-701M

Model AL-702M

Model AL-705M

Parts

 **Belmont**[®]


No.	PARTS No.	DESCRIPTION	QTY
1	CADN16A0	Light Bracket Frame	1
2	CADN17A0	Yoke	1
3	CADN18A0	Brakcet Cover	1
4	CADN19A0	Yoke Cover	1
5	CADN20A0	Yoke End Cap	2
6	CADN21A0	Socket Housing	1
7	CADN22A0	Lamp Cap Bracket	1
8	CADN23A0	Lamp Cap	1
9	CADN24A0	Reflector	1
10	CADN25A0	Front Shield	1
11	CADN26A0	Lamp Holder	1
11	CGDN38A0	Lamp Holder (USA)	1
13	CADN28A0	Lamp Inner Cap	1
14	CADN29A0	Halogen Lamp (12V 55WD)	1
15	CADN30A0	Yoke Spring (S)	1
16	CADN31A0	Yoke Spring (L)	1
17	CADN32A0	Lamp Spring	1
18	CADN33A0	Yoke Shaft (Left)	1
19	CADN34A0	Mirror Holder	1
20	CADN35A0	Mirror Fixing Nut	1

No.	PARTS No.	DESCRIPTION	QTY
21	CADN36A0	Yoke Washer	1
22	CADN37A0	Thrust Washer	2
23	CADN38A0	Yoke Packing	2
24	CADN39A0	Mirror Packing	2
25	CADN40A0	Bracket Cover Screw	1
26	CADN41A0	DU Bush (DUF1410-22)	1
27	CADN42A0	Yoke Shaft (Right)	1
28	CADN43A0	Yoke Rotation Shaft	1
29	CADN52A0	Sensor Fixing Plate	2
30	CADN55A0	Sensor Frame	1
31	CADN56A0	LED PCB Base	1
32	CADN57A0	Momentary Switch Cover	1
33	CADN58A0	Sesnor Switch	1
34	CADN60A0	Handle Right	1
35	CADN61A0	Handle Left	1
36	CADN83A0	Yoke Rotation Washer	1
37	CGCN37A0	LED PCB	1
38	CADN44A0	Manual Switch Frame	1
39	CGCN38A0	Manual Switch (S-303)	1
	CGCN97A0	Patient Mirror Set	(1)

No.	PARTS No.	DESCRIPTION	QTY
40	CGCN89A0	Patient Mirror Base & Stay	1
41	CADN79A0	Matient Mirror Frame	1
42	CADN80A0	Patient Mirror	1
44	CADN82A0	Patient Mirror Holding Spring	1
45	99O00001	Screw (M4-25)	1
46	99O00002	Cap Bolt (M5-42)	1
47	99O00003	Nut (M4)	2
48	99O00004	Flat Washer (5-10-0.8)	3
49	CADP11A0	Manual Switch Bracket	1
50	CACP34A0	Logo Sticker	1
51	CASN96AR	Stop Screw	1
52	CECN45A0	Tapping Screw (M3-6)	2
53	CECN46A0	Tapping Screw (M3-8)	7
54	CEDN01A0	Manual Switch (S-302)	1
55	99O00005	Screw (M3-5)	1
56	99O00006	Painted Screw (M3-6)	4
57	99O00007	Set Screw (M3-6)	1
58	99O00008	Screw (M3-10)	2
59	99O00009	Screw (M3-12)	2
60	99O00010	Screw (M3-12)	5

No.	PARTS No.	DESCRIPTION	QTY
61	99O00011	Screw (M4-12)	4
62	99O00012	Flat Washer (3-6-0.5)	2
63	99O00013	Nylon Washer (3.85-10-0.8)	1
64	99O00014	HIZ Washer (3.8-10-1)	1
65	CGEN03A0	Wire Harness & Connector Set (S Type)	1
	CADN65A0	Wire Harness 450mm & 3P Connector (Black/Brown/Red)	(1)
	CADN66A0	Heat Resisting Wire 380mm (White/Yellow)	(1)
	CADN67A0	Heat Resisting Wire 390mm (White)	(1)
	CATB01A0	2P Connector Female (Square Hole)	(1)
66	CGEN04A0	Wire Harness & Connector Set (M Type)	1
	CADN67A0	Heat Resisting Wire 390mm (White)	(1)
	CADN68A0	Heat Resisting Wire 50mm (White/Yellow)	(1)
	CADN69A0	Heat Resisting Wire 370mm (White/Yellow)	(1)
	CADN73A0	Heat Resisting Wire 370mm (White)	(1)
	CADP06A0	Wire Harness 90mm (Orange/Red)	(1)
	CADP07A0	Wire Harness 90mm (Black)	(1)
	CATB01A0	2P Connector Female (Square Hole)	(1)

HEAD SECTION for 720

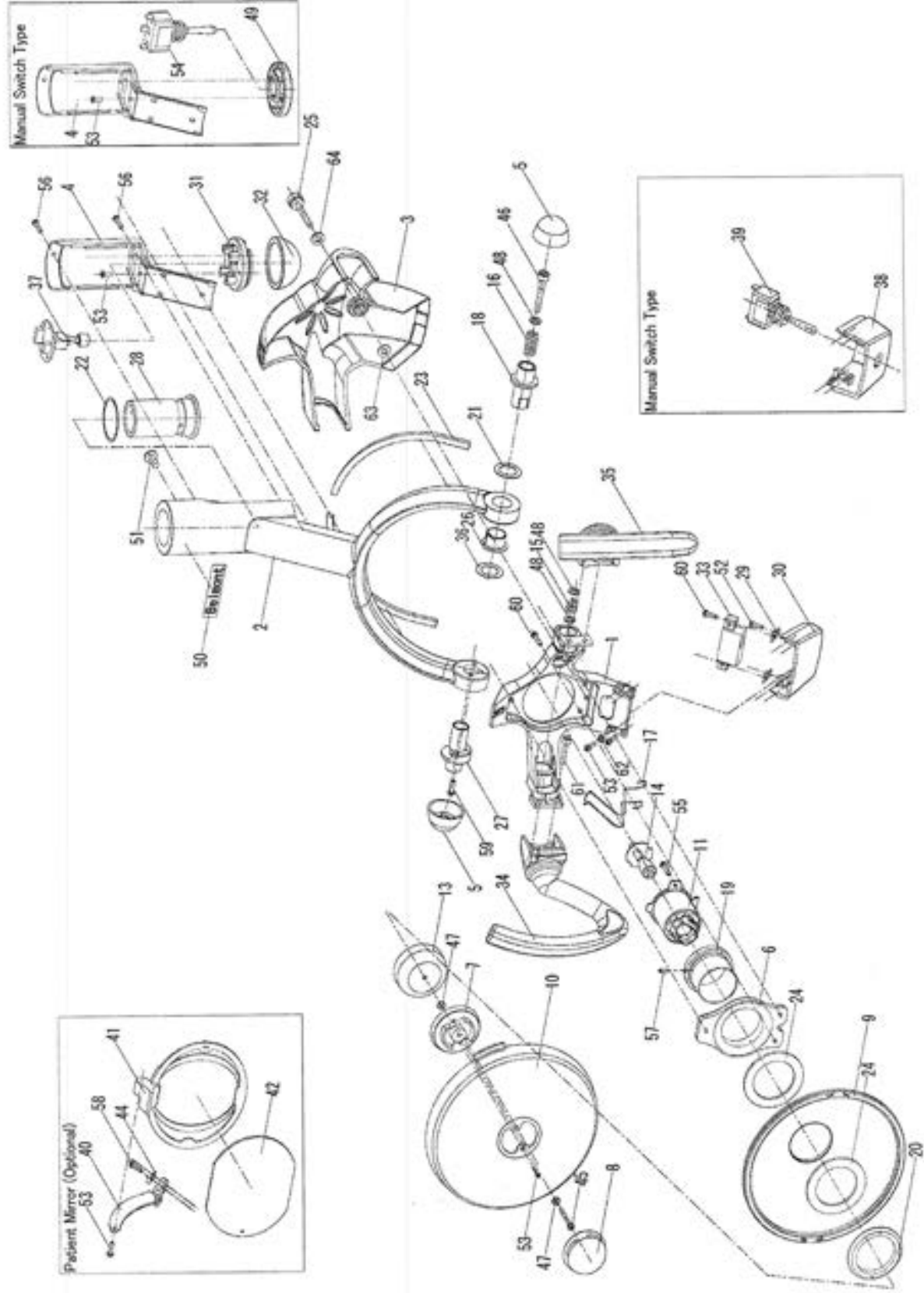


No.	PARTS No.	DESCRIPTION	QTY
	CGEN32A0	Balance Arm Assembly for 701M	(1)
	CGCN96AR	Balance Arm Assembly for 720S	(1)
	CGDN17AR	Balance Arm Assembly for 720M	(1)
1	CACN87A0	Yoke Joint	1
2	CACN88A0	Arm Joint	1
3	CACN89A0	Adjusting Joint	1
4	CACN90A0	Adjusting Rod Joint	1
5	CACN91A0	H-Bracket	1
6	CACN93A0	Balance Arm Spring	1
7	CACN94A0	Spring Nut	1
8	CACN95A0	Arm Nut	1
9	CACN96A0	Arm Spring Rod	1
10	CACN97A0	Rotation Pin	1
11	CACN98A0	Link Pin	2
12	CACN99A0	Arm Pipe (L)	1
13	CADN00A0	Arm Pipe (S)	1
14	CADN01A0	Brass Washer	1
15	CADN02A0	Washer (70W-1015)	1
16	CADN03A0	Spring Lock	2
17	CADN04A0	Brake Shoe	4

No.	PARTS No.	DESCRIPTION	QTY
18	CADN05A0	Brake Spring	2
19	CADN06A0	Balance Arm Cover	1
21	CADN08A0	Balance Arm Frame	1
22	CADN62A0	Slide Washer (7.2-14-0.25)	4
23	CADN10A0	Joint Cover Plate (Upper)	1
24	CADN12A0	Rubber Cover	1
25	CGEN34A0	Light Cable for 701M	1
25	CADN13A0	Light Cable for 720S	1
25	CGDN18A0	Light Cable for 720M	1
26	CADN84A0	Washer (DUT-10)	1
27	CADN85A0	Spring Stopper	1
28	CAJA05A0	Yoke Joint Bush	1
29	CAMP65A0	Stopper (L)	4
31	99O00101	Washer (12.5-22-2)	1
32	CANF62A0	Slide Washer (9.1-4-0.25)	4
33	CADN91A0	Spring Pipe	1
34	CADN09A0	Slide Washer (8.2-14-0.25)	4
35	99O00102	Tapping Screw (M3-6)	2
36	99O00103	Painted Screw (M3-8)	2
37	99O00104	Tapping Screw (M3-12)	1

No.	PARTS No.	DESCRIPTION	QTY
38	99O00105	Painted Tapping Screw (M3-12)	1
39	99O00106	Screw (M4-8)	8
40	99O00107	Screw (M4-10)	4
41	99O00108	Toothed Set Screw (M4-5)	2
42	99O00109	Set Screw (M4-5)	4
43	99O00110	Set Screw (M4-8)	2
44	99O00111	Set Screw (M4-12)	1
45	99O00112	Set Screw (M4-14)	4
46	99O00113	Painted Round Head Screw (M5-1)	2
47	99O00114	Countersink Screw (M8-18)	1
48	99O00115	Countersink Screw (M8-20)	1
49	CACN92AR	Pole Joint	1
50	99O00116	Countersink Screw (M8-16)	2
51	99O00117	Bolt (M4-15)	1
52	99O00118	Painted Round Head Screw (M6-8)	2

ARM SECTION for 720

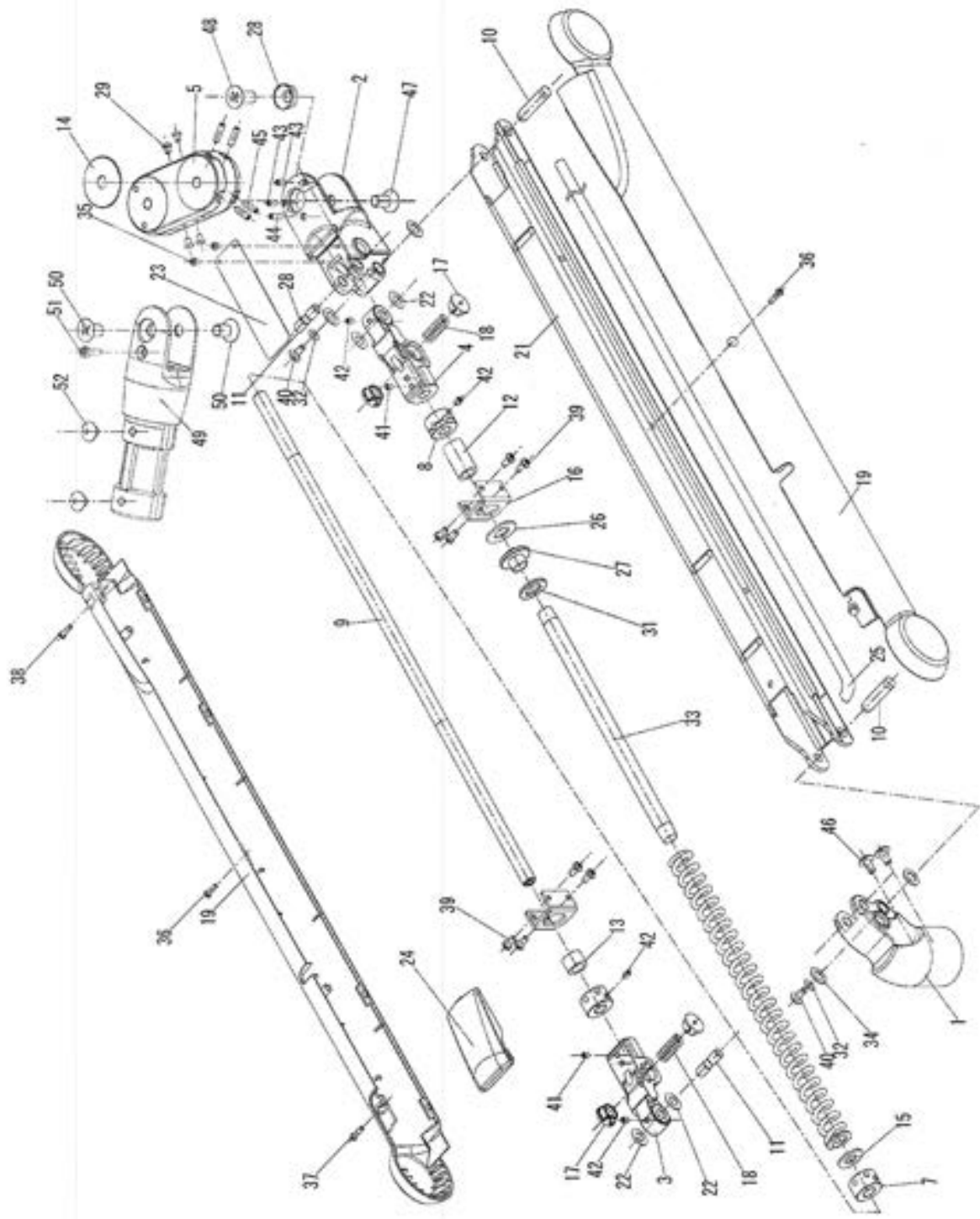


TB SECTION for 701M

No.	PARTS No.	DESCRIPTION	QTY
1	CADN70B0	Transformer DL02 (100V-120V)	1
2	CADQ25A0	Transformer Housing for 701M	1
3	CGEN37A0	Transformer Housing Cover (Lower)	1
4	CGEN36A0	Transformer Housing Cover (Upper)	1
5	CADQ28A0	Transformer Housing Cover (Front)	1
6	CGEN39A0	Transformer Housing Cover (Back)	1
7	CADQ30A0	Horizontal Arm	1
8	CADQ32A0	Horizontal Arm Bracket	1
9	CADQ33A0	Arm Support Bracket	1
10	CADQ34A0	Horizontal Arm Support	1
11	CADQ35A0	Washer for CT-Cable Bush	1
12	CACN91A0	H-Bracket	1
13	CADQ98A0	Horizontal Arm Joint	1
14	CAML90A0	Terminal Block	1
15	CAOD05B0	Pole Adapter	1
16	CEEN22A0	Fuse 1.5A	2
17	CENX56B0	Main Swith	1
18	CENZ25A0	CT-Cable Bush	1
19	CERN52A0	Fuse Holder	2
20	99O00201	Bolt (M6-14)	4

TB SECTION for 701M

No.	PARTS No.	DESCRIPTION	QTY
21	99O00202	Bolt (M8-20)	2
22	99O00203	Round Head Screw (M8-25)	4
23	99O00204	Bolt (M8-30)	1
24	99O00118	Painted Round Head Screw (M6-8)	2

TRANSBOX SECTION for 701M

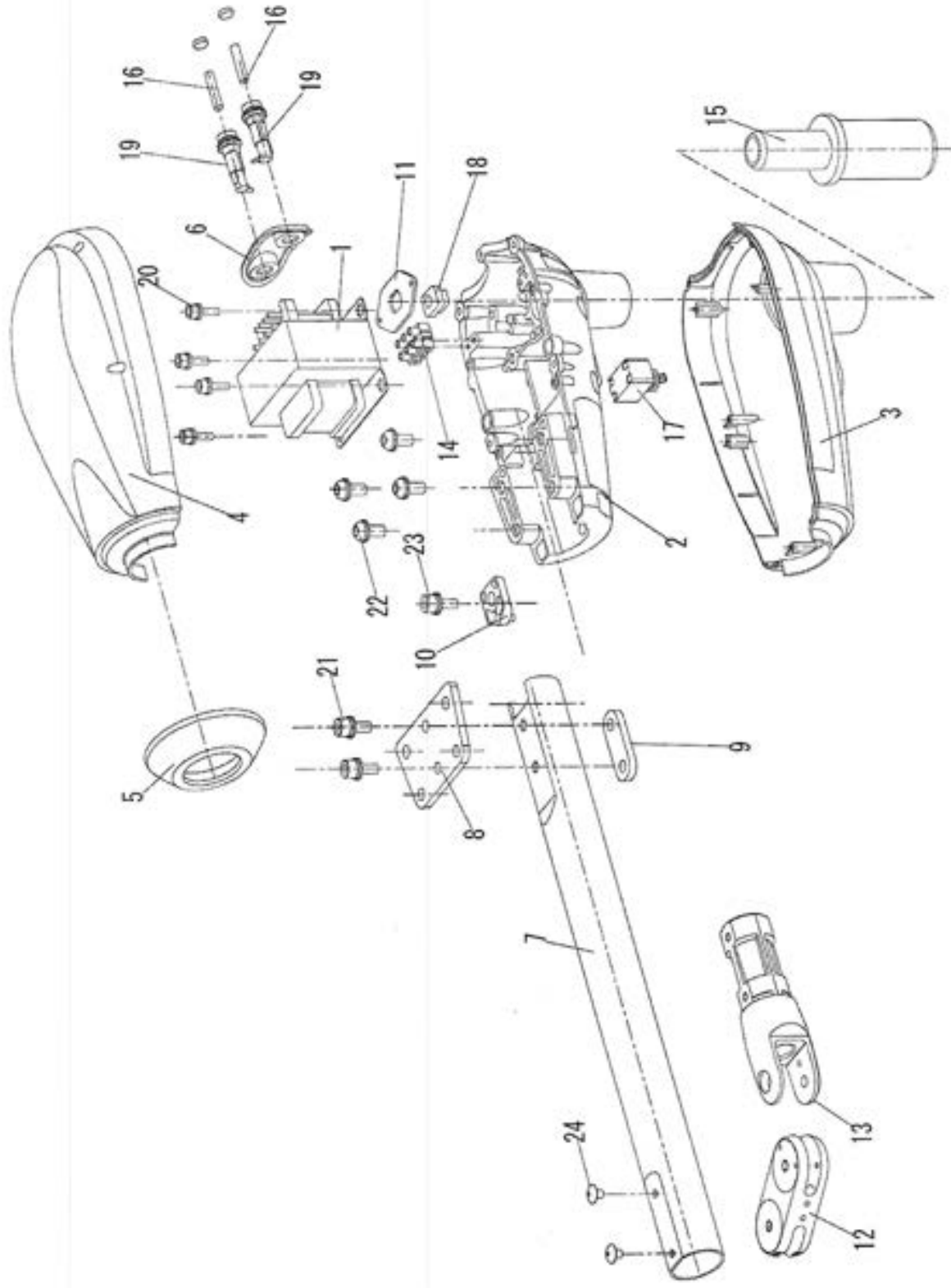


TB SECTION for 702M

No.	PARTS No.	DESCRIPTION	QTY
1	CADN70B0	Transformer DL02 (100V-120V)	1
2	CGEN38A0	Transformer Housing Cover (Lower)	1
3	CADQ27A0	Transformer Housing Cover (Upper)	1
4	CADQ28A0	Transformer Housing Cover (Front)	1
5	CGEN39A0	Transformer Housing Cover (Back)	1
6	CADQ30A0	Horizontal Arm	1
7	CADQ32A0	Horizontal Arm Bracket	1
8	CADQ33A0	Arm Support Bracket	1
9	CADQ34A0	Horizontal Arm Support	1
10	CADQ35A0	Washer for CT-Cable Bush	1
11	CADQ95A0	Transformer Housing for 702M	1
12	CACN91A0	H-Bracket	1
13	CADQ98A0	Horizontal Arm Joint	1
14	CADR34A0	Stop Screw	1
15	CAML90A0	Terminal Block	1
16	CAMK35A0	Transformer Housing Shaft	1
17	CAND32A0	Washer (43-29-0.8)	1
18	CEEN22A0	Fuse 1.5A	2
19	CENX56B0	Main Switch	1
20	CENX94A0	C-Ring	1

TB SECTION for 702M

No.	PARTS No.	DESCRIPTION	QTY
21	CENZ25A0	CT-Cable Bush	1
22	CERN52A0	Fuse Holder	2
23	99O00201	Bolt (M6-14)	4
24	99O00202	Bolt (M8-20)	2
25	99O00203	Round Head Screw (M8-25)	4
26	99O00204	Bolt (M8-30)	1
27	99O00205	Spring Washer (M4)	1


TRANSBOX SECTION for 702M

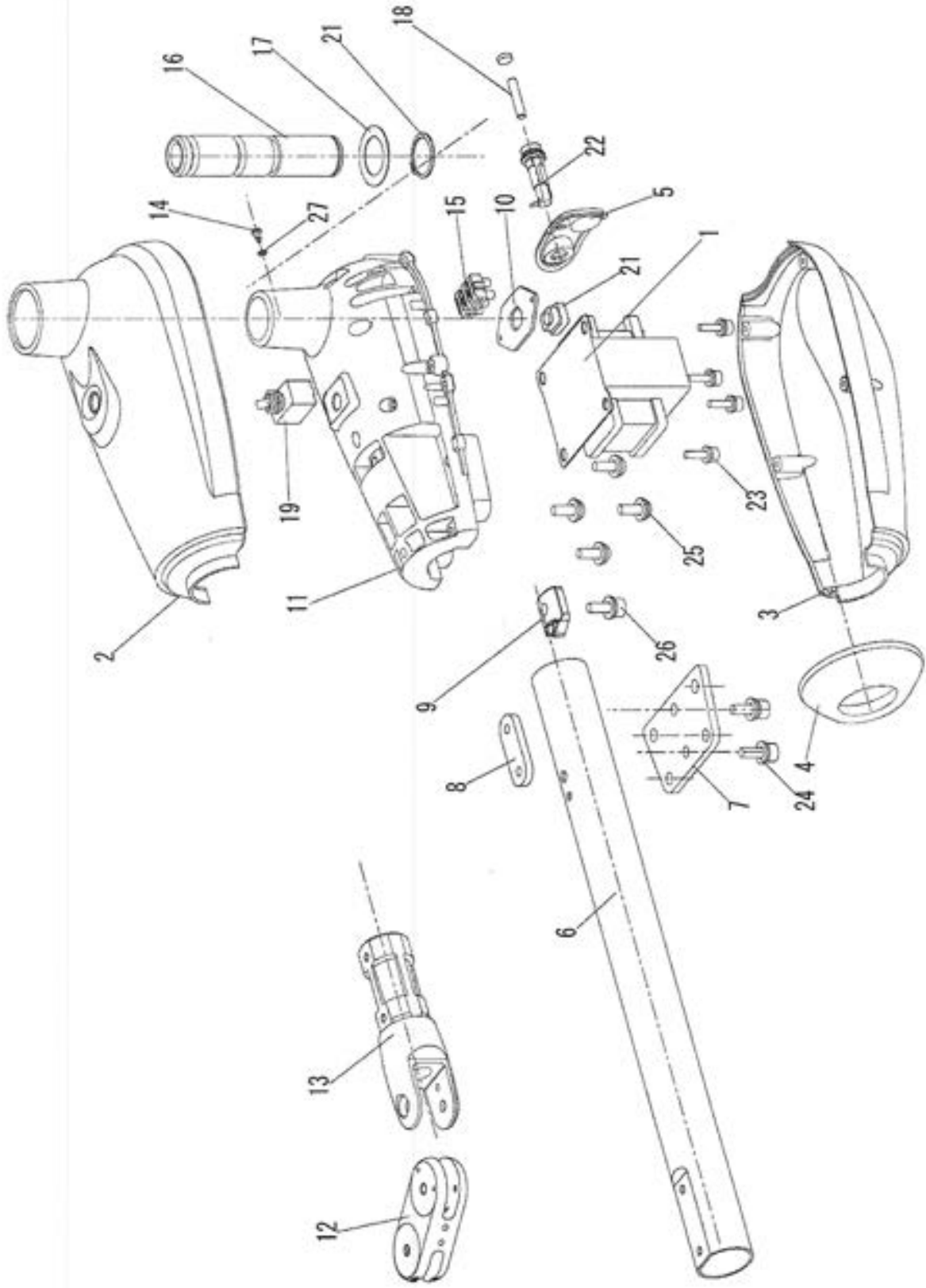

CEILING SECTION for 702M

No.	PARTS No.	DESCRIPTION	QTY
	CGQO13A0	Ceiling Flange Set	(1)
1	CAMI08A0	Mounting Plate with Nut (M8)	1
2	99O00301	Nut (M8)	4
3	TCPA01A0	Ceiling Flange	1
4	99O00302	Washer (M8)	4
5	99O00303	Spring Washer (M8)	4
6	99O00304	Nut (M8)	4
7	CAMK40A0	Roll Pin	1
8	HAMM11B0	Flange Cover	1
9	CAMG77AR	Ceiling Pole Set 340mm (Ral-9003) (8ft)	1
9	CAMG80AR	Ceiling Pole Set 680mm (Ral-9003) (9ft)	1
9	CAMG81AR	Ceiling Pole Set 1000mm (Ral-9003) (10ft)	1
10	CAMG76A0	Stopper Ring	1

CEILING SECTION for 702

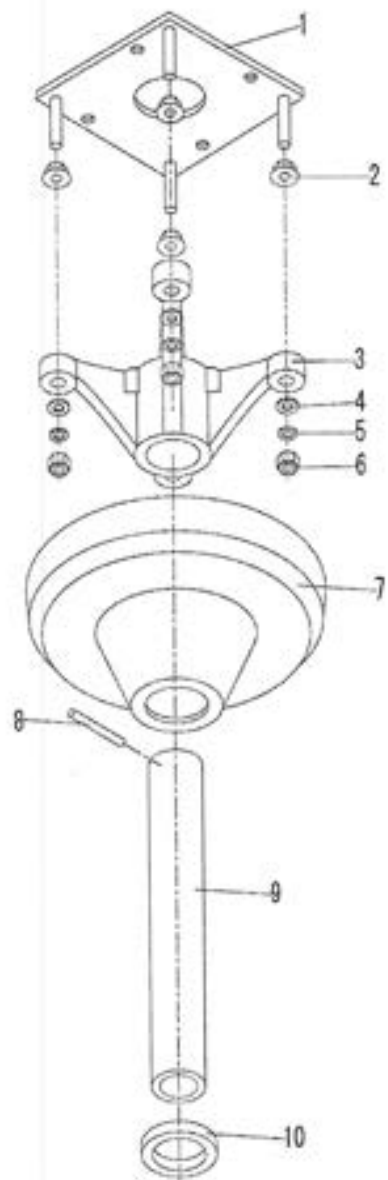


ARM SECTION for 705M

No.	PARTS No.	DESCRIPTION	QTY
	CGEN33A0	Balance Arm Assembly for 705M	(1)
1	CACN87A0	Yoke Joint	1
2	CACN89A0	Adjusting Joint	1
3	CACN90A0	Adjusting Rod Joint	1
4	CACN93A0	Balance Arm Spring	1
5	CACN94A0	Spring Nut	1
6	CACN95A0	Arm Nut	1
7	CACN96A0	Arm Spring Rod	1
8	CACN97A0	Rotation Pin	1
9	CACN98A0	Link Pin	2
10	CACN99A0	Arm Pipe (L)	1
11	CADN00A0	Arm Pipe (S)	1
12	CADN02A0	Washer (70W-1015)	1
13	CADN03A0	Spring Lock	2
14	CADN04A0	Brake Shoe	4
15	CADN05A0	Brake Spring	2
16	CADN06A0	Balance Arm Cover (Flame retardar	1
18	CADN08A0	Balance Arm Frame	1
19	CADN09A0	Slide Washer (8.2-14-0.25)	4
20	CADN10A0	Joint Cover Plate (Upper)	1

ARM SECTION for 705M

No.	PARTS No.	DESCRIPTION	QTY
21	CADN62A0	Slide Washer (7.2-14-0.25)	4
22	CADN84A0	Washer (DUT-10)	1
23	CADN85A0	Spring Stopper	1
24	CADN91A0	Spring Pipe	1
25	CADP65A0	Arm Joint	1
26	CAMG97A0	Arm Joint Shaft	1
27	99O00101	Washer (12.5-22-2)	1
28	CENX95A0	C-Ring	1
29	CENX96A0	Wave Washer	1
30	CADP38A0	Extension Tube	1
31	CADN12A0	Rubber Cover	1
32	CGEN35A0	Light Cable for 705M	1

ARM SECTION for 705M

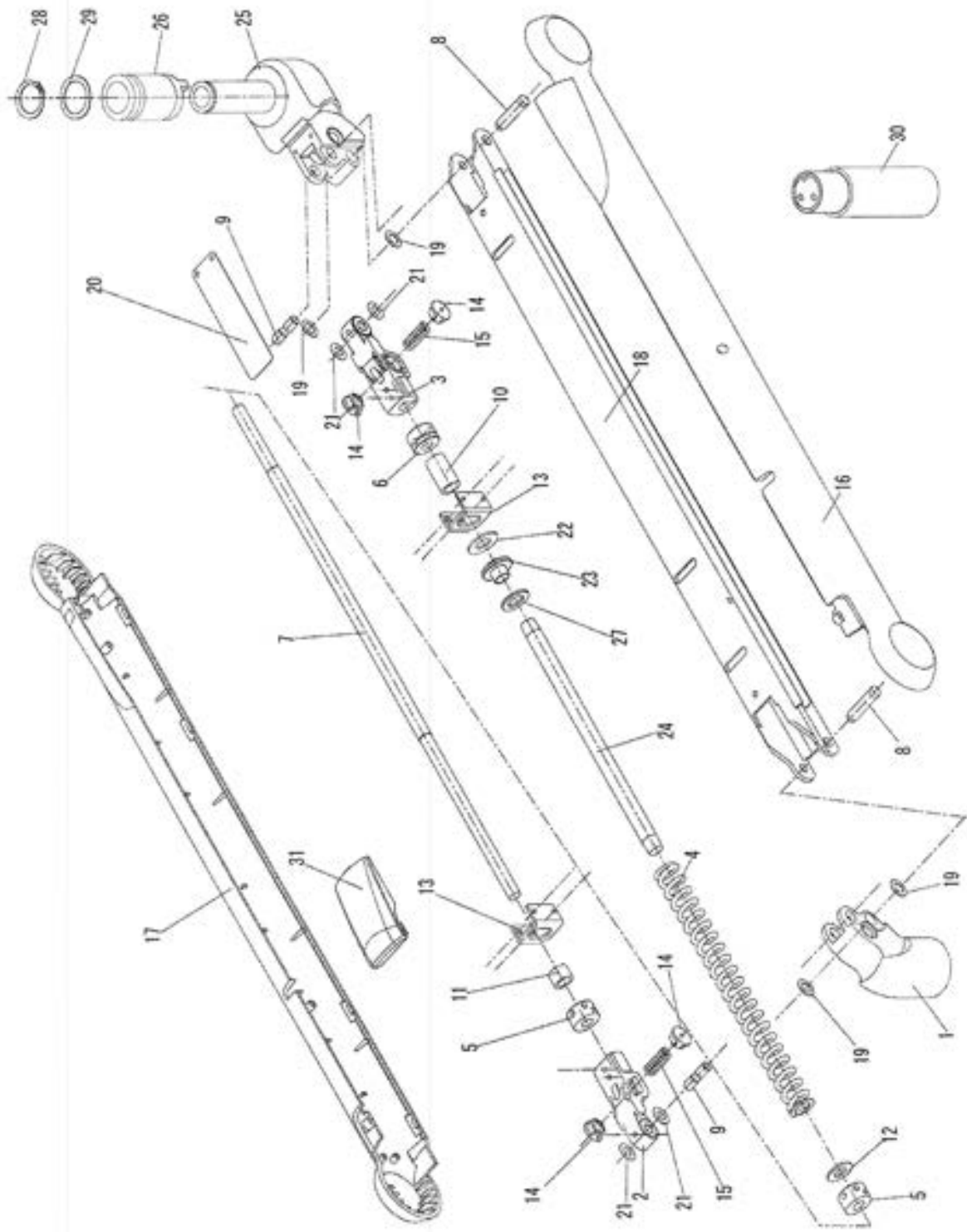

TB SECTION for 705M

No.	PARTS No.	DESCRIPTION	QTY
1	CADN70B0	Transformer DL02 (100V-120V)	1
2	CGEN40A0	Transformer Housing for 705M	1
3	CADR17A0	Transformer Plate	1
4	CADR18A0	Swith Bracket	1
5	CEEN22A0	Fuse 1.5A	1
6	CENX56B0	Main Swith	1
7	CERN52A0	Fuse Holder	1

TRANSBOX SECTION for 705M

